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16. USER MANUAL – ENGLISH VERSION

Please read carefully before use the following operating manual, in particular with the information contained in the sections "**Calibration**" and "**Final information**". The use in accordance with the instructions in the manual will ensure long-term and reliable operation of devices. The manual should be kept close to the device.


Read all notes preceded by this symbol carefully. Failure to follow the warnings may damage the tester.


ATTENTION! Any attempt to modify the tester carries the risk of electric shock and may prevent its proper functioning.


GENERAL APPEARANCE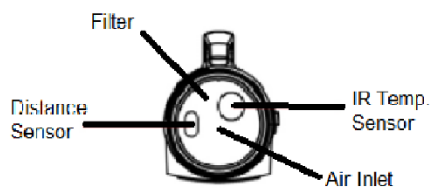

Top

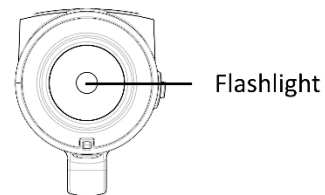Bottom

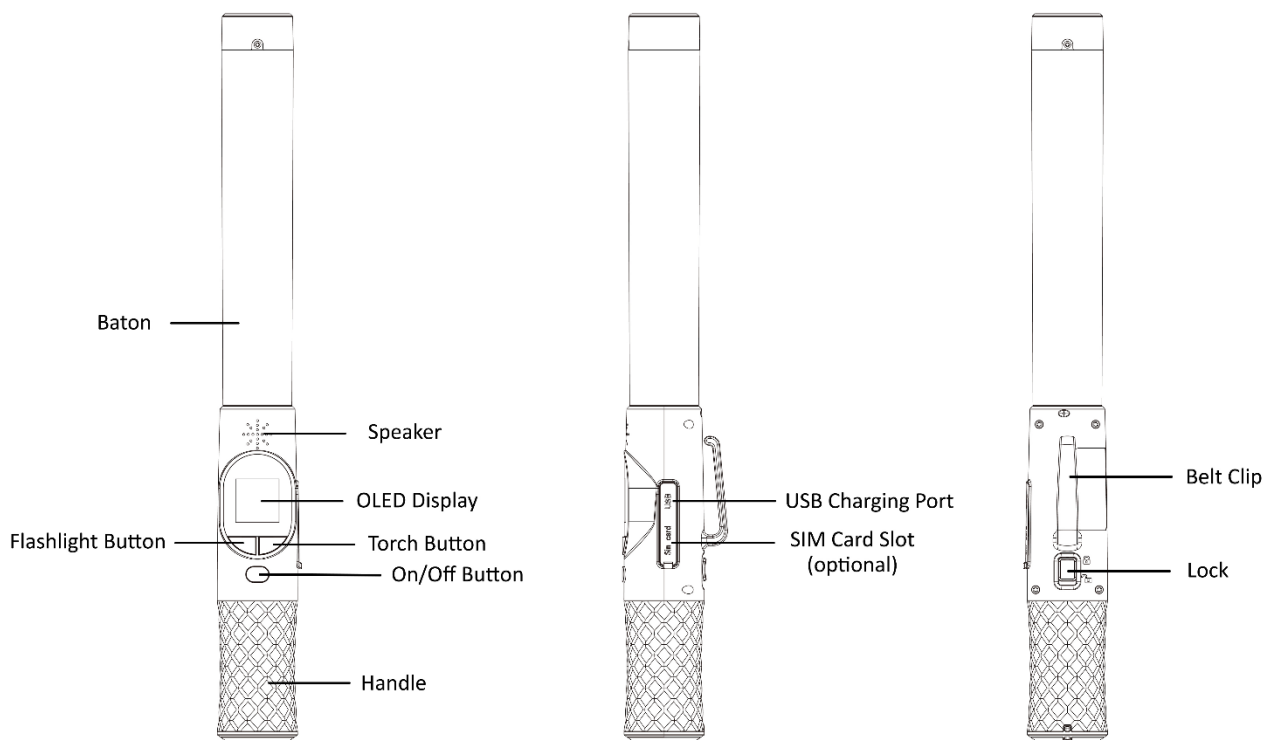POWER

The device is powered by a lithium battery: 3.7V with a capacity of 1500mAh. Before using the device for the first time, fully charge the battery. The time needed to fully charge the battery is about 3 - 4 hours. Remember that the battery only reaches full capacity after two or three cycles of complete charge and discharge.

The battery can be charged from the USB port of any device via the included cable or from an AC adapter with a miniUSB connector. Only use power adapters approved by the device manufacturer. If in doubt, please contact the service. The battery can be charged and discharged many times, but its performance decreases over time. With a discharged battery, the measurement results may be inaccurate.

IMPORTANT INFORMATION - BEFORE THE TEST

The test should be carried out no earlier than 15/20 minutes after drinking alcohol. Smoking should also be avoided for 15/20 minutes before the test.

Residues of alcohol and other factors in the oral cavity can falsify the results. Other factors include i.e. residues from: mouthwash, aromatic drinks, medicines, drops, strong mint candies, gum, vomit, etc. Before taking the measurement, it is recommended to rinse your mouth with clean water or an alcohol-free liquid. Never blow cigarette smoke into the measuring chamber.

The test should be carried out at a temperature not lower than -5°C and not higher than 50°C. Measurement within the recommended temperature range reduces the possibility of error due to condensation of water vapour from the exhaled air.

It is also recommended to store the device at room temperature when the device is not used for a longer period of time. Do not use or store the tester in places exposed to extreme temperatures. This may damage the electronic components.

It is best to take 3 measurements at equal, approximately 20-minute intervals and record the results. Comparing the recorded results will allow you to easily determine which phase of intoxication you are in and estimate when your body will cleanse itself of alcohol.

NOTE! The concentration of alcohol in your blood and exhaled air changes over time. Initially, it increases as alcohol is absorbed into your blood, then decreases as your body eliminates it. The most important information that the tester should indicate is which phase you are in: the phase of increased concentration or elimination. In order to obtain a reliable result, take measurements only when you no longer intend to consume alcohol.

TURNING THE DEVICE ON/OFF

Press and hold the power button (ON/OFF) for 2 – 3 seconds. The display will show the time until the next calibration. The device will then prepare to take the measurement, after which the message "Please blow" will appear. To turn off the appliance, press and hold the power button again (ON/OFF ; Perform the test) until POWER OFF appears on the display.

MEASUREMENT

AUTOMATIC MEASUREMENT

A measurement mode used when we want to accurately determine the alcohol content in the exhaled air.

In order to perform the test:

1. Unlock the device with the Lock button.
2. Press and hold the ON/OFF power button for 2 – 3 seconds
3. After a while, the display will show "Ready". To take a measurement, point the device so that the inlet opening is near the mouth of the test person.
4. The test person should take a deep breath and then blow evenly, not too strongly, into the mouthpiece for about 8 seconds, until the "click" signal is heard.
5. The tester then analyzes the sample and after a few seconds the measurement result is displayed.
6. After the air sample has been analyzed, the "No Alcohol" or "Alcohol Detected" test result will be displayed on the screen. If alcohol is detected, the breathalyser will also turn on the built-in warning light.
7. Press the ON/OFF button once; to move on to the next test.
8. Press and hold the ON/OFF button; for 2 – 3 seconds to turn off the device.

MANUAL MEASUREMENT

This option can be used, for example, when testing for the presence of alcohol in liquids, testing people who make it impossible, obstruct the test, are unable to blow the required amount of air or are unconscious.

In order to perform the test:

1. Unlock the device with the Lock button.
2. Press and hold the ON/OFF power button for 2 – 3 seconds.
3. After a while, the display will show "Ready". To take a measurement, point the device so that the inlet opening is near the mouth of the test person.
4. Then press the ON/OFF button once; to take an air sample. Sample collection will take about 8 seconds (the breathalyzer has the option to adjust the length of the sample collection).
5. The tester then analyzes the sample and after a few seconds the measurement result is displayed.
6. After the air sample has been analyzed, the "No Alcohol" or "Alcohol Detected" test result will be displayed on the screen. If alcohol is detected, the breathalyzer will also turn on the built-in warning light.
7. Press the ON/OFF button once; to move on to the next test.
8. Press and hold the ON/OFF button; for 2 – 3 seconds to turn off the device.

The results of measurements taken in manual mode are indicative and may not be precise.

RE-MEASUREMENT

If the alcohol concentration level in the previous measurement was high, despite the device being ready for measurement, the remaining alcohol vapors may not have completely evaporated from the measuring cell.


In case of a positive test, it is recommended to keep 2 - 6 minutes. space before the next measurement. This time is necessary for the alcohol to completely evaporate from the sensor, which could inflate the result of the next measurement. The time of evaporation depends on the alcohol concentration in the previous test.

If the test result was negative (reading 0.00 ‰), the next measurement can be performed immediately.

AUTOMATIC SHUT-OFF

When the device is not used for 5 minutes, the power is automatically disconnected to save battery life.

MENU

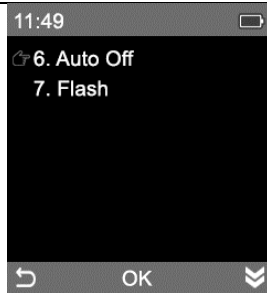1. Turn on the device and press the left function button to enter the home screen. 2. Navigate the menu with three buttons Left function button – return to previous Screen Right function button – move down Middle button – confirm the selection		
---	--	---

1. Records

Up to 8,000 measurement results can be stored in the device's memory with the date and time of measurement, concentration and the selected measurement mode.

1. Navigate through the menu, select "records" and confirm with the middle button. 2. The measurement number, test date and time, and measurement result will appear on the screen.	
3. Press the right function button to go to the previous result. 4. Press the left function button to return to the previous screen. 5. The device can store 8,000 results, 8,001 result will replace 1, 8,002 result will replace 2, [...] etc. 6. After calibration, all results will be erased.	

2. Device info

To check the serial number of the device, the software version and the date of the last calibration in yyyy/mm/dd format.

3. Calibration / 4. Setting

Service functions. Calibration and settings are password protected and must only be operated by a trained operator.

5. Date & Time

Set the date and time.

6. Auto Off

Set whether automatic shutdown of the device is needed, the default automatic shutdown time is 5 minutes.

7. Flash

Choose whether to have the warning light on when the test result is above the alarm level.

FLASHLIGHT FUNCTION

The BACscan® A8, thanks to the mounting of LEDs in the device, it can be used as a 3-step flashlight. To turn on the LEDs, press the "Flashlight activation button". Pressing this button again changes the type of emitted light according to the sequence: upper light, lower light, upper & lower light.

WARNING LIGHT FUNCTION

Thanks to the integrated warning light, the BACscan® A8 can be used, for example, as a hand-held siren for uniformed services. To turn on the warning light, press the "Warning light button".

CALIBRATION

Calibration applies to EVERY BREATHALYZER without exception.

The accuracy of the tester's readings may decrease during normal use. Every specified time/number of measurements the tester must be reconfigured to correctly interpret the readings of the aging sensor. The need for calibration is indicated by appropriate messages and results that significantly differ from reality - e.g. 0.3‰ despite no alcohol consumption.

The need for calibration is not a defect of the breathalyzer and is generally not subject to complaint.

It should not be expected that a breathalyzer that is not calibrated regularly and/or not used will indicate correct results. Calibrations are subject to a fee and can be performed at the AISKO Central Service (also by mail) or at another authorized breathalyzer service point. **More information is available by phone at +48 22 852 23 98, on the Alkomat.pl website and in a separate Warranty Card attached to the product.**

The accuracy of the tester indications may decline during normal use. According to the manufacturer's guidelines, in order to ensure accurate indications, it is recommended to calibrate the device at least every 6-12 months or 1000 measurements (whichever comes first). Calibration is performed against payment and may only be performed by an Authorized Service Center.

FINAL INFORMATION

- **Never take the measurement immediately after drinking alcohol. High concentrations can decalibrate or damage the measuring cell!**
- **Do not blow cigarette smoke into the device as this may damage the measuring cell!**
- Blowing too much may damage the sensor as a result of spilling it with saliva.
- If the temperature of the device differs significantly from the temperature of the room in which it is to be used, wait approx. 15 - 20 minutes for the device to adjust to the new temperature.
- Only use original mouthpieces for precise results. The mouthpieces are individually packed in hygienic, sterile packaging.
- Store the tester and accessories out of the reach of children.
- Do not use or store the tester in places exposed to both low and high temperatures. This can damage electronic components.
- Do not use or store the tester in places subject to excessive contamination (e.g. toxic gas vapors, gasoline, solvents, exhaust fumes, etc.) and / or high humidity.
- Do not use the device near strong electric or magnetic fields, e.g. near cordless phones, cell phones, etc.
- Do not allow the device to become flooded. If it becomes wet or wet, please dry it immediately.

- Do not use any chemicals to clean the tester. Wipe the device with a soft cloth.
- If the device has not been used for a long time, the first three results may have a greater measurement error.
- It is recommended to check and calibrate the breathalyzer regularly.

The tester cannot be used as a tool to unambiguously determine whether the tested person can drive a motor vehicle in accordance with applicable law. The manufacturer and the seller do not bear any responsibility for any damage or loss caused or incurred by the person tested with this tester. Incorrect indications may be the result of improper operation of the device, damage or the influence of many other factors. All decisions and actions based on the results of this device will be the sole responsibility of the tested person.

According to the law, this tester is not a proof device if it does not have a calibration certificate from an accredited laboratory.

Information for users on the disposal of electrical and electronic devices (for households)

The symbol of the crossed out waste container placed on the equipment means that the used equipment should not be placed in containers together with other household waste. Hazardous components contained in electrical and electronic equipment may cause long-term adverse changes in the natural environment, as well as be harmful to human health. The user who intends to get rid of the used device is obliged to hand it over to the collector of waste equipment. When buying new equipment, you can hand over the old equipment of the same type and function to the dealer. Waste equipment can also be delivered to collection points, the addresses of which are available on the websites of municipalities or at the offices of the offices. The household plays a very important role in the proper management of waste electrical and electronic equipment. Handover of waste electrical and electronic equipment to collection points contributes to the reuse, recycling or recovery of equipment and environmental protection.

THE MANUFACTURER, IMPORTER OR DISTRIBUTOR WILL NOT BE LIABLE FOR ANY MATERIAL AND INTANGIBLE DAMAGES CAUSED BY THE FAILURE OF THE METER AND THE HANDLING OF THE DEVICE NOT IN COMPLIANCE WITH THE USER'S MANUAL.

No part of this manual, whether in whole or in part, may be modified or processed in any electronic, photographic, mechanical or other way without the written consent of AISKO.