VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate www.vevor.com/support

Fold Down Shower Seat

MODEL: LY010

We continue to be committed to provide you tools with competitive price.

"Save Half", "Half Price" or any other similar expressions used by us only represents an estimate of savings you might benefit from buying certain tools with us compared to the major top brands and does not necessarily mean to cover all categories of tools offered by us. You are kindly reminded to verify carefully when you are placing an order with us if you are actually saving half in comparison with the top major brands.

VEVOR[®]

TOUGH TOOLS, HALF PRICE

Fold Down Shower Seat

MODEL: LY001

NEED HELP? CONTACT US!


Have product questions? Need technical support? Please feel free to contact us:

Technical Support and E-Warranty Certificate
www.vevor.com/support

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

Cautions

- 1: Please confirm whether the wall is a load-bearing pond before drilling;
- 2: Before drilling, please confirm whether there is a water pipe at the hole position;
- 3: Check whether the product accessories are complete before installation;
- 4: Make sure the drill bit is 8.5mm in size.
- 5: Make sure the hole depth is 6.5cm
- 6: The recommended height between the seat and ground is about 40-48cm.

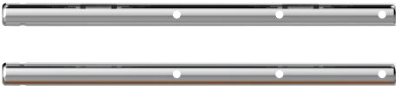Part List			
NO	Name	Pictures	QTY.
1	Main Body		3
2	Decorative cover		2
3	The load-bearing stick		2
4	Mounting plate		2
5	Countersunk screw M6*8		6+2
6	Nylon Bolt Cap		10+2
7	Screw ST8*70mm		10+2


8	Mounting Hole Guiding Sticker		1
9	Drill Bit φ 8.5*150		1
Tip: “+” is extra spare parts.			

Model	LY010
Specification（inch）	20*14
Max. Load (lbs/kg)	440/200

INSTALLATION


STEP1:

Arrange the planks in order, insert the load-bearing sticks into the planks and fix them with countersunk screws.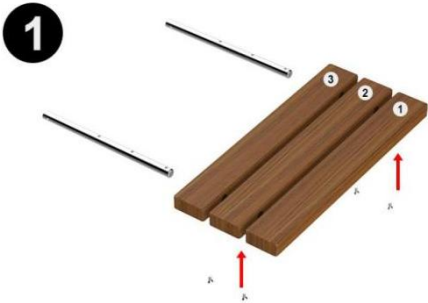

STEP2:


Measure the distance and attach the locator card to the wall.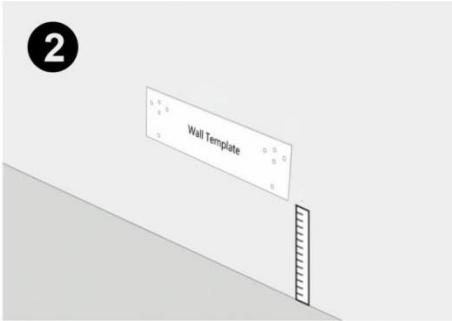

STEP3:

Drill holes in the marked points of the locator card.

(Drill bit is 8.5mm, punching depth is 65mm)

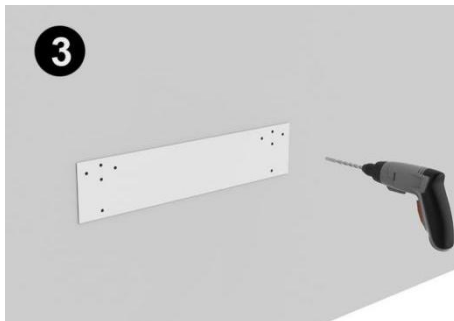
STEP4:

Insert the expansion bolt so that it is flush with the wall.

STEP5:


Align the holes and tighten the screws.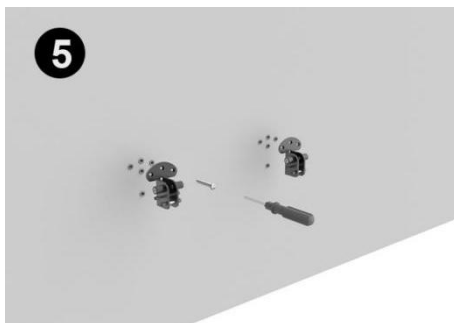

STEP6:

Place the decorative cover on the load-bearing stick; latch.


6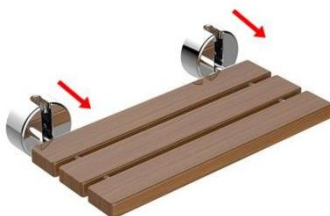

STEP7:

Link the assembled body to the mounting plate and insert the pin.

7

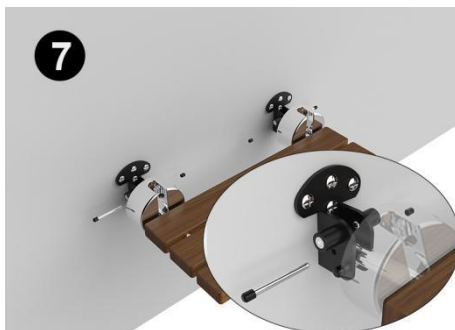STEP8:

Fasten the decorative cover, and the installation is complete.

8

Folding Scheme

Manufacturer: Shanghaimuxinmuyeyouxiangongsi
Address: Shuangchenglu 803nong11hao1602A-1609shi, baoshanqu, shanghai 200000 CN.
Imported to AUS: SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD NSW 2122 Australia
Imported to USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim Place, Rancho Cucamonga, CA 91730

UK	REP
----	-----

YH CONSULTING LIMITED.
C/O YH Consulting Limited Office 147,
Centurion House, London Road,
Staines-upon-Thames, Surrey, TW18 4AX

EC	REP
----	-----

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VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate

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Importerad till AUS: SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD
Importerad till USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim
E-CrossStu GmbH