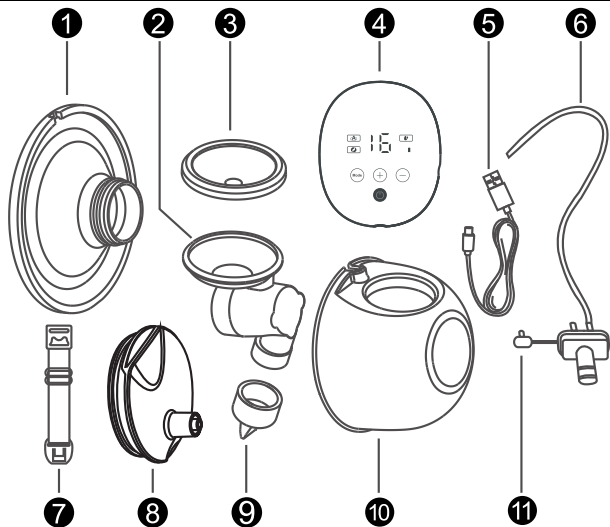
PL.....	4
EN.....	11
DE.....	18
RO.....	25
CZ.....	32
SK.....	39
FR.....	46
ES.....	53
IT.....	60
LT.....	67
LV.....	74
ET.....	81
BG.....	88
HU.....	95

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EN**IMPORTANT SAFETY INFORMATION**

- The device should not be used by persons (including children) with the limited physical, sensory, or mental capabilities, as well as individuals without the appropriate experience or knowledge, unless they are supervised or received appropriate instructions.
- Children must not play with the appliance.

—INTENDED USE—

The device is intended for expressing the milk from breasts.

This is a product intended for personal use. The use by more than one person carries a health risk and constitutes grounds for invalidating the guarantee.

The product is intended to be used inside. It must not be used for commercial purposes. Any other use shall be treated as improper and dangerous. The manufacturer shall not be liable for injuries that may result from incorrect use of the device.

This manual includes important information concerning the use of the product — read it carefully and keep for future reference.

—CAUTION—

Store the appliance, accessories and packaging out of reach of children. The cables and plastic packaging may wrap around a child's neck or may be swallowed, which may result in choking or suffocation. Keep the product away from children.

The device should be disconnected from the mains before cleaning. Do not use the device with a charging cable other than the one supplied.

—DO NOT RISK IT—

Do not use the appliance where use of electrical equipment is forbidden and where it may cause disruption or other hazards. Do not switch on and do not use the appliance near flammable materials. Do not place any flammable objects on the appliance as this may damage of the appliance or even cause a fire.

—WATER AND OTHER LIQUIDS—

The device's body should not be exposed to water or other liquids, including their immersion. This may negatively affect the operation and lifespan of the electronic and electrical components. You should avoid


operation in very moist environments — this may result in damaging the device, and in extreme cases to electric shock.

—BATTERIES—

Avoid exposing the built-in battery to very low or very high temperatures (below 0°C / 32°F or over 45°C / 113°F). The extreme temperatures may affect its life. Avoid contact of the battery with the fluids or metal objects as this may lead to its complete or partial damage. Do not disassemble, modify, short-circuit, or throw the battery into a fire — this may cause explosion and fire. The used or damaged battery should be placed in a special container. Battery in the device cannot be replaced by the user.

STRUCTURE

- | | |
|--------------------------------|-----------------------------|
| 1. Silicone cover x2 | 6. Silicone tubes x2 |
| 2. Funnel x2 | 7. Bra clip x2 |
| 3. Silicone membrane x2 | 8. Bottle lid x2 |
| 4. Body | 9. Valve x2 |
| 5. Power supply cable | 10. Bottle x2 |
| | 11. Adapter |

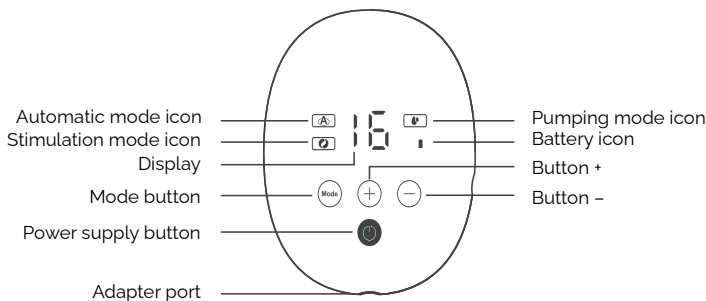TECHNICAL SPECIFICATION

Apning Breast Pump	
Model	DE-050647
Power supply	5V = 1A
Battery	Li-po 3,7V = 1200 mAh, 4.44Wh
Certificates	LFGB, EMC, RoHS
BPA free	Yes
Massage levels	None
Milk intake intensity levels	16
Bottle/container for milk	2 x 180 ml
Single/Double	Double
LED display	Yes
Charging port	DC
Automatic switch-off	Yes

ITEM ASSEMBLY

Place the silicone membrane (**3**) over the top part of the funnel (**2**, **Image A**), and insert the valve (**9**) into its bottom part, making sure that the valve is facing the correct direction (**Image B**). Repeat these steps with the second set of parts. Next insert the assembled funnel into the bottle (**10**, **Image C**), so that the ridges around the funnel align with the grooves in the bottle, and the membrane aligns with the opening in the bottle. Attach the silicone cover (**1**) to the milk bottle and funnel by aligning the indent in the cover with the indent on the edge of the bottle (**Image D**). Repeat these steps with the second bottle.

Place covers (**8**) over the openings in the top of the bottles and firmly press down to ensure there are no leaks. Attach one end of the silicone tubes (**6**) to the openings in the covers and the other to the adapter (**11**), which should be inserted into the port in the device's body (**4**, **Image E**).

You may also use a single bottle only. To do that, place the cap provided over one of the adapter ports, and attach a silicone tube to the other port on one end and to the assembled bottle on the other end. Connect the adapter to the body of the device.

To disassemble, perform the above steps in reverse order.

DEVICE OPERATION

When disconnecting the device from the power, always grasp the plug, not the cable.

Before the first use, wash and disinfect the silicone elements of the product.

To charge the device, connect the DC plug of the included charging cable to the charging port at the side of the device. Insert the other end of the cable into a 5V = 1A USB power adapter (not included).

The battery icon will flash once charging starts. When the battery is fully charged, the icon will emit a steady light. After the end of charging, disconnect the charging cable from the device and from the power supply.

To turn the device ON or OFF, briefly press the power button. Once switched on, pressing the Mode button shortly switches between the different operating modes.

The breast pump should be applied to the breast with the central part of the silicone shield, so that the nipple is placed in the shield's opening.

If the breast pump is being used with a non-flexible bra, the included bra clips (7) may be useful — simply detach the correct bra strap and attach the bra clip. Adjust its length as required.

Operating modes:

- **Automatic mode** : Starting with the lowest level of intensity, the device automatically increases it over time to mimic the different sucking patterns of a newborn; it switches off automatically after 30 minutes of operation.
- **Stimulation mode** : By imitating a newborn's initial rapid sucking pattern, it stimulates the breast to produce milk; it switches off automatically after 30 minutes of operation.
- **Pumping mode** : It imitates a newborn's slower but stronger sucking pattern, and switches off automatically after 30 minutes of operation.

All modes have 16 levels of intensity, which can be changed using the buttons + (plus) and – (minus). The intensity level of the stimulation mode and the pumping mode is memorized after the mode is changed or the device is switched off. Each time the automatic mode is switched on, the device will always start at the lowest intensity level.

To transfer the contents of the bottle, turn off the device and disconnect the silicone tubes from each bottle. Then pour the contents of the containers into, for example, a feeding bottle through the milk outlet channel found in the upper part of the container.

STORING THE PUMPED MILK

Storage temperature	Storage time
19°C–26°C	4–6 hours
< 4°C	72 hours to 8 days
-15°C	2 weeks
-18°C	3–6 months
-20°C	6–12 months
The above data are for informational purposes only.	

- Do not use the microwave or the boiling water to thaw the milk.
- The thawed milk may not be refrozen again.
- Do not mix the freshly pumped milk with the thawed milk.
- Dispose of the milk left after feeding.

CLEANING

WARNING! The device's body cannot be immersed in water or washed under running water!

Before cleaning, the device must be turned off.

Do not use any chemicals, alkaline agents or abrasives for cleaning as they may have a negative effect on the appliance's components.

The cover of the device's body should be cleaned using a dry cloth or a paper towel. In the case of strong dirt it is allowed to use a cloth slightly dampened with lukewarm water. Immediately afterwards wipe these spots with a dry cloth or a paper towel and allow the appliance to dry fully.

The silicone elements coming into contact with the milk should be cleaned after each use and disinfected once a day.

The silicone membrane (3) and the valve (9) are delicate elements. They should be cleaned with particular care and should not be subjected to prolonged disinfection in high temperature. Otherwise, they may be deformed or damaged, which may have an impact on the quality of the product's operation.

The funnel (2) is not suitable for warming or sterilisation in the microwave.

EXPLANATION OF SYMBOLS

Appliance packaging can be recycled according to local regulations.

Symbol designating corrugated cardboard (packaging material).

To be used solely inside.

PAP

This symbol means that the packaging contains an instruction sheet which should be read before use.

The product is suitable for handling foodstuffs.

Pursuant to Directive WEEE 2012/19/EU the product is marked with the symbol of a crossed-out waste container. The product must not be discarded with municipal waste, but should be delivered to a designated collection and recycling centre for used electrical and electronic equipment that safely disposes of such hazardous waste. Collection of such type of waste in designated places and their proper recycling contributes to the protection of natural resources. Appropriate recycling of used electrical and electronic equipment has a positive impact on human health and the environment. You should contact the communal administration or the device's seller in order to obtain the information about the disposal points for used electrical and electronic equipment.

The batteries and accumulators that must be disposed of to specially marked containers, handed over to special waste collection points or to sellers of electrical or electronic equipment, are marked with the symbol of selective collection. They shall not be disposed of together with the unsorted municipal waste. When recycling batteries and accumulators, you should comply with local regulations and the EU legislation.

THE USE OF THE MANUAL

Without the previous written approval of the manufacturer, no part of this operating manual may be copied, distributed, translated or handed over in any form, or using any electronic or mechanical means, including by

creating photocopies, registration or storage in any systems of storage or making the information available.

All the brands and names of the products are trade marks or registered trade marks of the companies' data.

The appearance or colour of the visual representation of the product may differ from that of the actual product.

WARRANTY AND SERVICE

The product has a 2-year warranty, starting from the date of sale. In case of defect the product should be returned to the shop of purchase, along with the relevant proof of purchase. The product should be complete and, ideally, in its original packaging.

EU DECLARATION OF CONFORMITY

mPTech Sp. z o.o. declares that the product complies with basic requirements and the remaining provisions of the so-called "New Approach" Directives of the European Union

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