INSPT0342/03.2024/V1

PT 0342 PAS BARKOWY YOUNG

- PL

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INSTRUCTIONS FOR USE
- SK

NÁVOD NA POUŽITIE
- RO

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INSTRUCTIONS FOR USE

INTENDED USE:

The YOUNG shoulder belt is a reusable medical device intended to support the treatment and rehabilitation process according to the relevant indications.

INDICATIONS:

- thoracic hyperkyphosis
- corrective in postural defects
- pain syndromes of the thoracic spine
- conservative treatment and rehabilitation of clavicle injuries not requiring surgical or orthopaedic intervention
- building a habit of proper body posture
- funnel-shaped chest structure

STORAGE:

Store the device in a dry place. Protect it from high temperatures and direct sunlight.

TARGET GROUP:

The device is designed for persons showing the symptoms described in the indications.

PRECAUTIONS:

The device should be used only on intact skin. Do not apply directly to wounds, damaged or pathologically affected skin.
The use of ointments, oils, creams, and similar substances may have a negative impact on the properties of the device, reduce its durability, and decrease its effectiveness.
In case of persistent or intensified pain, occurrence of allergic reaction, skin irritation, or any other adverse reactions associated with the use of the device, discontinue use and consult a doctor.
The appropriate use of the device may require the advice of specialist medical staff.
For hygienic reasons and for the sake of effectiveness, the device is intended for use only by one patient.
The selection of the correct size determines the proper functioning of the device.
Each use of the device should be preceded by verification of the signs of wear or ageing. Only an undamaged device guarantees optimal performance.
Do not use the device continuously for 24 hours unless advised otherwise by your doctor.
Product designed to be worn on clothing.

INFORMATION ON INITIAL PREPARATION OF THE PRODUCT:

It is recommended to wash the device before first use.

MAINTENANCE:

Fasten the Velcro before washing in order to avoid the device getting caught on something.

INSTRUCTIONS FOR USE:

Open the fasteners. Fit the shoulder braces in such a way that the pads on both straps protect the patient's armpits, placing the element with the steel between the shoulder blades. Pass the ends of the straps through the dedicated frames, located on the element with the steel. Adjust the length of both straps so that the belt is in the correct position before fastening the Velcro. Fasten the straps of the device at the front with Velcro.

COMPOSITION OF RAW MATERIALS PT 0342:

Poliester 67%, Elastodien 16%, Poliamid 10%, Poliacetal 7%

Any serious incident related to the device must be reported to the manufacturer and competent authority of the Member State where the user or patient is resident.

PRODUCT ADJUSTMENT:

A size chart is available on the packaging.

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SIGVARIS S.A.