Endoscope Full HD Camera Y-24

zenwire

Manual Intro

This manual introduces the functions and features of this product and the steps to use. Before using this product please read the instructions carefully to achieve the best results.

Instructions

1. Search ITIMO APP in Apple Store / Google Play or scan QR code to download and install directly.

2. Press the power button for 2 seconds to turn on
3. Enter the phone "ITIMO "-wifi", search for ITIMO -XXX in the network menu and click connect (Make sure you have downloaded our APP, then enter the phone settings and find our APP "ITIMO ". Select "Wireless Data" and then enable the "Wi-Fi and Cellular Data" option. Please complete the above steps to use.)
4. After wifi connection, open the app and start using it.
5. The app required for this product is completely free at all stages! Under no circumstances will we ask customers to provide any personal information! Should you encounter any payment requests or prompts to submit personal data, please ensure you haven't inadvertently clicked on a pop-up advertisement - such requests are in no way associated with our product!

Battery care

Do not allow the device to be fully discharged or stored with an empty battery. This may significantly reduce battery lifespan. Try to recharge the device after each use.

Warnings

1. Read the entire instruction manual carefully before using the product.
2. This product is not suitable for children under 3 years old
3. Do not allow children to use this product alone to avoid accidental injury
4. Do not put the product in liquid to avoid damage
5. The product has a built-in rechargeable lithium battery. If it is not used frequently, it must be charged once a month
6. When using, the product temperature will rise slightly (maximum not more than about 45 °C), please rest assured to use
7. The product is forbidden to use a charger or mobile power supply with a voltage higher than 5V to charge
8. Do not insert the endoscope too deeply into the ear canal to avoid injury
9. Clean the lens and the ear tip with alcohol or disinfectant before and after each use
10. Do not use the product if you experience ear pain, inflammation, or infection - consult a doctor instead
11. Keep the product out of reach of pets and do not use it on animals. It is not designed for veterinary use
12. Do not disassemble or modify the product
13. Store the device in a dry, cool place away from direct sunlight
14. Avoid using the device when hands are wet or in high-humidity environments
15. Do not expose the device to extreme temperatures

E-mail: sklep@zenwire.eu

Manufacturer: Zenwire Sp. Z o.o. Kolobrzaska 4, Poznań 60-418, Made in PRC.

The symbol shown on products or accompanying documentation indicates that faulty electrical or electronic devices must not be disposed of with household waste. Proper handling for disposal, reuse, or recycling involves taking the device to a specialized collection point where it will be accepted free of charge. In some countries, the product can be returned to the local distributor when purchasing another device. Proper disposal of the device allows for the conservation of valuable resources and prevents negative impacts on health and the environment, which could be endangered by improper waste handling. Detailed information on the nearest collection point can be obtained from local authorities. Improper disposal of waste is subject to penalties provided by relevant local regulations.

Business users in the European Union

If you need to dispose of electrical or electronic devices, please contact your nearest point of sale or supplier for additional information.

Disposal of waste outside the European Union

This symbol is only valid in the European Union. If you need to dispose of this product, please contact local authorities or the seller for information on the correct method of disposal.

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