GET READY!

babyono

GB	TWINNY double hands-free electronic breast pump.	2
PL	Podwójny laktator muszlowy elektryczny TWINNY.	6
BG	Двойна електрическа помпа за кърма TWINNY.	10
CZ	Elektrická dvojité odsávačka mateřského mléka TWINNY.	14
DE	Elektrische doppelschalige Milchpumpe TWINNY.	18
EE	TWINNY elektriline käed-vabad rinnapump.	22
ES	Sacaleches eléctrico doble tipo concha TWINNY.	26
FR	Tire-lait coquille électrique double TWINNY.	30
GR	Διπλό ηλεκτρικό θήλαστρο TWINNY.	34
HR	Dupla električna školjkasta izdajalica TWINNY.	38
HU	TWINNY elektromos dupla kagylós mellszívó.	42
IT	Tiralatte elettrico doppio a conchiglia TWINNY.	46
LT	TWINNY dvigubo korpuso elektrinis pientraukis.	50
LV	Dubults elektriskais krūts piena laktators TWINNY.	54
NL	Dubbele borstkolf schelp elektrisch TWINNY.	58
PT	Bomba tira leite elétrica TWINNY dupla.	62
RO	Pompă de sân electrică dublă de tip scoică TWINNY.	66
RU	Двойной электрический лактатор в форме раковины TWINNY.	70
SE	TWINNY dubbel konformad elektrisk bröstpump.	74
SK	Dvojfázová elektrická odsávačka mlieka TWINNY.	78
UA	Подвійний електричний лактатор у формі раковини TWINNY.	84
AR	TWINNY مضخة ثدي كهربائية مزدوجة بشكل الصدفة.	86

TWINNY double hands-free electronic breast pump.

ORIGINAL FEATURES

-  • Creative hands-free and wireless design, easy to carry
-  • Exclusive pumping mode to further satisfy the mother
-  • Digital screen clearly displaying all operating parameters
-  • Built-in battery for use at the time and location of your convenience

NOTE!

- Before using the breast pump for the first time, please read the manual and warnings. Keep the manual for future reference.
- Use the breast pump as intended and specified in the manual.
- Do not use the breast pump when it is physically damaged.
- Do not place the breast pump near devices generating a strong magnetic field.
- Keep the breast pump out of the reach of children.
- Keep the pumping unit of the breast pump in a dry place and do not submerge it in water.

INTENDED USE

- expressing milk from lactating breasts
- relieving engorgement when breasts are full
- storing excessive breast milk to nurse at any time

BEFORE EVERY USE


Find a secluded location and make sure that all parts that come into direct contact with breast milk (Breast Shield, Collection Bowl, Silicone Gas Storage, Silicone Valve) have been sterilized and assembled properly.

1. Wash your hands thoroughly with soap and warm water before handling the pumping set, electric breast pump or any other attachments.
2. Massage your breasts gently and apply a hot compress with a warm towel sitting or lying in a comfortable position.


3. Make sure the pumping set is cleaned, sterilised and dry before pumping.

PARTS INCLUDED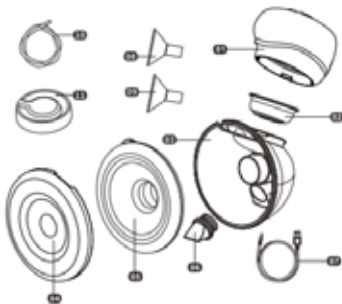

- 01 main pump unit
- 02 silicone gas storage
- 03 collection bowl
- 04 dust cover
- 05 silicone breast shield 28mm
- 06 silicone valve
- 07 USB charging cable
- 08 silicone insert 24 mm
- 09 silicone insert 21 mm
- 10 tubing
- 11 gas storage cap

CONTROL PANEL INSTRUCTIONS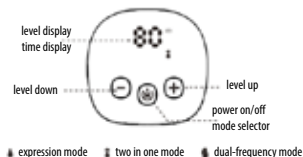


PRODUCT PARAMETER INFORMATION

Model	S018
Input Power	100-240V, 50/60Hz
Suction Strength	$\leq 0.038\text{MPa}$
Charging Time	—150 mins
Rated Power	5W
Output Power	5V/1A
Battery Capacity	1100 mAh
Running Time	—120 mins

MODE	LEVEL RANGE
Expression	1-9
Two-in-one	1-9
Dual-frequency	1-9

ASSEMBLY INSTRUCTIONS

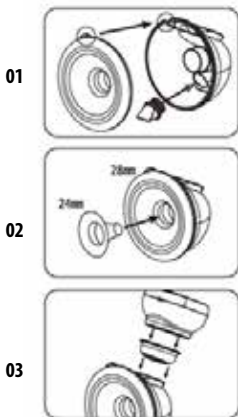01. Insert the duckbill valve into the interface of the bowl and install the breast shield to the bowl. Please note that it is installed place when the milk outlet of the bowl fits the protrusion of the breast shield.
02. Choose 28 mm breast shield or 24 mm or 21 mm insert, depending on your needs.
03. Install the pump unit-silicone gas storage-storage bowl as shown in the above picture.

NOTE: When using single pumping, please make sure the silicone plug is inserted tightly.

When using double pumping

1. Pull out the silicone plug
2. Connect the tubing to the main pump unit and the collection cup

OPERATING INSTRUCTIONS

Maintain proper posture and relax. Place the breast shield on your breast with the nipple in the centre.

Press and hold on top of main pump unit to turn on the power. The digital display screen will turn on and the 1 minute timer will start once the main pump unit is being turned on. If the main pump unit is idle for 1 minute, it will turn off automatically.

Press to select desired mode.

Select to enter EXPRESSION MODE. Touch + to increase suction strength, - to reduce suction strength. Suction strength can be set between L1 and L9. Each setting offers a different pumping strength, from weak to strong.

Select to enter TWO-IN-ONE MODE. The breast pump will provide a pumping cycle of massaging eight (8) times and pumping once (1). This is a good way to relieve pain in full breasts and to produce plenty of breast milk in little time. Touch + to increase suction

strength, - to reduce suction strength. Suction strength can be set between L1 and L9. Each setting offers a gentle massage and different pumping strength, from weak to strong.

Select  to enter DUAL-FREQUENCY MODE. This is a creative new pumping experience on both high and low suction strength settings. The pumping circle of weak to strong is an effective way to correct inverted nipples. Touch + to increase suction strength, - to reduce suction strength. Suction strength can be set between L1 and L9.

Sleep Mode: The default operating cycle of the main pump unit is 30 minutes. After 30 minutes, the pump will turn off automatically.

Screen Lock: When the touch screen is idle for 1 minute, it will be locked automatically. Press  to activate the touch screen.

When finishing pumping, pour the breast milk from collection bowl into the nursing bottle and start nursing or put the breast milk in the a fridge or freezer. Recommended method: remove the pump unit and slowly rotate the shell upside down above the bottle – the milk should flow through the hole.

BATTERY INSTRUCTION

- When the battery is charging, the battery indicator flashes blue and white.
- When the battery is fully charged, it will turn blue. A red light indicates low battery, which should be recharged immediately.
- Take good care of the built-in battery to extend its lifespan. It is recommended to NOT use the Main Pump Unit when charging the battery.
- Do NOT remove or change the built-in battery without professional assistance.

SAFETY GUIDELINES

Do not use the Electric Breast Pump without consulting your doctor if you have mastitis or inverted or flat nipples.

- The breast pump is personal device. Use by more than one person may produce a health risk.
- Keep the Electric Breast Pump out of the reach of children.

- Do not use Electric Breast Pump when sleepy or overly drowsy.
- Do not use the Electric Breast Pump when it is physically damaged.
- For best results, do not use the Electric Breast Pump with components of other manufacturers.
- Do not leave the Electric Breast Pump unattended when it is connected to a source of power supply.
- Do not touch the Electric Breast Pump if it falls in water and unplug it immediately.
- Do not expose the Electric Breast Pump directly to sunlight.
- Do not perform any repairs without authorised professional assistance.

CLEANING AND STERILISATION

- Before using the electric breast pump, disassemble, wash, and sterilise all parts of the Pumping Kit.
- Partially dry components may have adverse effects on the operation of your electric breast pump. Please make sure that all parts are completely dry before use.

GENERAL CLEANING INSTRUCTIONS

Do not wash or sterilise the Main Pump Unit, Power Adapter, or Charging Cable.

- Clean and sterilize all parts that come into direct contact with breast milk: Breast Shield, Silicone Insert, Silicone Gas Storage, Silicone Valve, Collection Bowl.
- Wash the parts in warm water to avoiding milk contamination and bacteria.
- Do not use solvents or abrasives.
- To clean the electric pump base, wipe the base with a damp cloth dipped in water or a water and mild soap solution.
- Suggested sterilisation methods: hot water boiling for 30 sec, dedicated sterilisers. Do not use microwaves and washers.

NOTE: Before each use, visually check individual components for cracks, chips, tears, stains, discoloration, or deterioration as they may produce adverse effects in operation. If any damage to the device is noticed, please stop using the device until the appropriate parts are replaced.

STORING/THAWING BREAST MILK

- Breast milk should be stored and thawed your correctly. This is the only way to make sure that all

important nutrients are preserved and bacterial contamination is minimised.

NBREAST MILK FREEZING/THAWING GUIDELINES

When filling bottles/bags, consider room for possible expansion.

- Label the bottles/bags with the date, on which the milk was expressed.
- Do not store breast milk on the door of the refrigerator. Keep it in the bottom back area of the refrigerator where the temperature is the coolest.
- Thaw frozen breast milk in the refrigerator overnight or hold the bottle or bag under warm running water (max. 37°C).
- Shake the bottle or bag gently to combine any separated fat.
- Do not thaw frozen breast milk in a microwave oven or in a pan of boiling water (risk of burning).

TROUBLESHOOTING

1. FOR LOW SUCTION STRENGTH OR NO SUCTION

- Check the components for correct assembly.
- Make sure that the collection bowl, silicone gas storage, and main pump unit all fit.
- Make sure that the breast shield seals the breast.

2. IF THE PUMP GETS WET

- Turn off the pump. Unplug the power adapter from the electrical outlet.
- Do not turn the pump upside down. The control panel must be upwards.
- Keep the pump in a dry and warm place for 24 hours.
- Do not expose the pump to direct sunlight.
- Do not accelerate the drying process with a source of heat.

3. IF THE PUMP DOES NOT TURN ON

- Check if the power cord is properly connected to the pump.
- Check the electrical outlet for power.
- Contact the store, which sold you the pump.

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