VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate www.vevor.com/support

TOILET SAFETY RAILS

MODEL: FBL900631

We continue to be committed to provide you tools with competitive price.


"Save Half", "Half Price" or any other similar expressions used by us only represents an estimate of savings you might benefit from buying certain tools with us compared to the major top brands and does not necessarily mean to cover all categories of tools offered by us. You are kindly reminded to verify carefully when you are placing an order with us if you are actually saving half in comparison with the top major brands.

VEVOR®

TOUGH TOOLS, HALF PRICE

TOILET SAFETY RAILS

MODEL: FBL900631

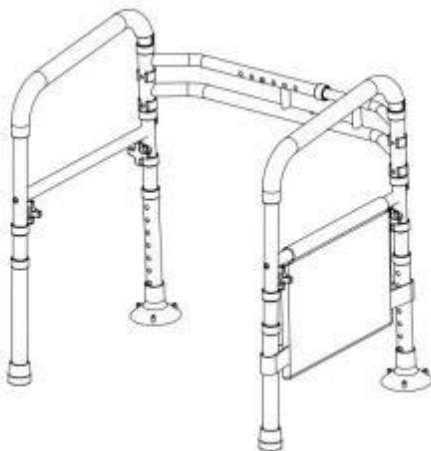NEED HELP? CONTACT US!

Have product questions? Need technical support? Please feel free to contact us:

Technical Support and E-Warranty Certificate
www.vevor.com/support

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

Warning-To reduce the risk of injury, users must read the instructions manual carefully.

WARNING:

Read the manual completely to avoid harm product damage. Seek professional help if you encounter discomfort, pain, or worsening symptoms. If necessary, use supervision in order to assemble and use this product correctly and safely. Do not exceed the 300lb weight limit.

Regularly check for secure assembly and contact support if in doubt. Ensure the product's legs are even and locked before use, and discontinue usage if there are signs of wear, damage, or missing parts. The product is to be used on flat, stable surfaces. Do not disassemble if not necessary, as it might compromise the longevity of the product. Do not modify the product unless specified in the manual. Use only parts approved by VEVOR for this specific product. Confirm the compatibility of the toilet rail with your toilet before first use. The product isn't built to withstand pressure or impact and isn't to be used as a ladder or loaded with heavy objects.

Keep away from children and pets. Contains potentially hazardous small parts - choking hazard. Exercise caution during operation and assembly. Avoid extreme temperatures or open flames. If an allergic reaction occurs, cease use immediately and consult a healthcare professional.

IMPORTANT




1. Don't exceed the weight capacity - 300 lb max.
2. Use on stable surface only.
3. Regularly inspect for signs of wear, damage, or loose parts.
4. Clean regularly using a mild detergent and warm water, rinse Off any soap residue. After cleaning, dry thoroughly.
5. Before use, confirm with your healthcare provider that the product meets your specific needs. The following guidelines are not a substitute for medical advice.

SAVE THESE INSTRUCTION

Model	FBL900631
Rated bearing capacity (lbs)	300
Width adjustment range (mm)	440-590
Height adjustment range (mm)	620-770
Net weight	2.55kg
Product size (mm)	505-655*525*620-770

When unpacking this product, check to make sure the parts listed below are included and carefully inspect for any damage that may have occurred during transit. If any part is missing or damaged, do not attempt to assemble or use the product.

NO	PICTURE	PART NUMBER	QTY	NO	PICTURE	PART NUMBER	QTY
1		H rod welded assembly - outside	1	7		rotary knob	4
2		H rod welded assembly - inside	1	8		Suction cup foot pad	2
3		pipe handrail	2	9		Proximity Card	4
4		Cross bar connection tube	2	10		foot pad	2
5		Handrail connector	4	11		gasket	4
6		inner jut pipe	4	12		storage bag	1

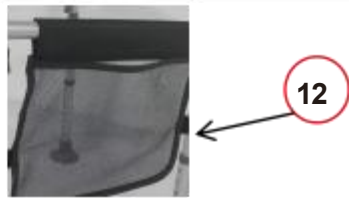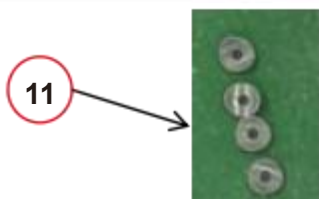
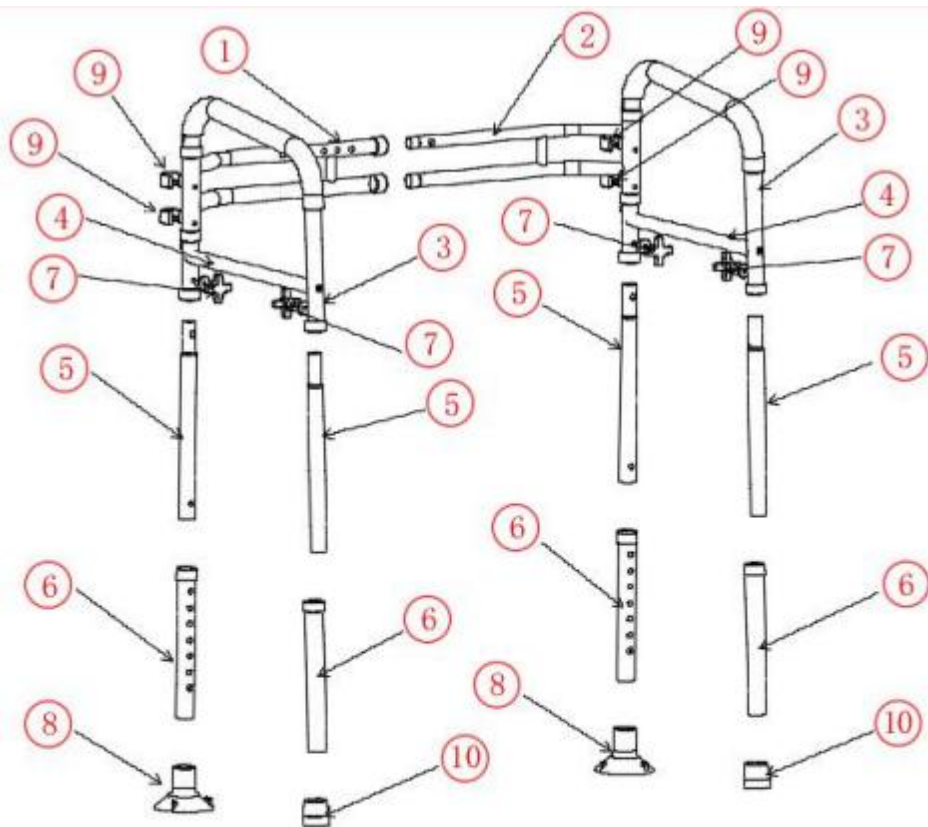
1. Open the Handrests


- 1** Open until both holes are exactly aligned

- 2** Press pins all the way

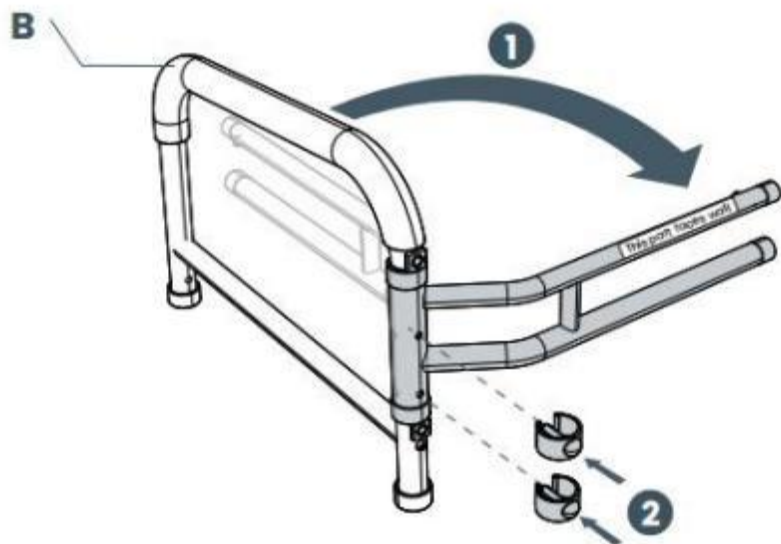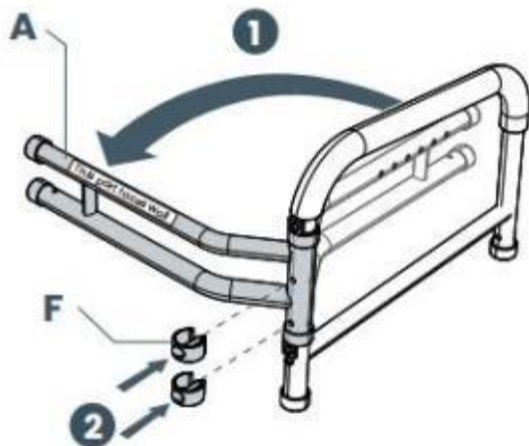
2. Adjust the Width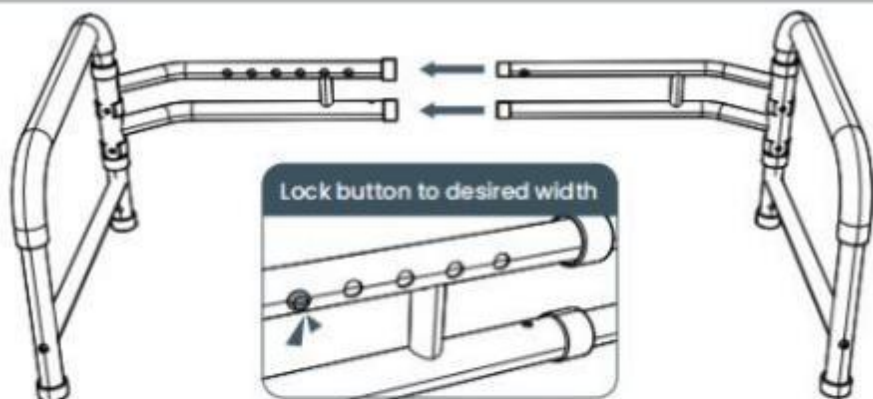


Don't worry, you will be able to easily adjust width after assembly too



3. Attach All Four Legs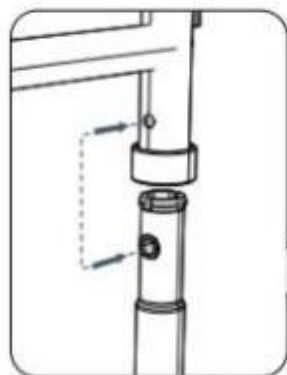



Align the two holes while inserting legs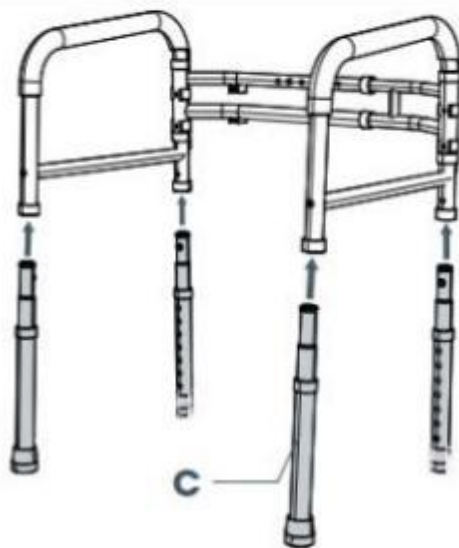

4. Secure All Four Legs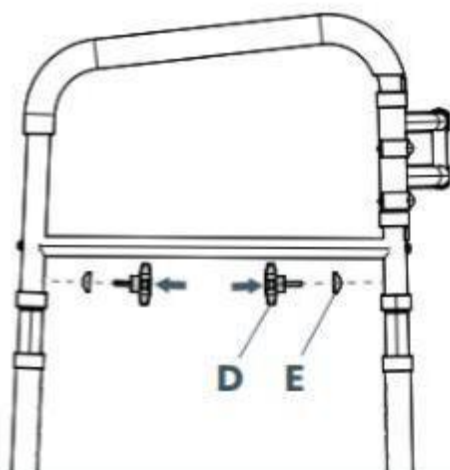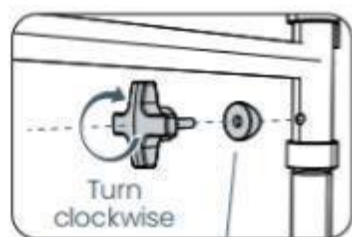

5. Adjust the Height

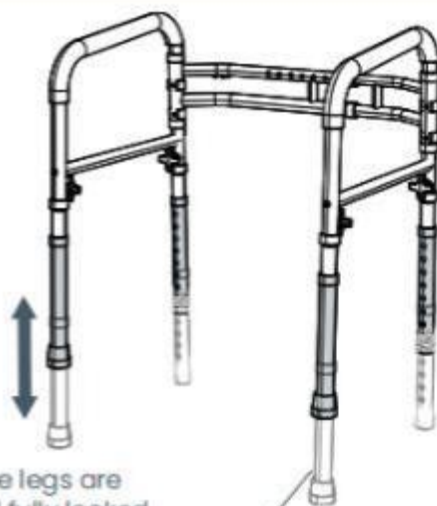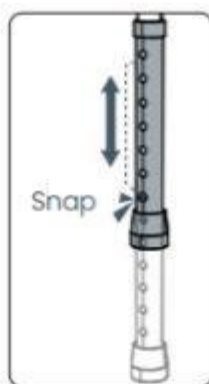⚠ Ensure the legs are even and fully locked

6. Remove the original straight foot pad and install the sucker foot pad into the foot tube as shown

7. Install storage bags as shown in the following figure.

8. Place the assembled toilet rail on the toilet, as shown in the picture

Physiotherapist

1. Before first use, adjust the height and the width of the rail to your toilet.
2. Back up to the toilet with your assistive device (if available) until the back of your legs are touching the toilet.
3. Reach back for the rail, hold firmly, and slowly lower yourself into a seated position on the toilet.
4. To return to a standing position, place your hands on the rails firmly and push yourself up into a standing position.
5. Once you are standing, gradually let go of the rails, and only then, grab the handle of your assistive device (if available).

VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate

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