Technical Support and E-Warranty Certificate www.vevor.com/support

HUMAN BODY TEACHING MODELS

USER MANUAL

MODEL:LD-HS1 LD-HS2 LD-HS3

We continue to be committed to provide you tools with competitive price.

"Save Half", "Half Price" or any other similar expressions used by us only represents an estimate of savings you might benefit from buying certain tools with us compared to the major top brands and does not necessarily mean to cover all categories of tools offered by us. You are kindly reminded to verify carefully when you are placing an order with us if you are actually saving half in comparison with the top major brands.

VEVOR®

TOUGH TOOLS, HALF PRICE

HUMAN BODY TEACHING
MODELS

MODEL:LD-HS1 LD-HS2 LD-HS3

NEED HELP? CONTACT US!

Have product questions? Need technical support? Please feel free to contact us:

Technical Support and E-Warranty Certificate
www.vevor.com/support

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

SAFETY INSTRUCTIONS

WARNING:

1. Read these instructions before using this product. Failure to do so can result in serious injury.
2. After use, please assemble all the parts and keep them vertically.
3. Avoid high temperature and direct sunlight.
4. Remove minor stains with neutral detergent. The use of alcohol, benzene and other substances may damage the product.

SAVE THESE INSTRUCTIONS

PRODUCT PARAMETER

Model	LD-HS1 LD-HS2 LD-HS3
Type	Injection and blood drawing exercise kit
Arm Material	PVC

DETAILED LIST

LD-HS1	Arm	1	
	Nursing Care Pad	1	
	Syringe 1ml	1	
	Syringe 2.5ml	1	
	Syringe 5ml	1	



	Syringe 10ml	1	
	Scalp Needle	3	
	Infusion Bottle	2	
	Gloves	10 pairs	
	Indwelling Needle	2	
	Blood Collection Beedle	2	
	Vacuum Vessel 5ml	2	
	Blood Powder	1	
	Tourniquet	1	
	Stents	1	
	Storage Bag	1	
LD-HS2	Arm	1	
	Nursing Care Pad	1	
	Syringe 1ml	1	
	Syringe 2.5ml	1	
	Syringe 5ml	1	

	Scalp Needle	1	
	Infusion Bottle	1	
	Gloves	10 pairs	
	Indwelling Needle	1	
	Blood Powder	1	
	Tourniquet	1	
	Stents	1	
	Storage Bag	1	
LD-HS3	Arm	1	
	Nursing Care Pad	1	
	Syringe 5ml	1	
	Infusion Bottle	1	
	Blood Powder	1	
	Tourniquet	1	

	Stents	1	
	Storage Bag	1	

INSTRUCTIONS

Installation method: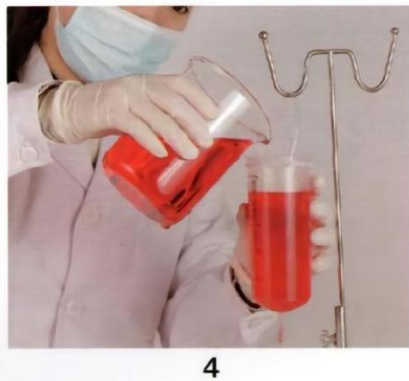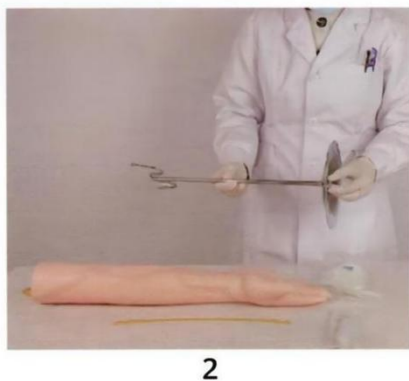


Step 1: Take the arm model, infusion tube, bracket and other accessories out of the box.



Step 2: Tighten the fixing nut to fix the bracket on the hole position of the tray.

Step 3: Blood simulation: add red powder and water to prepare red ink, and then pour it into the hanging bottle (or container)


Blood draw steps: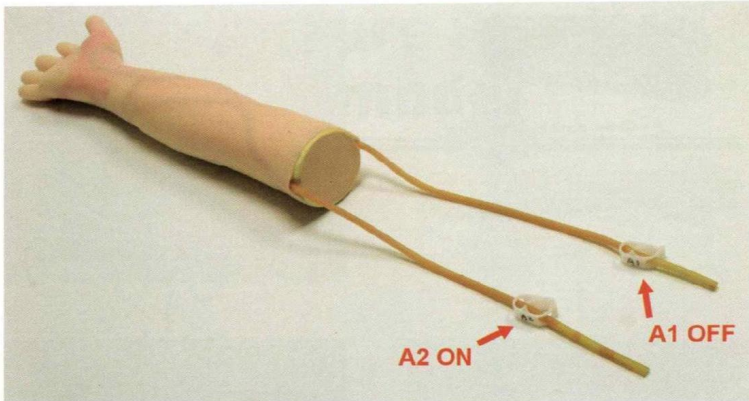

Step 1: Close one of the pipe clamps (A1) on the arm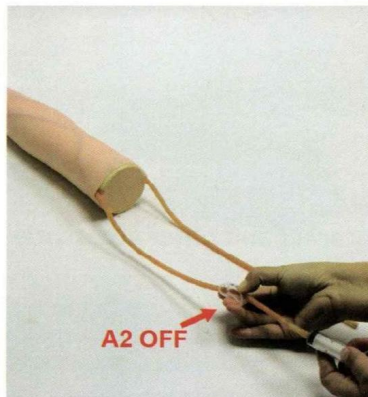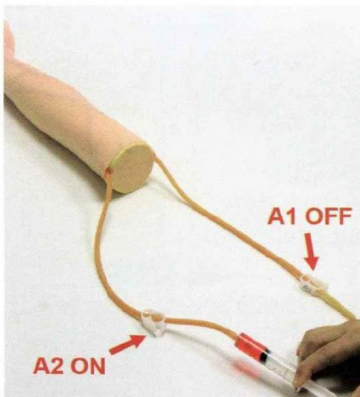

Step 2: After the syringe draws the red ink, fill the other hose (A2) with red ink, and immediately close the pipe clamp(A2) .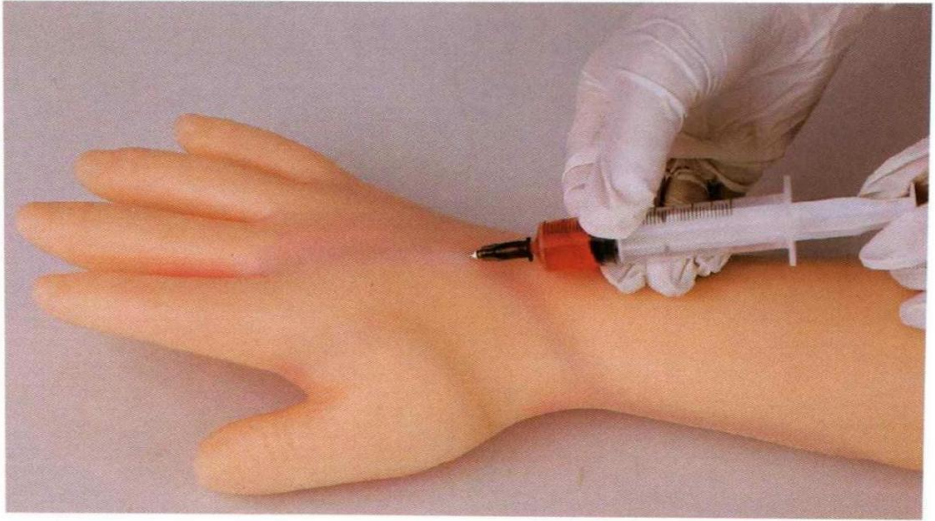

Step 3: Hold an empty syringe and stab it into one of the veins on the arm. Draw blood, and you can see the red liquid entering the syringe.


Note:

If the red ink does not enter the syringe.


The reason as follow:

- a. The vein was completely punctured;
- b. The injected red ink is insufficient;


Blood Transfusion: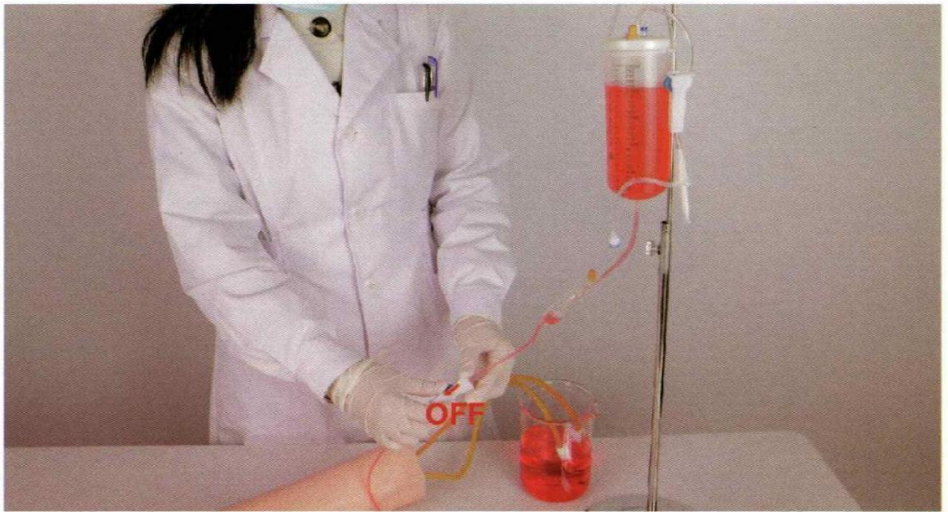

Step 1: First, the infusion bottle is filled with red ink, hanging it on a stand then turn off the flow regulator.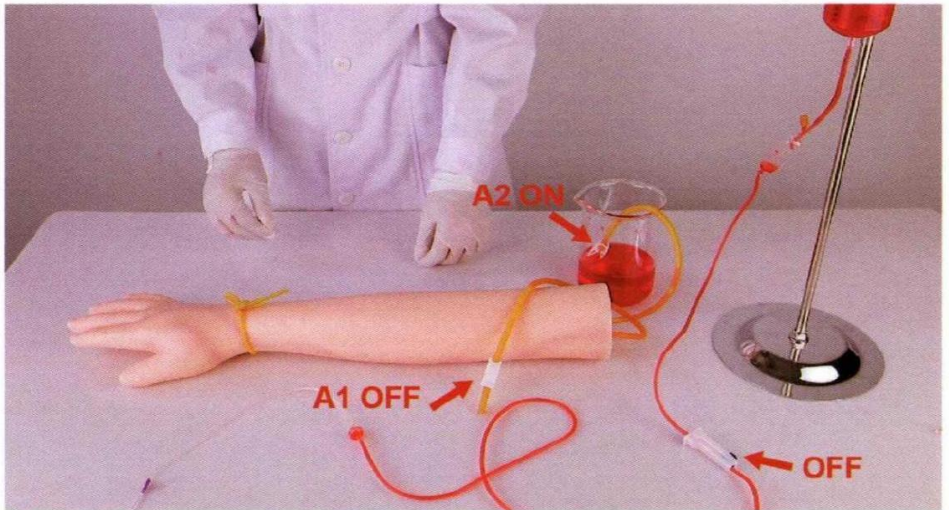

Step 2: Close the pipe clamp(A1) ; open the pipe clamp(A2).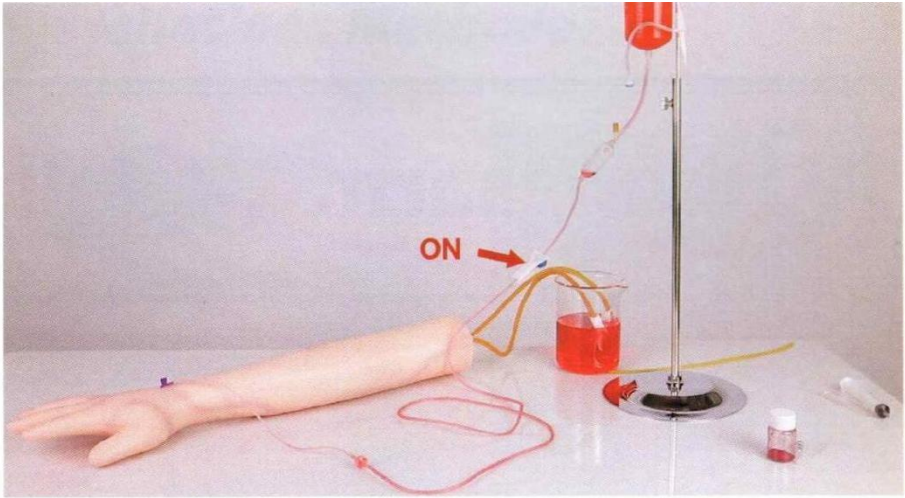


Step 3: The butterfly needle is accurately inserted into the vein of the arm, turn on the flow regulator, the red fluid flows out of the venous tube A2 after circulating in the venous tube. (Use the appropriate container to liquid)

Note:

If the red ink does not outflow from the vein A2.

The reason is as follow:

- The vein was completely punctured;
- The injected red ink is insufficient;
- The A1 pipe clamp isn't hung on the bracket (need hang it as picture show)

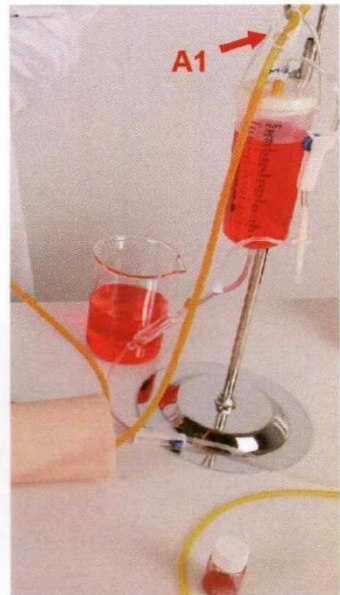Manufacturer: Shanghaimuxinmuyeyouxiangongsi
Address: Shuangchenglu 803nong11hao1602A-1609shi,
baoshanqu, shanghai 200000 CN.
Imported to AUS: SIHAO PTY LTD. 1 ROKEVA STREET-
EASTWOOD NSW 2122 Australia
Imported to USA: Sanven Technology Ltd. Suite 250, 9166
Anaheim Place, Rancho Cucamonga, CA 91730

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YH CONSULTING LIMITED.
C/O YH Consulting Limited Office 147, Centurion House,
London Road, Staines-upon-Thames, Surrey, TW18 4AX

VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate

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E-CrossStu GmbH