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EN

fit4med®

Universal Knee Stabilizer with Hot/Cold Therapy Model: 839

Intended Use:

The universal knee stabilizer with hot/cold therapy is designed to provide comprehensive support and relief for various ailments associated with the knee joint. Thanks to the possibility of adjustment and the use of gel inserts, this stabilizer is excellent for alleviating joint and muscle pain, reducing swelling, and accelerating regeneration after injuries, surgeries, or intense physical exertion. It can also be used to provide additional stabilization of the knee during sports activities or in everyday functioning. Heat therapy helps to relax tense muscles and reduce joint stiffness, while cold therapy effectively reduces pain and swelling, alleviating inflammatory conditions. Our stabilizer is suitable for both physically active individuals and those struggling with chronic knee problems, offering them comfort and support in their daily lives.

Composition:

Neoprene (80%), Elastane (10%), Nylon (10%), Compress: Non-toxic gel in a nylon bag

Size:

Universal, fits both left and right legs

Properties/Action:

The knee stabilizer has been designed to provide comprehensive support and comfort for the knee. Thanks to its special construction and adjustable straps, the stabilizer effectively stabilizes the knee joint, reducing the risk of further injuries and supporting the healing process. The use of gel inserts for hot and cold therapy allows for effective pain relief, reduction of swelling, and reduction of inflammatory conditions.

Preparation for Use:

- **Cold Therapy:** Place the gel insert in the freezer for at least 2 hours before use.
- **Hot Therapy:**
 - **Method 1 (Water Bath):** Immerse the gel insert in hot (not boiling) water for 5-10 minutes. After removing it from the water, check the temperature of the insert by touching it with your hand. The insert should be warm but not hot. Ensure that the temperature is appropriate before placing it in the stabilizer,

- **Method 2 (Microwave):** Heat the gel pack in the microwave in short intervals (e.g., 15-20 seconds), checking the temperature between cycles until the desired warmth is achieved.

IMPORTANT: Exercise caution to avoid overheating the pack. It should be warm but not hot to prevent burns. The maximum temperature to which the packs can be heated is 70 degrees Celsius (158 degrees Fahrenheit).

Usage Time and Frequency:

- **Cold Therapy:** Apply for 15-20 minutes, with breaks of 2-3 hours between applications, as needed.
- **Heat Therapy:** Apply for 20-30 minutes, several times a day.

Indications:

The knee stabilizer is designed to support the healing process and alleviate discomfort associated with various conditions and injuries. It is especially recommended in cases of sprains and strains of the knee ligaments, contusions, and post-traumatic edema, as well as meniscus injuries. Physically active individuals may use it to relieve knee pain caused by excessive exertion, tendinitis, bursitis, and iliotibial band syndrome. The stabilizer is also effective in chronic knee conditions such as osteoarthritis, rheumatoid arthritis, and arthritis. It can be used in the postoperative period, for example, after arthroscopic procedures, ligament reconstruction, or knee replacement, to aid the rehabilitation process. In addition, the knee stabilizer is helpful in cases of knee instability, knee pain caused by overload or degenerative changes, as well as for support during rehabilitation and to improve comfort during physical activity. We always recommend consulting with a doctor or physical therapist before using the stabilizer in the event of serious injuries or conditions to ensure it is appropriate for your situation.

Contraindications:

Do not use the product if the diagnosis is uncertain. Do not use in case of known allergy to any of the components. Do not place the product so that it comes into direct contact with damaged skin.

Necessary Precautions:

Before each use, check that the product is in proper condition. Do not use the product if it is damaged. It is recommended that the first application be carried out under the supervision of a healthcare professional.

It is essential to strictly adhere to the usage guidelines and protocols recommended by medical personnel.

For hygiene reasons and to ensure effectiveness, this product should not be reused on another patient. In case of mild or severe discomfort, pain, changes in limb size, or sensory disturbances, remove the product and seek advice from a healthcare professional.

Undesirable Side Effects:

The product may cause skin reactions (redness, itching, burning, blisters, etc.) or even wounds of varying severity. In the event of an allergic reaction, seek immediate medical attention. Any serious incident involving the product should be reported to the manufacturer.

How to Use / Application:

1. Unfasten all four stabilizing straps.
2. Place the stabilizer on the knee, ensuring it lies evenly against the surface of the knee.
3. Insert the prepared gel packs into the inner pockets of the stabilizer.
4. Begin fastening the straps, starting with those closest to the knee, gradually moving to the upper and lower ones. Adjust the tension of the straps so that the stabilizer fits snugly on the knee but does not restrict movement or cause discomfort. Ensure that the straps are not fastened too tightly to avoid disrupting blood circulation.

Maintenance:

The product is suitable for washing under the conditions specified in this instruction on the label. It is machine washable at 30 degrees Celsius (86 degrees Fahrenheit); use a laundry mesh if possible. Fasten hook and loop fasteners before washing. Do not use detergents, softeners, or aggressive chlorine-containing agents. Do not dry clean. Do not tumble dry. Do not iron. Dry in a spread-out state. Store gel packs in a cool, dry place, away from direct sunlight and heat sources. In the event of deformations or structural alterations of the product, dispose of it in accordance with applicable regulations.

Kit Contents:

- Universal Knee Stabilizer (fits both right and left knees)
- 3 x Gel Packs for Hot/Cold Therapy
- Instructions for Use

Disposal:

After use, dispose of the product in accordance with applicable regulations.

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