AD-06

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User Manual

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Handleiding

PL

EN

DE

HU

NL

OVERMAX

You • unlimited

A

B

C

D

E

2

3

4

5

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Drogi Kliencie!

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Introduction

Dear customer!

Thank you for placing trust in us and choosing Overmax.

Thanks to high-quality materials and modern solutions, we provide you with a product perfect for everyday use. We are convinced that the product will satisfy your requirements as it has been manufactured with utmost care. Before you use the product, please read this manual carefully.

Should you have any remarks or questions concerning the product, please do not hesitate to contact us:

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Description

- A. mouthpiece inlet**
- B. display**
- C. start test / see test results**
- D. see test results**
- E. switch on and off / show number of tests done / show last result**

BATTERY

The breathalyzer needs two AA batteries to operate. To replace the batteries, press the cover and slip it off the battery compartment (at the back of the device). Take old batteries out and put new batteries in; pay special attention to polarity. Close the battery compartment.

15 MINUTES BEFORE MEASUREMENT

Before you start the test, please wait for at least 15 minutes after drinking alcohol. Rinse your mouth with water before you initiate the test.

Do not drink alcohol or smoke cigarettes for these fifteen minutes. If you do so or if you forget to rinse your mouth, the test result may be false. This results from existence of alcohol remains. If you ignore the aforesaid guidelines, the test result may be incorrect and you can also damage the sensor!

OPTIMAL TEST TEMPERATURE

It is advisable to conduct tests and store the device at the room temperature. Do not store the product if temperatures are extreme and if conditions are highly humid (for example in the car in winter or summer).

HOW TO TAKE A TEST

1. Put the mouthpiece in a relevant inlet (upper left corner of the device, marked in fig. 2).

Note – if a test is done without the mouthpiece, the results may be inaccurate. It is advisable to replace the mouthpiece before every measurement.

2. Press and hold the on key for 2 seconds. After switching on, the device will start heating sensors up. The entire procedure takes up to 20 seconds (the counter will be showed on the display, see: fig. 3).

Next choose one of two modes: **automatic** or **manual**.

Automatic mode – easier test mode. AD-06 monitors the user's breath and then decides when to take the sample on its own (end of air exhalation).

Manual mode – usually used when the automatic mode is unfeasible.

AUTOMATIC MODE

- A. When you can see „rdy“ (abbreviated from „ready“, see: fig. 4) and the device emits a short sound, take a deep breath and blow into the mouthpiece which was previously inserted into the device. Make sure you exhale long and evenly (not too fast, at regular pace).

- B.** The device will emit a sound and show a message on the display (see: fig. 5). This will mean that you have exhaled properly and the device has collected the sample of breath.
- C.** Blow into the mouthpiece until you can hear two loud and distinctive „clicks“. This means the product has fully collected the sample.
- D.** The breathalyzer will start analyzing the data (fig. 5). Wait for results.

MANUAL MODE

- A.** When you can see „rdy“ (abbreviated from „ready“, see: fig. 4) and the device emits a short sound, press the left key (C) to activate the manual mode.
- B.** The display will show a message (fig. 5) and emit a continuous sound.
- C.** Take a deep breath and blow into the mouthpiece continually. Keep on blowing until you can hear a double „click“. At this moment stop blowing in.
- D.** The device starts analyzing the breath and shows”[--- „ (calculating). Wait for a while to see alcohol level result.
- E.** The alcohol level result will be displayed in any format (‰, ‰, mg / l) according to the BAC display format specific to the given country. If you wish to change this format, press the left key (C).

IMPORTANT!

- In some cases the device may automatically prolong the sound before blowing in and show the result 0.00. This happens because a very low content of other substance in the sensor may be measured if you do not use the product for a long time. This being the case, retake the test.
- In some cases the device may show the result $\square\square\square$. This happens when the temperature of the test is too low or too high. Before you retake the test, you may be required to wait for a couple of minutes until the sensor inside the device reaches the room temperature.

- The alcohol level result will be presented on the display for 30 seconds. The device will switch off later automatically. You can also shut it down at any time by pressing the on/off key (E) and holding for 3 seconds.

RECOMMENDED TIME BETWEEN TESTS

After you finish the first test and before you start another, you are advised to wait for at least a couple of minutes (5-10 minutes at best) to prevent accumulation of alcohol remains on the sensor and keep the proper condition of the sensor.

TEST MEMORY MODE:

To check the total number of breathalyzer tests, please follow the steps below:

- When counting, press on/off key (E) to display the total number of tests done. This number helps you decide when to perform calibration. For more details, see „Calibration“.
- If you want to reset this counter, long-press the left key (C) until you can see the zero on the counter.
- It is advisable to re-calibrate the device once a year or after 300 tests, depending on what happens earlier.

DISPLAYING LAST 50 TESTS:

- Press the on/off key (E) to display history of tests done (they will be showed on the screen, marked with serial number „1“ and then with relevant result)
- Press the right key (D) to display previous results.
- To leave the test memory mode, press the on/off key (E) again.

SYSTEM MESSAGES

Message	Meaning of the message
H	The device has detected high alcohol level (above 4.00‰ BAC / 0.400%BAC /2.00 mg/L).

Fla	Insufficient or uneven breath detected. Retake the test.
Err	The sensor of the device is not ready. It happens when there is high alcohol or dirt remains from the last test on the sensor. This being the case, switch the device off and wait for a couple of minutes before you take another test.
rPt	The device was unable to finish the test. Retake it.
Out	Low battery. Use new AAA alkaline batteries.
Lo	The temperature of the test is too low or too high. Restore room temperature for the device. Before you take another test, wait for a couple of minutes until the sensor inside the device reaches the room temperature.

SAFETY PRECAUTIONS

- Wait for at least 15-20 minutes after drinking alcohol or smoking cigarette and before you start the test. Remember to rinse your mouth with water.
- For more precise results, it is advisable to calibrate the device every 12 months or 300 tests.
- The device operates at the temperature of 10 – 40°C, with optimal precision for tests at room temperature (i.e. ca. 20 – 26°C).
- If the test result is lower than 0,10 ‰ or 0,10 % BAC or 0,050 mg / L, it is natural if you did not drink alcohol before. It happens because in some cases it is possible to measure other substance than alcohol.
- Do not use the device to specify and decide about driving vehicles. The alcohol level result is for information purposes only. The manufacturer and retailer waive any legal liability for the estimated/measured test value.

MAINTENANCE

To keep the device in a good condition, it is advisable to use the device at least once a month.

CALIBRATION

- The precision of alcohol level testers may change after 6-12 months of use, depending on the number of tests and working conditions.
- If the device shows inconsistent test results, fails to show any results or displays very high or very low results, it must be calibrated.
- To perform calibration, please contact the service center.

The pictures are for reference only. The real design of the products may differ from the pictures presented.

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OVERMAX

declares that the product:

product group:	breathalyser
product name:	AD-06
model symbol:	OV-AD-06
description:	breathalyser

meets the essential requirements of the following Directives:

2014/30 / EU - EMC Directive

and meets the requirements of the following harmonized norms and standards:

EN 55011: 2016 - Industrial, scientific and medical equipment - Radio frequency disturbance characteristics - Limits and measurement methods

EN 61326-1: 2013 - Electrical equipment for measurement, control and use in laboratories - Electromagnetic compatibility (EMC) requirements - Part 1: General requirements

This UE declaration of conformity has been issued under the sole responsibility of the manufacturer. The document together with the technical documentation is the basis for marking the product with the mark 

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Warranty conditions:

1. The Overmax brand with its registered office at A. Kręlewskiego 1, 61-248 Poznań, Poland guarantees that the equipment is operational in accordance with the technical-operating conditions described in the manual.
2. The warranty is granted for a 24 months period from the date of sale. If the Buyer is a company (VAT invoice), the warranty is granted for a 12-month period.
3. The warranty is valid in the country of purchase.
4. Revealed manufacturing defects shall be removed during the warranty period free of charge by the Authorised Manufacturer Service.
5. Any defects or damages discovered and reported within the warranty period will be removed free of charge within 21 working days, but in justifiable cases (for instance, spare parts must be brought from abroad) this period may be extended by another 30 days.
6. Any defects or damages to the equipment should be reported and delivered to the service site immediately after being revealed.
7. Warranty for LCD displays, touch screens, and batteries is 6 months from the date of purchase. In case of displays of a diagonal ranging from 2.5" to 10.1", the presence of up to 3 incorrect pixels (white, black, green, red or blue) is acceptable.
8. Accessories included with the devices (e.g. Chargers, remote controls, keyboards, mice, earphones, cables) are under 3 month warranty from the date of purchase.
9. In case of an unjustified complaint (complaint of efficient equipment or the mechanically damaged one), the person who reports the complaint will be charged for transportation cost.

10. The customer is entitled to replacement of the equipment to another with the same or similar specifications if the service determines that the removal of defects is not possible.
11. The service is not responsible for any loss of data.
12. The warranty will be respected only if the customer attaches (to the faulty device) a damage description, all of the accessories received at the moment of the equipment purchase, and proof of purchase containing the date of sale.
13. The warranty does not cover: mechanical, electrical, thermal or intentional damages and defects caused by them, as well as any defects and disadvantages arising from the use of the device that do not affect its proper operation, damages resulted from improper usage or non-compliance of the instructions described in the manual. This also applies to signal fading due to the distance from the transmitter, poor antenna installation and damages of the equipment, due to fact, that it has not been delivered to the service in a safe container.
14. The Service may refuse to repair products in case of damage to the warranty seal or any traces of unauthorised repair.
15. If the fault is not covered by the manufacturer's warranty, paid repair may be offered by the Service.
16. All reclaimed items are checked by the Authorised Service for correct operation.
17. Neither BrandLine Group Sp. z o. o. nor the Authorised Service is responsible for any damages or losses arising from the inability to use the device being repaired.
18. The equipment that has not been collected from the service site within 3 months can be charged with storage cost appropriate to the storage time.
19. Delivery of the equipment which is incomplete or lacks of proper packaging, is equivalent to the failure of the warranty conditions by the Buyer, and it can provide a basis to refusal of the repair or extension of the repair period.
20. In case of sending a product for repair, when no defect is found, the expenses for the expert assessment shall be fully borne by the consumer.
21. If a product (sent for repair) works properly, and the reason of its invalid operation is a low battery, the consumer shall bear the cost of the expert assessment.
22. The product warranty shall not exclude, limit or suspend the Customer rights resulting from the guarantee.

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