VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate www.vevor.com/support

TUB TRANSFER BENCHES

We continue to be committed to provide you tools with competitive price.

"Save Half", "Half Price" or any other similar expressions used by us only represents an estimate of savings you might benefit from buying certain tools with us compared to the major top brands and does not necessarily mean to cover all categories of tools offered by us. You are kindly reminded to verify carefully when you are placing an order with us if you are actually saving half in comparison with the top major brands.

VEVOR[®]

TOUGH TOOLS, HALF PRICE

TUB TRANSFER BENCHES

MODEL:FBL650326

NEED HELP? CONTACT US!

Have product questions? Need technical support? Please feel free to contact us:

Technical Support and E-Warranty Certificate
www.vevor.com/support

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

Symbol	Symbol Description
	Warning: To reduce the risk of injury, the user must read the instructions manual carefully.
	This symbol, placed before a safety comment, indicates a kind of precaution, warning, or danger. Ignoring this warning may lead to an accident. To reduce the risk of injury, fire, or electrocution, please always follow the recommendation shown below.

PART LIST


When unpacking this product, check to make sure the parts listed below are included and carefully inspect for any damage that may have occurred during transit. If any part is missing or damaged, do not attempt to assemble or use the product.

NO	PICTURE	PART NUMBER	QTY	NO	PICTURE	PART NUMBER	QTY
1		Seat frame	1	6		Backrest tube	2
2		Chair foot	4	7		Armrest	2
3		Backrest	1	8		Shower buckle	1
4		Knob screw	4	9		Cushion	1
5		Shim	4	10		Back cushion	1

SEMBLY INSTRUCTIONS



1. Installation Steps

1..Invert the seat frame and place it on a flat surface, then insert the 4 chair foots ② into the tube mouth of the seat frame ① in sequence (ensuring that the marble on each chair leg pops out normally).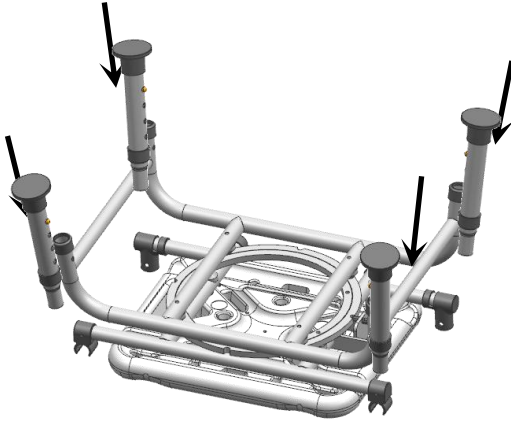


The chair legs are adjustable in four levels, and users can choose the appropriate seat height according to their height through the pinball hole on the chair legs.

2.Place the seat frame with the assembled chair legs on a flat surface, and then insert the two armrests ⑦ from both sides of the seat frame (ensuring that the marble on each armrest pops out normally)。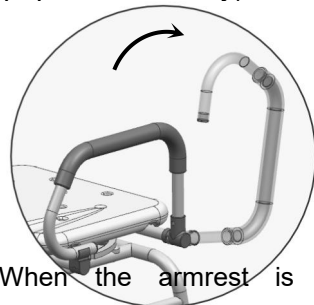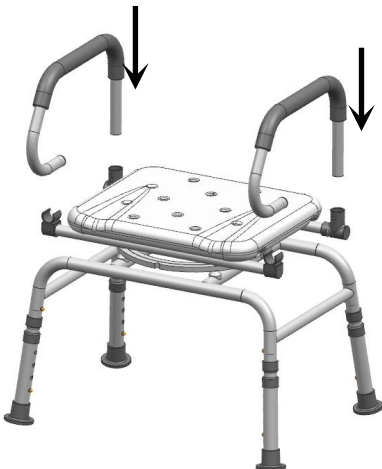


When the armrest is not needed, you can use force to pull it up and rotate it back in the direction of the arrow.

3.Align the holes of backrest ③ and backrest tube ⑥, and then lock them in sequence with four knob screws ④ and washers ⑤

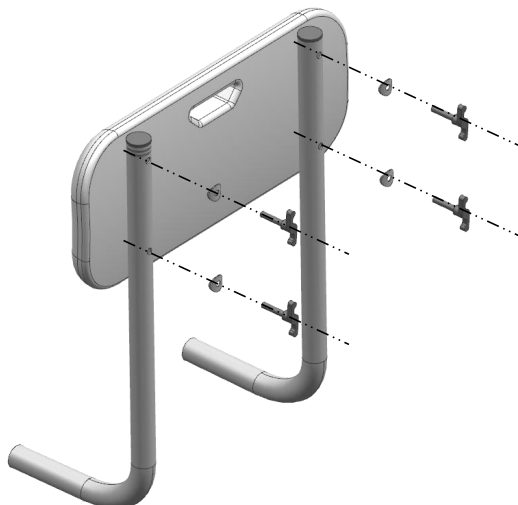
4.Insert the assembled backrest into the corresponding pipe opening of the main frame in the direction shown in the diagram (ensure that the marble of each backrest pipe pops out normally)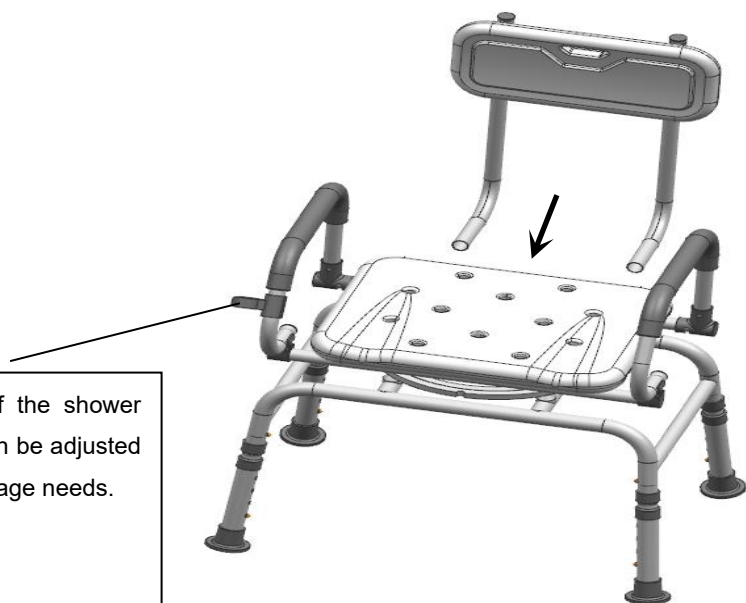

The position of the shower head buckle can be adjusted according to usage needs.

5. The assembled product is shown in the right figure. A pull rod can be seen below the right side of the seat plate. When the pull rod is pulled, the direction of the seat plate can be rotated and adjusted (seat cushion ⑨ and backrest cushion 10 can be freely pasted and used according to needs).

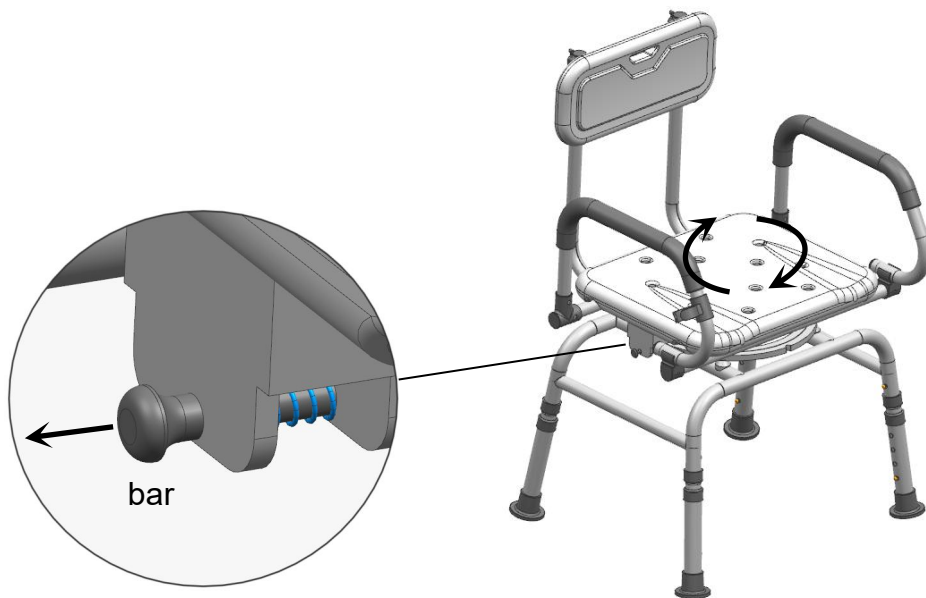SUBSEQUENT CONSULTATION: READ CAREFULLY

Warning:

- *Put the chair stably on a flat floor
- *Building the chairs in good condition.
- *Stay away from open flames.
- *Avoid the overload at the chair.

Precautions before assembly:

- *Prepare a clean and dry mounting area.
- *Verify that the product is not damaged.
- *Check that none of the pieces are missing.

Maintenance:

*Clean with a soft fabric.

*Regularly remove leaves and or water collected on the canvas.

*Check the seating regularly.

Storage conditions:

*The fabric should be stored completely dry, in a clean and dry place to avoid the risks of mold.

Caution with the risks of pinching or cuts during assembly.

The manufacturer declines all responsibility in case of damages due to a bad installation.

Weight limited

Manufacturer: Zhongshan Bliss Medical Instrument Co., Ltd

Address: 2F-2, 3-6F & 7F-1, No. 63, Tongxing East Road, Yumin Community, Xiaolan, Zhongshan, Guangdong, 528414, CHINA

Imported to AUS: SIHAO PTY LTD. 1 ROKEVA STREET EASTWOOD
NSW 2122 Australia

Imported to USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim Place, Rancho Cucamonga, CA 91730

YH CONSULTING LIMITED.

C/O YH Consulting Limited Office 147, Centurion
House, London Road, Staines-upon-Thames,
Surrey, TW18 4AX

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Mainzer Landstr.69,
60329 Frankfurt am Main.

VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate

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Tillverkare: Zhongshan Bliss Medical Instrument Co., Ltd
Importerad till AUS: SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD
Importerad till USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim
E-CrossStu GmbH