VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate www.vevor.com/support

BED RAIL MODEL: LQX-110050

We continue to be committed to provide you tools with competitive price.

"Save Half", "Half Price" or any other similar expressions used by us only represents an estimate of savings you might benefit from buying certain tools with us compared to the major top brands and does not necessarily mean to cover all categories of tools offered by us. You are kindly reminded to verify carefully when you are placing an order with us if you are actually saving half in comparison with the top major brands.

VEVOR[®]

TOUGH TOOLS, HALF PRICE

BED RAIL

MODEL: LQX-110050

NEED HELP? CONTACT US!

Have product questions? Need technical support? Please feel free to contact us:

Technical Support and E-Warranty Certificate
www.vevor.com/support

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

INSTRUCTIONS

Thank you very much for choosing this bed rail. Please read all of the instructions before using it. The information will help you achieve the best possible results.

WARNING:

Please read this manual carefully before using the product. Failure to do so may result in serious injury.

ASSEMBLY PRECAUTIONS

1. Assemble needs to follow these instructions. Incorrect assembly can be a hazard.
2. Wear safety goggles and heavy-duty work gloves during assembly.
3. Do not assemble when tired or under the influence of alcohol, drugs or medication.
4. Weight capacity and other product capabilities only apply to a wholly and correctly assembled product.
5. Keep the assembly area clean and well-lit.
6. Keep bystanders out of the area during assembly.
7. Assembly should be done by adults only.
8. **WARNING:** This product contains small parts. If swallowed, it will pose a choking hazard. Keep these small parts away from children when assembling!

USE PRECAUTIONS


WARNING: TO PREVENT SERIOUS INJURY AND DEATH FROM

TIPPING:

1. Use as intended only. Do not sit or stand on the product.
2. Inspect before every use. Do not use it if parts are loose or damaged.
3. This product is only used in bed, need with nursing bed.

- 4.If any parts are missing, broken, damaged, or worn, stop using this item until repairs are made and/or factory replacement parts are installed.
- 5.The maximum weight-bearing capacity of this product is 450 lbs

Part List

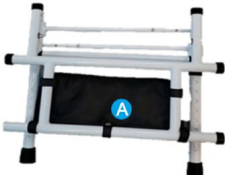NO.	Part name	Pictorial view	Quantity
A	Folding frame		1
B	U-shaped flat tube		1
C	I-shaped tube		2
D	U-shaped armrest tube		2
E	Safety belt		1
F	Storage bag		1
G	C-shaped buckle		4
H	Grip nut		2

ASSEMBLY STEPS

STP1:

Take out the folding frame,unfold it 180° and assemble the upper and lower parts together.

STP2:

Insert C-shaped buckles on both sides to fix.

STP3:

Align the supporting flat tubes on both sides at 90° , and then press the marbles on both sides to insert into the U-shaped flat tubes. After insertion, the marbles will pop out to be firmly fixed.


STP4: Turn the plastic screws on the side of the handrail to lock the U-shaped flat tube

STP5:

Take out the U-shaped handrail tube and insert it on both sides.

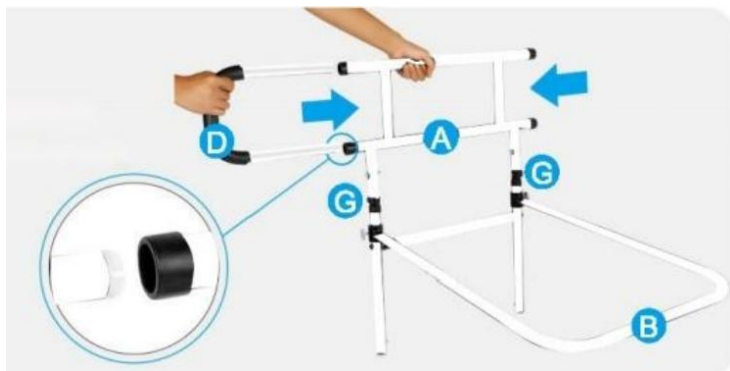
STP6:

Insert C-shaped buckles on both sides to fix.


STP7:

Install-shaped tube on both sides

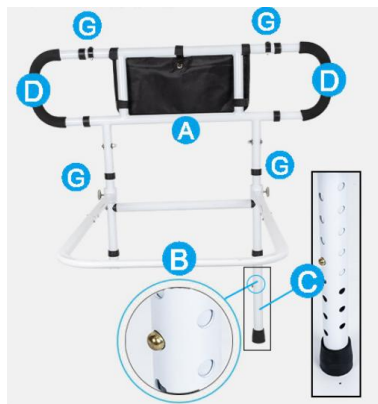**STP8:**

Finally, install the storage bag on the product

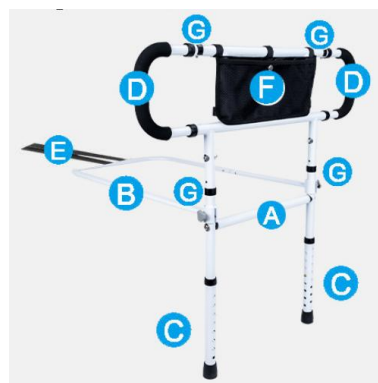**TIP:**

With a fixed tape for the user to secure the fixation (You can choose to use it or not to use it.)

Sanven Technology Ltd.

Address: Suite 250, 9166 Anaheim Place, Rancho Cucamonga, CA 91730

Made In China

VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate

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