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Model: MEDF4-587;830

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Knee compression brace

Model: MEDF4-587;830

WARNING: This is a Class I medical device, reusable. Please read the following instructions for use before using.

Intended use: Support for the knee joint during activities, reducing the risk of cramps and "lactic acid" during physical activities. Aids in rehabilitation after injuries, stabilizes, protects the injured joint, and stimulates blood circulation. Provides relief for post-traumatic and rheumatic pain, effectively reducing swelling. The brace keeps the knee in a safe position. The product is intended solely for use in the mentioned indications.

Size: The brace is not universal and requires selecting the appropriate size. Ensure that the purchased size is correctly chosen. Please refer to the size chart before purchasing.

Composition:

55% nylon, 10% spandex, 35% latex silk.

Indications:

- for weak, unstable, or arthritic knees,
- chronic knee pain, injuries, post-traumatic and rheumatic pain,
- patellar ligament inflammation, chronic inflammatory conditions,
- during rehabilitation after an injury,
- during competitions and physical exertion,
- for preventative use.

Contraindications:

Do not use the product if the diagnosis is uncertain. Do not use if an allergy to any of the components has been identified.

Do not place the product in direct contact with damaged skin.

Necessary precautions:

Before each use, check if the product is in proper condition.

Do not use the product if it is damaged.

Choose the appropriate size.

Remember that wearing and selecting the stabilizer should always be consulted with the treating physician or physiotherapist.

For hygiene reasons and to ensure effectiveness, do not reuse this product on another patient.

In case of mild or severe discomfort, pain, changes in limb size, or sensory disturbances, remove the product and seek advice from a healthcare professional.

Adverse effects:

The product may cause skin reactions (redness, itching, burning, blisters, etc.) and even wounds of varying severity. In the case of an allergic reaction, seek medical assistance immediately. Any serious incident related to the product must be reported to the manufacturer.

Instructions for use:

Put on the product by pulling it up from the bottom to the height of the knee joint. To avoid potential issues with blood circulation, it is important to choose the appropriate size of the brace. To determine the correct size, measure the thigh circumference (15 cm above the center of the knee) and the calf circumference (5 cm below the center of the knee). Measure the leg while seated or standing with the knee slightly bent. Use only during the day with breaks to ensure proper blood flow.

Maintenance:

The product is suitable for washing under the conditions specified in the instructions on the label. It is suitable for hand washing at a temperature of 30 degrees. Before washing, ensure that the Velcro straps are fastened. Do not use detergents, fabric softeners, or aggressive chemicals containing chlorine. Do not dry clean. Do not tumble dry. Do not iron. Lay flat to dry.

In case of deformation or structural damage to the product, dispose of it in accordance with applicable regulations.

Disposal:

After use, dispose of the product in accordance with applicable regulations.

Mona
ul. Leszczynowa 8, 62-070
Dąbrówka, Poland
+48 607 106 591,
www.fit4med.com

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