Rubber Latex Condoms, please read carefully. Always handle the condom carefully including removal from the package so as to avoid damage to the condom by finger nails, jewellery, etc. Condoms made from natural rubber latex may cause allergic reactions including anaphylactic shock. A condom should never be used for a second time, condoms are for single use only and should be discarded after use. The risk of failure with condoms of 100% protection is also higher. The use of a condom cannot guarantee 100% protection against pregnancy, HIV, or Sexually Transmitted Infections (STIs). If in doubt always consult a doctor. If additional lubrication is desired, use the type of lubricant recommended for use with condoms e.g. for Natural Rubber Latex Condoms a water-based lubricant such as Pessaire TLC or Light Lube and the need to avoid oil based lubricants such as Vaseline, baby oil, petroleum jelly, baby lotion, massage oils, margarine, etc., as these are deleterious to the integrity of the latex condom. Use of condom without additional sex increases the risk of breakage. It is recommended to use additional lubrication if vaginal lubrication is insufficient or for anal sex to minimize the risk of condom breaking or coming off. Consult a doctor or pharmacist about the compatibility of topical medicines that might come in contact with the condom. Carrying condoms in your pocket or purse may damage them. Do not use condoms if you have unprotected sex or your method of contraception has failed, contact your Sexual Health Clinic or GP for emergency contraception advice preferably no later than 72 hours. Keep away from children, children can choke or suffocate on unflattened or broken condom fragments. Use of condom during oral sex increases the risk of asphyxiation, choking or condom breaking. A latex condom is a medical device. Any complaint or incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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EC REP Advena Ltd, Tower Business Centre,
Pasante Europe Ltd