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Model: MEDF4-1061;1082

WARNING: This is a Class I medical device, reusable. Please read the following instructions for use before using.

Intended use:

Relief, maintaining the knee in the proper position, preventing uncontrolled movements and possible injuries. The cutout for the patella provides good stabilization and prevents displacement. It can be used during rehabilitation after injuries to support the healing process and restore full joint function. The product is intended for use only in the specified indications. The brace is not universal and requires the selection of the appropriate model for the left/right knee. It is important to ensure that the purchased product is suitable for the specific side.

Composition:

62% neoprene; 18.5% nylon, 12.5% polyester, 7% spandex.

Size:

The brace is not universal and requires the selection of the appropriate model for the left/right knee. It is important to ensure that the purchased product is suitable for the specific side. Thigh circumference: 37-46 cm / Calf circumference: 31-40 cm.

Application:

The knee brace combines comfort and stability, with supports that allow for maintaining the lateral structure without rolling the material during the movement of the knee joint.

It allows for relieving the knee, supports the lateral stabilization of the knee joint, and provides additional support, helping to improve overall stability and eliminate twisting in the knee. The open design features two hook-and-loop fasteners. Additionally, the brace has an elastic peripheral strap fastened with Velcro. Three independent fasteners provide quick adjustment of compression, increasing user comfort.

Indications:

Sprains and strains, knee instability, rheumatic pain and swelling, preventive use during increased effort, preventive use after injuries, sports activity.

Contraindications:

Do not use the product if the diagnosis is uncertain. Do not use if an allergy to one of the components has been identified. Do not place the product in direct contact with damaged skin.

Necessary precautions:

Before each use, check whether the product is in proper condition. Do not use if the product is damaged. Remember to always consult your treating physician or physiotherapist about wearing and selecting the stabilizer.

For hygiene reasons and to ensure effectiveness, this product should not be reused on another patient. In case of mild or severe discomfort, pain, changes in limb size, or sensory disturbances, remove the product and seek advice from a healthcare professional.

Adverse effects:

The product may cause skin reactions (redness, itching, burning, blisters, etc.) and even wounds of varying severity. In the case of an allergic reaction, seek medical assistance immediately. Any serious incident related to the product must be reported to the manufacturer.

How to Use: Position the stabilizer so that the round opening is correctly placed over the kneecap. Adjust the straps with Velcro and fasten them in front of the knee joint to ensure natural movement of the joint and align the support point. Then, fasten the remaining two straps (the upper and lower) for comfortable compression. The elastic, adjustable straps ensure a precise fit for the knee brace and provide optimal compression for the joint. To avoid possible issues with blood circulation, do not tighten the brace too much.

Maintenance: The product is suitable for washing under the conditions specified in the instructions on the label. It is suitable for hand washing at a temperature of 30 degrees. Before washing, ensure that the Velcro straps are fastened. Do not use detergents, fabric softeners, or aggressive chemicals containing chlorine. Do not dry clean. Do not tumble dry. Do not iron. Lay flat to dry.

In case of deformation or structural damage to the product, dispose of it in accordance with applicable regulations.

Disposal: After use, dispose of the product in accordance with applicable regulations.

Mona
ul. Leszczynowa 8, 62-070
Dąbrówka, Poland
+48 607 106 591,
www.fit4med.com

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