USER MANUAL

Upholstered commode chair

Product code: BLUE-TIM

1. Characteristics

The upholstered commode chair is made of powder-coated steel frame. It has height-adjustable legs finished with non-slip rubber tips. The option to adjust the chair allows it to be tailored to individual patient needs, while the side armrests provide added stability and safety during use. The chair is equipped with a toilet seat and a conveniently extendable plastic sanitary container. A soft, upholstered seat with backrest enhances user comfort.

Set includes: chair frame, backrest, armrests (2pcs), legs (4 pcs), locking clips for legs (4 pcs), locking knobs (6 pcs), soft seat, sanitary container.

Elements of the commode chair:

- A. backrest
- B. frame
- C. legs
- D. anti-sliding tips
- E. sanitary container
- F. armrests
- G. upholstered seat

2. Application

The commode chair is ancillary equipment which helps the user to perform hygienic activities. The chair is recommended to people with dysfunctions and reduced mobility who have problems with unassisted walking and for those who lost the ability of unassisted sitting down and standing up. The chair functions as a toilet owing to the toilet bucket located under the seat. The chair facilitates performing physiological activities and taking care of the user.

3. Assembly, adjustment and usage

UNFOLDING

Step 1: Turn the chair frame upside down and insert the four legs into the mounting holes. Tighten them using the mounting knobs.

Step 2: Adjust the legs to the desired height and secure them using locking clips.

Step 3: Turn the chair upright and place it on the ground.

Note: All four legs must be adjusted to the same height and rest firmly on the ground for stability.

Step 4: Insert the backrest into the chair frame and tighten it using the two locking knobs.

Step 5: Place each armrest onto the armrest frame so that it fits into the groove and extends outward.

Important: Do not install the armrests in reverse, as this will reduce the seat width between them. Once properly positioned, secure each armrest from underneath using Allen screws.

Step 6: Slide the sanitary container along the guides located under the seat.

Step 7: Place the upholstered seat cushion on top of the toilet seat.

HEIGHT ADJUSTMENT

To adjust the height of the chair, pull out the locking clips, select the desired height and lock the set height afterwards. The legs are adjustable in increments of 2,5 cm.

Note: all legs must be set at the same level. All legs should stand firmly on the ground.

USAGE

Lift the soft seat, and remove the lid of the sanitary container. After use, cover the container with the lid and place the soft seat on the toilet seat

Note! Remember to empty the toilet sanitary container after each use.

4. Technical data

Overall width	59 cm
Overall depth	52 cm
Backrest height	41 cm
Seat height from the ground	44-54 cm
Overall height from the ground	83-93 cm

Seat dimensions	40 cm x 45 cm
Chair weight	8,1 kg
Maximum capacity	136 kg
Frame colour	blue

5. Notes

Timago International Group does not bear responsibility of improper use of the product, failure to comply safety regulations and misuse.

6. Maintenance

Carry out regular maintenance of the technical condition of the chair:

- Clean the commode chair regularly using only water and mild detergents. Dry thoroughly.
- **Do not use** for cleaning: strong polishing agents and detergents, abrasive agents bleach, petrol or solvents (it may cause frame damage, visible defects and in consequence - loss of warranty rights).
- Empty and disinfect the sanitary container and the toilet seat after each use.

7. Storage and transport conditions

The product should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

9. Labels

	Reference number		Manufacturer
	LOT number		UDI code

	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors. Learn more at: **www.timago.com**. Thank you for choosing Timago!

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