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Tkanina zewnętrzna: 60% poliester, 40% PVC

Tkanina wewnętrzna: 20% poliester, 80% PVC

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Magnetic Lumbar Belt with Tourmaline.

USAGE INSTRUCTIONS: Open the packaging and remove the cover. Wrap around the desired body area, with the lighter side of the belt facing inward, ensuring the belt does not restrict movement. Make sure the selected body area is relaxed and unrestricted to reduce the risk of further damage. To enhance the heating effect, moisten the material in the center of the belt before application.

WARNING: The product contains strong magnetic fields that may harm individuals with pacemakers. Do not use the belt for more than 1-2 hours per day. It is recommended to check the heated area every 15 minutes. If redness, irritation, or discomfort occurs, remove the belt immediately. In the case of persistent pain in the heated area, it is important to seek a proper diagnosis. Consult a doctor regarding the

safe use of the product. Do not use the belt while sleeping, near heat sources, or during exercise. Avoid applying additional pressure to the belt. Ensure proper air circulation around the belt. Discontinue use immediately if skin discoloration occurs. Not suitable for children, elderly individuals, or those unable to operate the belt independently. Do not use on irritated or damaged skin. Do not use the product in combination with creams applied to the skin, or in cases of rheumatoid arthritis, poor circulation, nerve damage, sensory disorders, skin diseases, vascular disorders (Raynaud's disease), diabetes, or pregnancy. For external use only. Do not use the belt if allergic to latex. The product packaging contains natural latex rubber, which may cause skin allergic reactions. Plastic packaging can be hazardous. To avoid the risk of suffocation, keep away from children and infants.

Washing Instructions: Hand wash only. Do not bleach. Air dry only. Do not iron. Do not dry clean.

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