

正面：

背面：

序号	时间	更改内容	更改人	标记	数量
1	2021/05/31	统一字体	钟华	V1.01	1
2	2023/3/17	调整版面	钟华	V1.02	1
3	2023/3/22	更新地址信息	钟华	V1.03	1

URIT

Operation Guide for Hemoglobin Meter

URIT Medical Electronic Co., Ltd.

Address:No.D-07 Information Industry District, High-Tech Zone, Guilin, Guangxi 541004, P.R. China  
Tel:+86(773)2288586 Fax:+86(773)2288560  
E-mail: service@urit.com Web:www.urit.com

Operation guide cannot replace instruction for use.  
Read instruction for use for Hemoglobin Meter carefully before use.

Rev:1.03

Required Items

Test Strip Inlet

Test Hole

Test Strip Bracket

Code Strip Slot

Power Switch Button

Test Strip Slot

Printer Connector

Hemoglobin Meter

Lancing Device + Lancet

Device Body

Release Button

Lancet

Device Cap

Lancet Holder

Lancet Cap

Depth Adjuster

Test Strip Package

Test Strip

Instrument operation

1.Set the time after using the device for the first time or after replacing the battery.

2.In the off state, insert the code card, and then press the power button. The screen will display the code, compare the code with the code on the test strip package, and continue to use when the code is correct.

3.Press the power button to turn on the machine, when the test bar icon flashes, insert the test strip.

4. This flashing droplet symbol indicates that the device is ready to collect a blood sample.

5.Disinfect the finger tip.  
Assemble the fingerstick and press the trigger button while gently push the figerstick against your finger.

6.Gently press your finger to help the blood flow.

7.Bend the test strip and suck the blood.  
Don't touch the test strip that sucked blood.

8.Testing.

9.Display results.

10.Discard or dispose of used blood collection strips and lancets as required by your local laws.

Symbols	Description	Measures
SUN	Strong light environment	Strong light will interrupt with the optical system of the device. Please avoid sun light, and other strong lighting.
E10	Wrong test result	1. Check the validity and the code of test strip. 2. Blood sampling method or sample volume need may not meet the requirements.
E11	Removed test strip during detection	Please re-test with a new test strip
E12	Unable to finish the test	1. Check the validity of test strip. 2. Use another test strip and repeat the test.
E13	Wrong test strip insertion	1. Test strips should be inserted all the way in, if not, please pulled out completely and re-inserted. 2. Please make sure the droplet icon is flashing before taking the blood sample.
E14	Brightness check low	Please perform maintenance and care steps as instructed in operation manual.
E15	Brightness check high	The test strip may not be valid, or it's expired.
E16	Abnormal ADC value at the end point	Make sure that the teststrip hasn't been moved during the test.
E17	Blood sample taken when the droplet icon is not flashing	Please make sure the droplet icon is flashing before taking the blood sample.

技术要求：

- 1、材质为200g铜版纸，上光，符合RoHS要求，（黑色细线边框尺寸95\*135mm以外内容及边框不印刷）；
- 2、纸张规格为380mm \* 135mm；公差±3mm
- 3、正反双面彩色印刷,印刷的字迹应清晰、无污迹、模糊及变形，
- 4、同一批印刷的颜色不得有用肉眼可分辨的差异；
- 5、印刷成品每两页间压出压印痕线,以封面为第一页按四联风琴折叠好，装进防潮袋，每袋数目为100的倍数，不足100单独装袋，每袋在外面标明数量。

保密：此图及其全部知识产权（含著作权）归桂林优利特医疗电子有限公司所有。未经桂林优利特医疗电子有限公司预先书面许可，严禁出于任何目的，对此图的全部或者部分内容（包括但不限于图中信息、数据、运算结果等）泄漏、使用、拷贝或者复制。					
拟制		虹吸式血红蛋白分析仪操作指南英文版	代号：PHP050000065		
审核			版本	第1页	共1页
工艺			V1.03	比例	4:5
批准					

S4