# **VEVOR<sup>®</sup>**

**TOUGH TOOLS, HALF PRICE**

Technical Support and E-Warranty Certificate

[www.vevor.com/support](http://www.vevor.com/support)

## **Transport Wheelchair**

**MODEL: FS210ABEJP-61**

We continue to be committed to provide you tools with competitive price.

"Save Half", "Half Price" or any other similar expressions used by us only represents an estimate of savings you might benefit from buying certain tools with us compared to the major top brands and does not necessarily mean to cover all categories of tools offered by us. You are kindly reminded to verify carefully when you are placing an order with us if you are actually saving half in comparison with the top major brands.

# VEVOR<sup>®</sup>

TOUGH TOOLS, HALF PRICE

**Self-Propelled Wheelchairs**

**MODEL: FS210ABEJP-61**

## **NEED HELP? CONTACT US!**

Have product questions? Need technical support? Please feel free to contact us:

**Technical Support and E-Warranty Certificate**  
**[www.vevor.com/support](http://www.vevor.com/support)**

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

## CAUTIONS FOR OPERATION

- 1.Do not drive the wheelchair on a slope more than 8km/h
- 2.Do not use the wheelchair on the bumpy road surface,
- 3.The locks are only used for parking the wheelchair on the ground.
- 4.Do not use the hand lock to stop the chair when it is moving.

### Part List

| NO<br>. | Pictuer                                                                             | Name          | QTY<br>. |
|---------|-------------------------------------------------------------------------------------|---------------|----------|
| 1       |    | Wheelchair    | 1        |
| 2       |   | Foot Assembly | 1        |
| 3       |  | Support Tube  | 1        |

## Unfold Wheelchair

## Product Installation

### Step 1:

Foot assembly mounting.

Same operation on the other side.

## Step 2:

Tighten the strap.

## Step 3:

Support tube mounting.

**A. Twist out the club handle.**

## **B.Assembly support tube.**

## **Brake Introduction**

### **1.Hold the brake.**

**2.Release brake.**

## Product Folding

**Step 1:**

Release support tube.

**Step 2:**

Pull up the seat cushion.

### **Product Maintenance:**

1. Do not scratch or scratch the surface of the product to prevent material damage.
2. Daily cleaning and maintenance with the neutral synthetic detergent, wipe with wet cloth, then wipe with dry cloth.
3. Do not contact with strong acid or alkali to prevent corrosion.

### **Product Storage:**

Packaged commode chairs should be stored at the temperature not exceeding -10°C~40°C and a relative humidity not exceeding 80%. No corrosive gases, the room is well ventilated and clean. During transportation, it should be protected from moisture, light, and heat sources.

**Manufacturer:** Shanghaimuxinmuyeyouxiangongsi

**Address:** Shuangchenglu 803nong11hao1602A-1609shi, baoshanqu, shanghai 200000 CN.

**Imported to AUS:** SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD NSW 2122 Australia

**Imported to USA:** Sanven Technology Ltd. Suite 250, 9166 Anaheim Place, Rancho Cucamonga, CA 91730

|    |     |
|----|-----|
| UK | REP |
|----|-----|

YH CONSULTING LIMITED.  
C/O YH Consulting Limited Office 147,  
Centurion House, London Road,  
Staines-upon-Thames, Surrey, TW18 4AX

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**VEVOR<sup>®</sup>**

**TOUGH TOOLS, HALF PRICE**

**Technical Support and E-Warranty Certificate**

**[www.vevor.com/support](http://www.vevor.com/support)**

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**Importerad till AUS:** SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD NSW 2122 Australien
**Importerad till USA:** Sanven Technology Ltd. Suite 250, 9166 Anaheim Place, Rancho Cucamonga, CA 91730
E-CrossStu GmbH