

ICE PACK 100g

SUCHY LÓD



REF

WB-ICP-100

TIMAGO.COM



INTCO MEDICAL (HK) CO., LTD.

Flat/RM19C, Lockhart Centre, 301-307
Lockhart Road, Wan Chai, Hong-Kong, China

EC REP

MedNet EC-REP GmbH

Borkstraße 10, 48163 Münster, Germany



Timago International Group

Spółka z o.o. i Spółka – Spółka komandytowa
ul. Karpacka 24/12, 43-316 Bielsko-Biała, Polska

MD

CE
0197



5 902983 724834



LOT

ICE PACK 100g

SUCHY LÓD



REF WB-ICP-100

TIMAGO.COM

EN INSTRUCTIONS FOR USE:

Hold the bag at one of the seals and shake it. Press the center of the bag with both hands until the inner bubble pops. Shake to mix the contents. Use the cooling pack to cool the desired part of the body. The effective cooling time is 20-40 minutes. Avoid excessive cooling of the body resulting in bruising of the skin and/or burning at the site of cooling

CAUTION:

- do not freeze or apply to open wounds,
- do not open the bag - in case of contact with skin or eyes, rinse thoroughly with water and seek medical advice immediately,
- after use, the product should be disposed in accordance with applicable regulations,
- for single use only.

PL INSTRUKCJA STOSOWANIA:

Chwyc torebkę za jeden ze zgrzewów i wstrząśnij nią. Naciśnij środkową część torebki obiema rękami, aż pęknie wewnętrzna bańka. Potrząśnij aby wymieszać zawartość. Uzyskaną torebkę chłodzącą wykorzystaj do schłodzenia odpowiedniej części ciała. Efektywny czas działania torebki chłodzącej wynosi 20-40 minut. Należy unikać nadmiernego schłodzenia ciała objawiającego się zasinieniem skóry i/lub pieczeniem w miejscu schłodzenia.

UWAGA:

- nie zamrażać oraz nie stosować na otwarte rany,
- nie rozrywać torebki - w przypadku kontaktu zawartości ze skórą lub oczami, należy obficie spłukać wodą i natychmiast skontaktować się z lekarzem,
- po wykorzystaniu, produkt powinien być utylizowany zgodnie z obowiązującymi przepisami,
- produkt przeznaczony do jednorazowego użytku.