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INSTRUCTIONS FOR USE

Issue number and date: 1 / 11.2024

**Aluminium rehabilitation walker
Model CH100AM**

Introduction

Thank you for choosing our product. We are confident that it will meet your expectations. We encourage you to read this document before using the product for the first time. These instructions are intended to provide the necessary information on the safe use of the medical device. The manual provides detailed guidance on the device's functions and capabilities. Retaining the document allows this information to be easily found in the future, facilitating correct use.

Description of the medical device

The aluminium tri-functional rehabilitation walker is a device that combines lightness, functionality and comfort. Four-wheel design, foldable with seat. Made of high-quality aluminium, the product allows for easy folding and height adjustment, adapting to the individual user's needs. Conveniently shaped brakes ensure safety during use and the reclining seat increases comfort. A removable shopping bag, larger 20-cm-diameter wheels, height-adjustable handles and reflective elements for increased visibility complete the package. A padded backrest bar provides additional support for the user.

Intended use

A medical device designed for people with restricted mobility, providing stable support when walking. It enables safe movement, supports verticalisation and allows gradual load-bearing of the lower limbs. The walker significantly improves the quality of life for people with reduced mobility, increasing their independence. It allows for safe and independent activities of daily living, such as standing up or walking. It provides a solution for the elderly, as well as for people with various disabilities.

Contraindications

Verticalisation problems and upper limb injuries that prevent support on the device. People with these conditions should avoid using a walker for their own safety.

Product design

Construction element	Description
A	Guide handle with brake
B	Backrest with padding
C	Bag included
D	Horizontal locking mechanism
E	Upholstered seat
F	Handle height adjustment lock

Assembly and adjustment

The product is supplied in self-assembly form.

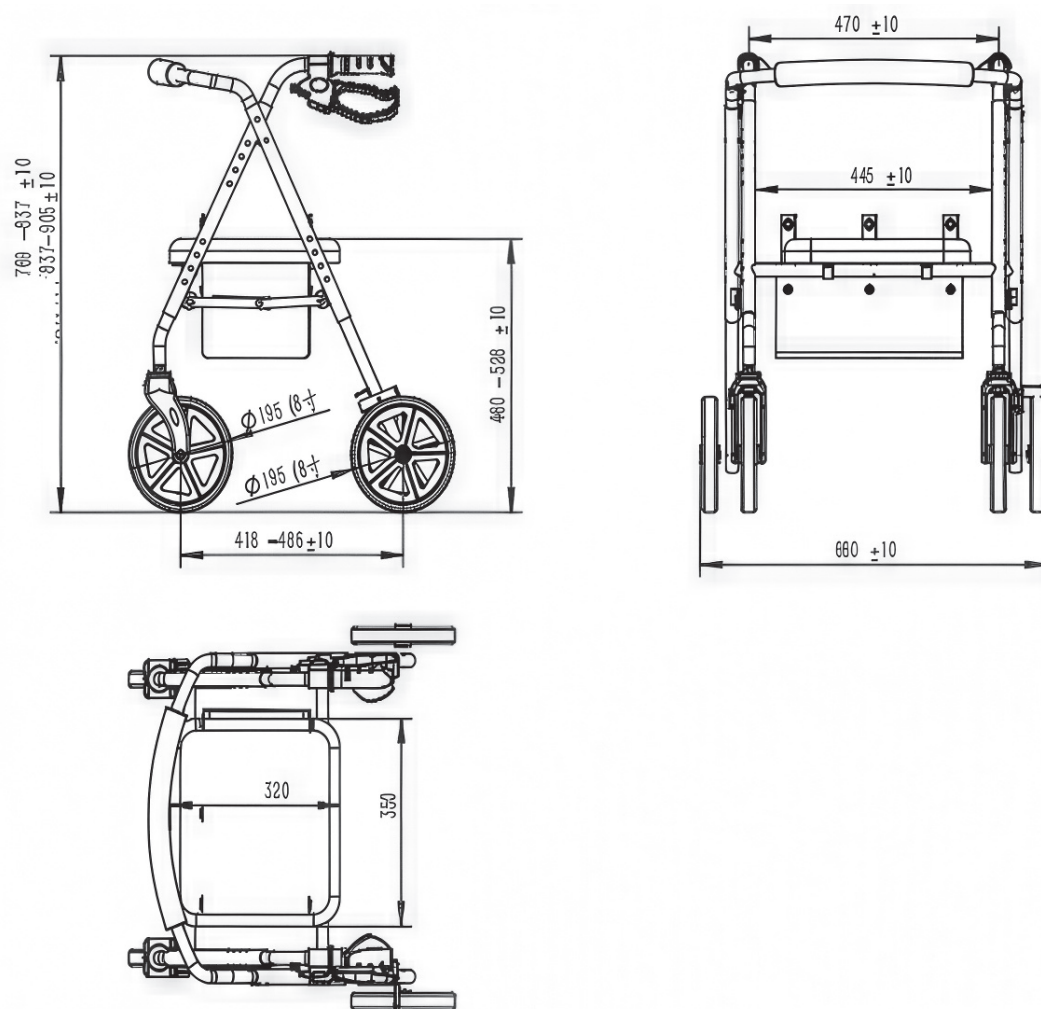
- When removed from the packaging, check that all parts are present as previously listed.
- Make sure there is no visible mechanical damage; if in doubt, contact your distributor.
- Do not use the walker until all the components have been correctly connected.
- Unfolding the walker starts by pulling the frame apart and then locking the horizontal strut hinges by pressing down on the bottom of the small item bag.
- The wheels should be mounted in the lower holes of the side frames.
- Set the correct wheel height so that it is identical for all four of them.
- Place a knob with a screw in the mounting hole on the outside of the side frame and tighten it.
- Check that the wheels are securely fastened to the walker.

Fit handles in the top holes of the side frames.

- Adjust their height so that it is identical for both handles.

- Place a screw in the mounting hole on the inside of the side frame, and the knob on the outside, then tighten it.
- Make sure the handle is firmly attached to the support frame. The handles should be at a height that ensures the comfort and safety of the user, and both should be at the same height.

To change the height of the handles, unscrew the fixing knobs, remove the washers and remove the screws; then adjust the handles to the desired height. Once you have set the correct height, tighten the knob and make sure it is securely fastened to prevent the handles from moving. The backrest must be fitted into the correct brackets on the side frame of the walker (the snaps on the backrest must fit into the holes of the holders). Once the backrest is fitted, pull it to make sure it is correctly fixed.

The user or their carer should consult a healthcare professional if they are unsure whether the medical device is appropriate for the patient in question.

	Maximum permissible user weight – 136kg.
	Any use other than for the intended purpose is considered inappropriate ☐ and DANGEROUS. The manufacturer and/or distributor shall not be liable for damage caused by improper, incorrect and/or unreasonable use.

Safety and usage rules

Maintenance and cleaning

- Regular cleaning of the product should be carried out with water, a sponge and mild detergents intended for plastics, such as soap or a mild cleaning agent approved by the National Institute of Hygiene. It is important to dry the product thoroughly after each use.
- Avoid the use of: strong detergents, abrasives, bleach, gasoline and any solvents, as these can cause damage to the frame, visible defects and lead to a loss of warranty.
- It is a good idea to check the condition of structural components at least once a month, including fixing bolts, as well as pipes for any cracks. In addition, it is advisable to inspect the wheels regularly; if they are found to be worn, they should be replaced with new ones.
- The walker's frame should be cleaned with a damp cloth and the seat can be washed with mild detergents.
- Regularly remove debris from walker's moving components such as wheels, screws and clamp locks.
- Maintaining the cleanliness of all components is key to maintaining the functionality of the product; they must be stored in appropriate conditions. Storage in rooms with high humidity (e.g. baths, laundries) or outdoors, combined with a lack of hygiene, can lead to a rapid loss of functional and aesthetic properties.
- Avoid exposing the walker to extreme weather conditions (rain, snow, intense sun), as some parts may change colour.

Storage and transport conditions

The walker should be stored in a dry and well-ventilated place, protected from moisture and extreme temperatures to prevent damage to the materials. Contact with chemicals that may affect its durability and performance should be avoided. During transport, the product should be secured in such a way as to prevent displacement and damage. Use appropriate protective packaging.

Disposal

Disposal of the product must be carried out in accordance with the applicable local environmental regulations.

Product life

The expected product life is up to 5 years if used in accordance with the information in the instructions for use. The product can be used after this period if it is undamaged and meets the specified requirements for safe use.

Warranty

- The product is covered by a 24-month warranty from the date of purchase, against any defects in material or workmanship.
- The warranty covers the free replacement and/or repair of parts found to be manufacturing defects.
- The warranty does not cover parts subject to normal wear and tear.

- The warranty is no longer valid if the product has been tampered with/modified, if the defect is the result of misuse or if the damage cannot be attributed to the manufacturer.

NOTE

Any serious incident involving the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is residing.

Explanation of symbols

	Conformity of the device with EU Regulation 2017/745 on medical devices		Medical device
	Take a look at the instructions for use		Maximum user weight
	Model		Batch/series number
	Date of production		Warning
	Unique identification code of the device		Medical device manufacturer

Product content: Aluminium, nylon, PCV, PU, steel. **Colour:** Black / navy blue (frame)

Technical data:

- Seat width: 35 cm
- Seat depth: 32 cm
- Overall width: 60 cm
- Overall depth: 70 cm
- Seat height: 53 cm (can be lowered if necessary)
- Handle height from the floor: 91 cm, 88.5 cm, 86 cm, 83.5 cm
- Wheels: Diameter: 20 cm
- Product weight: 7 kg

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ACTION S.A.