Pas magnetyczny z turmalinem

Model: MEDF4-1227; 1228

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Model: MEDF4-1227; 1228

WARNING:

This is a Class I medical device, reusable. Please read the following instructions for use before using.

Intended use: Stabilization and support of the spine, reduction of tension in the lumbar region. Assistance in alleviating pain in the lower back and maintaining proper posture. Magnets and tourmaline crystals reduce muscle tension, improve circulation, and accelerate the recovery process. It decreases the risk of injury and helps increase the stability of the lower spine.

Size: The belt is not universal. Before purchasing, please check the size in the table. To select the appropriate size, measure the circumference of the waist just below the navel.

Composition: Polyester, elastodiene fiber, tourmaline, magnet. Strength of one magnet: 400-500 g.
Properties/Action: Stabilization of the spine. Embedded magnets in the back reduce muscle tension and alleviate pain. Tourmaline crystals improve circulation and accelerate healing. It fastens with Velcro and has tension straps for easy adjustment. It is well-ventilated and absorbs sweat.

Indications: Support for the lower back to alleviate pain, stabilize, and improve posture. Assistance in rehabilitation after injuries or in cases of chronic pain conditions.

Contraindications: Do not use the product if the diagnosis is uncertain.

Do not use if an allergy to one of the component parts has been identified.

Do not place the product in direct contact with damaged skin.

Magnets: pacemakers, electronic implants, hearing aids, pregnancy, and breastfeeding.

Tourmaline: individuals with high fever, pregnancy and breastfeeding, children, individuals with allergies, or altered temperature perception.

Necessary precautions: Before each use, check whether the product is in proper condition.

Do not use the product if it is damaged.

Choose a size appropriate for the patient.

It is recommended that the first application be performed under the supervision of a healthcare professional.

Strictly follow the guidelines and usage protocols recommended by medical personnel.

For hygiene reasons and to ensure effectiveness, do not reuse this product on another patient.

In case of mild or severe discomfort, pain, significant redness of the skin, discontinue use and seek advice from a healthcare professional. Do not use as a primary treatment, but rather to support the proper healing process. If you feel that your body temperature is very high, the product should be removed.

Adverse effects: The product may cause skin reactions (redness, itching, burning, blisters, etc.) and even wounds of varying severity. In case of an allergic reaction, seek medical assistance immediately. Any serious incident related to the product must be reported to the manufacturer.

How to Put on the Product:

Place the product on a flat surface, then wrap it around your lower back, ensuring that the magnets and tourmaline crystals are positioned on your spine. Fasten the Velcro at the front and adjust the length of the tension straps. Check that the belt is comfortable and does not restrict movement.

Maintenance: The product is suitable for washing under the conditions specified in the instructions on the label. It is machine washable at a temperature of 30 degrees; if possible, use a laundry mesh bag. Before washing, make sure to fasten the Velcro. Do not use detergents, fabric softeners, or aggressive chemicals containing chlorine. Do not dry clean. Do not tumble dry. Do not iron. Do not wring out. Dry in a laid-out position.

In case of deformation or structural damage to the product, dispose of it in accordance with applicable regulations.

Disposal: After use, dispose of the product in accordance with applicable regulations. Keep the instructions for future reference.

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