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## EN Information booklet

Particle-filtering half mask EN 149:2001 + A1:2009 FFP2 NR

Filtration of at least 94 % of pollutants/particles from the air, up to a size of 0.6 µm

Limits of use: protection against harmful or low-toxicity and solid, aqueous and non-volatile liquid particles and aerosols, at up to 10 times the individual threshold value in accordance with DIN EN 149:2009-08

CE certification 2016/425/EU FORCE Certification A/S 2605 Brøndby (Code Nr. 0200)

Made in Germany (Vogtland, Saxony)

The full technical data sheet is available in PDF format in the download area of the TechniSat homepage at [www.technimask.de](http://www.technimask.de). The complete text of the EU Declaration of Conformity is available at the following web address: <http://konf.tsat.de/?ID=23083>

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## EN

### How to use

1. Unfold the mask. Please make sure that the nose clip is fully bent open (at a 180° angle). Check for damage and contamination before use.
2. Hold the mask by the ear loops with both hands. Place your chin in the mask first, then pull the mask over your nose. The nose clip should be positioned just above the nose bridge. Pull ear loops behind your ears.
3. Adjust the nose clip with both index fingers from the center outwards.
4. Check correct fit of mask. The Mask should fit tightly and without gaps onto the face.

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Certyfikat CE 2016/425/EU FORCE Certification A/S 2605 Brøndby (Code Nr. 0200)