VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate www.vevor.com/support

STANDARD ASSISTED DRIVING

MODEL:FBL860516

We continue to be committed to provide you tools with competitive price.

"Save Half", "Half Price" or any other similar expressions used by us only represents an estimate of savings you might benefit from buying certain tools with us compared to the major top brands and does not necessarily mean to cover all categories of tools offered by us. You are kindly reminded to verify carefully when you are placing an order with us if you are actually saving half in comparison with the top major brands.

VEVOR®

TOUGH TOOLS, HALF PRICE

STANDARD
ASSISTED DRIVING

MODEL: FBL860516

NEED HELP? CONTACT US!

Have product questions? Need technical support? Please feel free to contact us:

Technical Support and E-Warranty Certificate
www.vevor.com/support

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

INSTRUCTIONS

Preface

First of all, let us express our heartfelt thanks to our customers who have purchased our equipment. This manual is for customer service to make better use of this equipment. Written with this equipment. Hope the company's products and services bring you the same courtesy.

Pay attention

This product must be used in strict accordance with the use method of this manual, misuse of any other methods caused by misuse. The Company does not give any guarantee of any kind, including (but not limited to) the marketability offered to the Company for a particular purpose. Implied warranties with fitness.

The company accepts no liability. The Company shall not be liable for any errors contained in this material, or for any accidental or consequential damages resulting from the provision, actual performance or use of this manual.

This manual contains proprietary information protected by patent law. All rights reserved. Without the written consent of the Company, this manual is not allowed to be photocopied, copied, or translated any part into another language.

The content contained in the manual may be changed without the user's notice.

Manufacturer's liability

The Company shall be held responsible for the safety and reliability of the performance of the equipment only if the assembly operation, returning, improvement and maintenance are carried out by personnel approved by the Company, the electrical equipment concerned conforms to national standards, and the equipment is used in accordance with the operating instructions.

Guarantee

Manufacturing process and raw materials: The Company guarantees that the Equipment (except parts) are in regular use and repair condition, free of manufacturing process and raw materials faults for one year from the date of shipment. None of our inspections is perfect.

PRECAUTIONS

Precautions for use

Warning:

1. If the equipment feels loose, falls or is damaged, please contact the company in time;
2. Keep work area clean and well lit. Cluttered or dark areas invite accidents.
3. Keep children and bystanders away while operating, distractions will cause you to lose control.
4. During normal use, please use with the accompaniment of family or friends.
5. Reasonably use according to one's own situation, and do not overuse when the body is not in a timely manner.
6. BEFORE USE, PLEASE CONFIRM WHETHER THE COMPONENTS ARE COMPLETE, WHETHER THE BRAKE IS SENSITIVE, AND ASSEMBLE THE ENTIRE PRODUCT IN STRICT ACCORDANCE WITH THE INSTRUCTIONS.
7. CLEANING PRODUCTS NEED TO BE WASHED WITH WATER AND DRY IN TIME. IF THE SURFACE COATING OF THE PRODUCT HAS SCRATCHES AND FALLS OFF, REPAIR MATERIALS NEED TO BE USED IN TIME TO ACHIEVE THE BEST ANTI-RUST EFFECT.
8. PLEASE ENSURE THAT THE REAR WHEEL OF THE PRODUCT IS LOCKED TO AVOID SLIDING WHEN THE USER STAYS IN THE SLOPING ENVIRONMENT.
9. PLEASE ENSURE THAT THE FOLDING MECHANISM OF THE SEAT PLATE IS LOCKED WHEN USING THIS PRODUCT.

SAVE THESE INSTRUCTIONS

PRODUCT PARAMETERS

Product Name	Standard assisted driving
model	FBL860516
Load bearing capacity/lb	350lbs

PRODUCT COMPONENTS

1		Left main frame	1	2		Right main frame	1
3		Hind foot canal	2	4		Front left foot wheel set	1
5		Front right foot wheel set	1				

INSTALLATION STEPS

1. As shown in the figure: Unfold the left and right main frames, hold down the marbles on the right frame, align them with the left main frame, and push them inward until the marbles pop out.

2. As shown in the figure: Insert the front left foot wheel assembly into the main frame, align it with the hole position, hold down the marble and push it inward until the marble pops out.

3. As shown in the figure: Insert the front right foot wheel assembly into the main frame, align it with the hole position, hold down the marble and push it inward until the marble pops out.

4. As shown in the figure: Insert the rear foot tube into the left side of the main frame, align it with the hole position, hold the marble and push it inward until the marble pops out.

5. As shown in the figure: Insert the rear foot tube into the right side of the main frame, align it with the hole position, hold the marble and push it inward until the marble pops out.

Nice Service: 100% tested before leaving the factory and 24-hour friendly email customer service.

Manufacturer: Zhongshan Bliss Medical Instrument Co., Ltd

Address: 3-6F & 7F-1, No. 63, Tongxing East Road, Yumin Community
Xiaolan, Zhongshan, Guangdong, 528414, CHINA .

Imported to AUS: SIHAO PTY LTD. 1 ROKEVA STREET EASTWOOD NSW 2122
Australia

Imported to USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim Place,
Rancho Cucamonga, CA 91730

UK	REP
----	-----

YH CONSULTING LIMITED.
C/O YH Consulting Limited Office 147,
Centurion House, London Road,
Staines-upon-Thames, Surrey, TW18 4AX

EC	REP
----	-----

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VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate

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Tillverkare: Zhongshan Bliss Medical Instrument Co., Ltd
Importerad till AUS: SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD NSW 2122
Importerad till USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim Place, Rancho
E-CrossStu GmbH