Technical Support and E-Warranty Certificate www.vevor.com/support

PET WHEELCHAIR USER MANUAL

We continue to be committed to provide you tools with competitive price.

"Save Half", "Half Price" or any other similar expressions used by us only represents an estimate of savings you might benefit from buying certain tools with us compared to the major top brands and does not necessarily mean to cover all categories of tools offered by us. You are kindly reminded to verify carefully when you are placing an order with us if you are actually saving half in comparison with the top major brands.

VEVOR[®]

TOUGH TOOLS, HALF PRICE

PET WHEELCHAIR

Model: SS2-XS, SS2-S, SS2-M, SS2-L

NEED HELP? CONTACT US!

Have product questions? Need technical support? Please feel free to contact us:

Technical Support and E-Warranty Certificate
www.vevor.com/support

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

SPECIFICATIONS

Model	SS2-XS
Product Size	445×260×375mm
Load	15lb

Model	SS2-S
Product Size	495×295×475mm
Load	15-35lb

Model	SS2-M
Product Size	630×360×495mm
Load	35-48lb

Model	SS2-L
Product Size	795×440×605mm
Load	48-88lb

Components

Sheff

Wheels

Toolkit

Dogleash

Dog leg fixed belt


NOTE:

This product has undergone multiple tests and user feedback, and has shown excellent results in helping pets recover their health.

Not all pets will use it at the beginning of wearing it (some pets may have a fear of unfamiliar equipment). At this time, it is necessary to let the pet adapt to the wheelchair, comfort the pet, and start supporting the pet to walk slowly. Then, use a traction rope to pull the pet until the pet can walk freely.


Instructions for use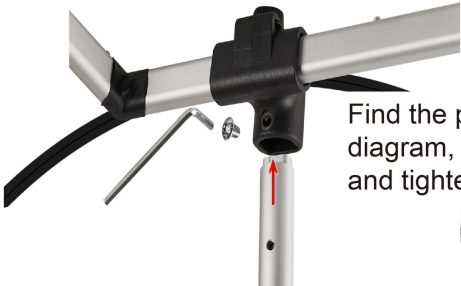

Find the pole and tool shown in the diagram, insert the pole into the hole, and tighten the screw with L tool.

Loosen the nut, press the ball, and adjust to the appropriate gear.

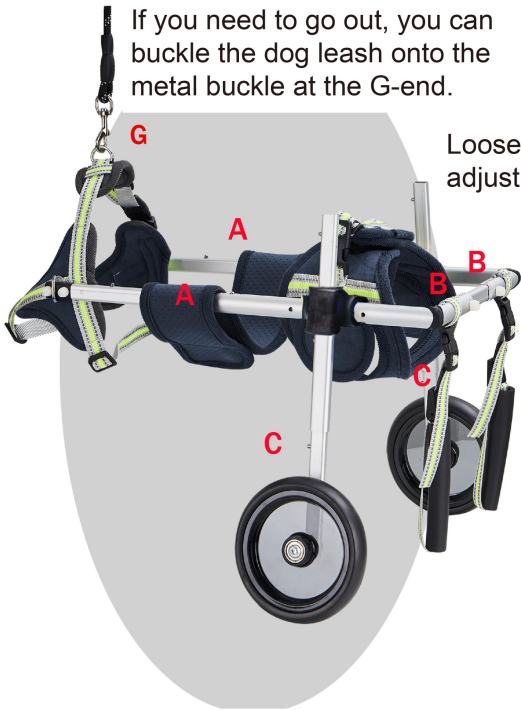If you need to go out, you can buckle the dog leash onto the metal buckle at the G-end.

AA corresponds to length
BB corresponds to width
CC corresponds to height

Put the dog's rear leg into hole E and adjust the dog's hip circumference.

If the dog's calf is injured, it can be placed on a sponge.

Tie the straps as shown in the diagram.

Adjust the belt size to support the dog's abdomen.

Manufacturer:Shanghaimuxinmuyeyouxiangongsi
Address: Shuangchenglu 803nong11hao1602A-1609shi, baoshanqu, shanghai 200000 CN.
Imported to AUS: SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD NSW 2122 Australia
Imported to USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim Place, Rancho Cucamonga, CA 91730

EC	REP
----	-----

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YH CONSULTING LIMITED.
C/O YH Consulting Limited Office 147, Centurion House,
London Road, Staines-upon-Thames, Surrey, TW18 4AX

VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate

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Importowane do AUS: SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD
Importowane do USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim
E-CrossStu GmbH
C/O YH Consulting Limited Biuro 147, Centurion House,