Model: MEDF4-1239

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Lattice Upper Limb Sling

Model: MEDF4-1239

WARNING: This is a Class I medical device, reusable. Please read the following instructions for use before use.

Intended use:

Immobilization and support of the arm and forearm immobilized in a cast after surgeries or injuries. It provides relief for the arm in case of injuries such as fractures of the forearm bones or dislocations of the elbow joint. It enhances patient comfort during recovery by providing adequate support. It can also be used as a temporary dressing in injury situations, serving as a practical solution for wound stabilization. Additionally, it allows for proper positioning of the limb and supports the healing process.

Size: Universal, fits both the right and left hand.

Composition: Polyester.

Properties:

The sling effectively supports and relieves the arm that is immobilized in a cast, increasing patient comfort during treatment and promoting safe recovery. It can be used as a temporary dressing for injuries such as elbow dislocations and forearm fractures. Made from mesh fabric, the sling provides good airflow, making it comfortable to wear, especially during the summer months. The adjustable shoulder strap allows for perfect height adjustment to meet the individual needs of the patient, enhancing user comfort. The hook-and-loop fastening system enables quick assembly, making the use of the sling simple and convenient.

Indications:

Immobilization of the upper limb, support for the arm while wearing a cast, temporary dressing in case of forearm fractures or elbow joint dislocations. Contraindications: Should not be used in the presence of open wounds, abrasions, skin eczema, or varicosities.

Necessary precautions:

Before each use, check whether the product is in proper condition.

Do not use the product if it is damaged.

It is recommended that the first application be performed under the supervision of a healthcare professional.

Strictly follow the guidelines and usage protocol recommended by medical personnel.

For hygiene reasons and to ensure effective operation, do not reuse this product on another patient.

In case of mild or severe discomfort, pain, changes in limb size, sensory disturbances, or changes in the color of the fingertips, remove the product and seek advice from a healthcare professional.

Adverse effects:

The product may cause skin reactions (redness, itching, burning, blisters, etc.) and even wounds of varying severity. In case of an allergic reaction, seek medical assistance immediately. Any serious incident related to the product must be reported to the manufacturer.

How to put on the sling:

Place the sling on a flat surface. Then, holding the arm in the proper position, gently slide it into the designated area of the sling. Use the height adjustment strap to customize the sling to your needs. Once the arm is in the sling, simply fasten the hook-and-loop fastening system, which will provide stable support for the limb.

Maintenance:

The product is suitable for washing under the conditions specified in the instructions on the label. It is machine washable at a temperature of 30 degrees; if possible, use a laundry mesh bag. Before washing, make sure to fasten the Velcro. Do not use detergents, fabric softeners, or aggressive chemicals containing chlorine. Do not dry clean. Do not tumble dry. Do not iron. Do not wring out. Dry in a laid-out position. In case of deformation or structural damage to the product, dispose of it in accordance with applicable regulations.

Disposal:

After use, dispose of the product in accordance with applicable regulations.

Mona
ul. Leszczynowa 8, 62-070
Dąbrówka, Poland
+48 607 106 591,
www.fit4med.com

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