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**+48 607 106 591,**

**www.fit4med.com**

# Immobilizing Wrist, Hand, and Finger Orthosis

**Model: MEDF4-1237;1238,1271;1272**

**NOTE:** This is a Class I reusable medical device. Please read the following instructions for use before using.

**Intended Use:** Immobilization and relief during the treatment of injuries, as well as in the convalescence period after an injury. It maintains the appropriate joint in the correct position and relieves injured bones, joints, and muscles through stable immobilization without simultaneous heavy loading. It helps maintain proper body posture. Intended for both therapeutic use after an injury and prophylactic use during increased physical activity.

**Size:** The brace is not universal. Before making a purchase, please check the size in the table. Additionally, the brace is designed specifically for either the left or right hand, so proper selection is required.

## Composition:

Polyamide-Polyurethane-Elastane-Polyester-Aluminum

## Properties/Action

Immobilizes the wrist and the 4th and 5th fingers using two rigid formable stays.

The stay on the palm side is shaped at a 20-degree angle at the finger level.

Allows complete opening for easy insertion of the hand and fingers.

Separates the fingers to prevent chafing of immobilized fingers against each other.

## Indications

Postoperative/post-traumatic immobilization of the 4th and 5th metacarpal bones

Non-surgical fracture of the 4th and 5th metacarpal bones

Consequences of fracture and sprain of the 4th and 5th fingers

After complete immobilization in a cast

## Contraindications:

Do not use if the diagnosis is uncertain

Do not use in case of known allergy to any of the product's components

Do not place the product in direct contact with damaged skin

Do not use in case of unstable fractures

## Necessary Precautions

Check the product's condition before each use.

Do not use the product if it is damaged.

Select the size that fits the patient.

It is recommended that the first application is supervised by a healthcare professional. Strictly follow the guidelines and usage protocol recommended by medical personnel. For hygiene reasons and to ensure effectiveness, do not reuse this product for another patient. If you experience any slight or severe discomfort, pain, changes in limb size, sensory disturbances, or changes in finger color, remove the product and seek advice from a healthcare professional.

## Undesirable Side Effects

The product may cause skin reactions (redness, itching, burning, blisters, etc.) and even wounds of varying severity. In case of an allergic reaction, seek medical assistance immediately. Any serious incident related to the product should be reported to the manufacturer.

## How to Apply

Place the orthosis on a flat surface. With the hand open and facing up, gently insert the 4th finger into the designated splint.

Fasten the strap to immobilize the 4th and 5th fingers together, which will lock the upper part of the orthosis at the finger level.

Turn the hand to the side and fasten the middle strap. Ensure that the stay on the palm side fits snugly against the hand.

Fasten the last strap by threading it through the designated buckle and tighten it to secure the fit.

To avoid potential circulatory problems, do not tighten the straps around the fingers too tightly.

## Maintenance

The product is washable under the conditions specified in this manual and on the label. It is suitable for machine washing at 30 degrees. If possible, use a laundry bag. Before washing, make sure to fasten the Velcro straps. Do not use detergents, softeners, or aggressive agents containing chlorine. Do not dry clean. Do not tumble dry. Do not iron. Gently squeeze to remove excess water and lay flat to dry.

Keep this instruction manual for future reference. In case of deformations or structural deformations of the product, dispose of it in accordance with applicable regulations.

## Disposal

After use, dispose of the product in accordance with applicable regulations.

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