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**Materiał:** 50% bawełna, 50% poliestr

## **EN - Tourmaline Self-Heating Neck Belt**

**INSTRUCTIONS FOR USE:** Wrap around the neck with the lighter side facing inward, ensuring the belt does not restrict movement. Make sure the selected body area is relaxed and unrestricted to reduce the risk of later damage. To enhance the heating effect, moisten the inner material of the belt before use.

**WARNING:** The product contains strong magnetic fields that may be harmful to individuals with pacemakers. Do not use the belt for more than 2 hours per day. It is recommended to check the heated area every 15 minutes. If redness, irritation, or discomfort occurs, remove the belt immediately. In case of persistent pain in the heated area, it is essential to obtain a proper diagnosis. Consult a doctor about the safety of using the product. Do not use the belt during sleep, near heat sources, or while exercising. Avoid applying additional pressure to the belt. Ensure proper air circulation around the belt. If skin discoloration occurs, discontinue use immediately. Not suitable for children, elderly individuals, or those unable to use the belt independently. Do not use on irritated or damaged skin.

Do not use the product in combination with creams applied to the skin or in cases of rheumatoid arthritis, poor circulation, nerve damage, sensory disorders, skin diseases, vascular disorders (e.g., Raynaud's disease), diabetes, or pregnancy. For external use only. Do not use the belt if allergic to latex. The product packaging contains natural latex rubber, which may cause skin allergic reactions. Plastic packaging can be dangerous. To avoid the risk of suffocation, keep it out of reach of children and infants.

**Washing Instructions:** Hand wash only. Do not bleach. Hang to dry. Do not iron. Do not dry clean.

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