
axion GmbH
Mollenbachstraße 13
D-71229 Leonberg
GERMANY
www.axion.shop

www.axion.shop/symbols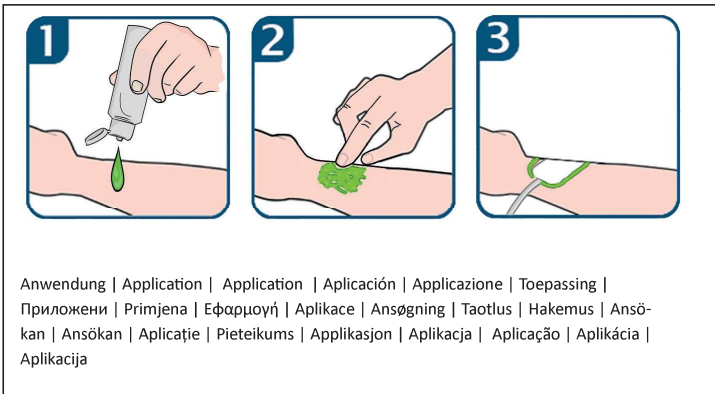

Anwendung | Application | Application | Aplicación | Applicazione | Toepassing |
Приложени | Primjena | Εφαρμογή | Aplikace | Ansøgning | Taotlus | Hakemus | Ansö-
kan | Ansökan | Aplicație | Pieteikums | Aplikasjon | Aplikacija | Aplicação | Aplikácia |
Aplikacija

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GBR | INSTRUCTION FOR USE ELECTRODE GEL: Conformity marking | Manufacturer | UDI marking | Observe instructions for use | **Intended use:** Electrode-accessories
(Contactgel) for optimal transmission of electrical impulses (conductivity) in connection with TENS and EMS electrodes for therapeutic and diagnostic medicine. The product is intended for OTC use. The pa-
tient group is not restricted, if necessary, by the user (doctor) | **Indications:** For external use on intact skin | **Contraindications:** Do not use if sensible to parabens or phenoxyethanol | **Attention**
- **Observe accompanying documents/warning notes:** Do not use on mucous membranes, eyes or open wounds | Keep away from children | Do not clean nor disinfect | Store in a
cool and dry place, do not expose to direct sunlight | Minimum shelf life | Only use within the shelf life after opening. | Non-sterile | **Ingredients:** Water, carbomer, sodium hydroxide, sodium
chloride, propylene glycol, phenoxyethanol, methylparaben, ethylparaben | Does not contain formaldehyde | **MD** medical device | **LOT** Batch code | **REF** Article number

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