GET READY!

GB	PICO Electric breast pump	2
PL	Laktator elektryczny PICO	8
BG	Електрическа помпа за кърма PICO	14
CZ	Elektrická odsávačka mléka PICO	20
DE	Milchpumpe PICO	26
EE	Elektriline rinnapump PICO	32
ES	Extractor eléctrico PICO	38
FR	Tire-lait électrique PICO	44
GR	Ηλεκτρική συσκευή άντλησης μητρικού γάλακτος PICO	50
HR	Električna pumpa za dojke PICO	56
HU	Elektromos, kagylós mellszívó készülék PICO	62
IT	Tiralatte elettrico PICO	68
LT	Elektrinis pientraukis PICO	74
LV	Elektriskais krūts piena sūknis PICO	80
NL	Elektrische borstkolf PICO	86
PT	Bomba de leite elétrica PICO	92
RO	Pompă de sân electrică PICO	98
RU	Молокоотсос электрический PICO	104
SE	Elektrisk bröstpump PICO	110
SK	Elektrická odsávačka materského mlieka PICO	116
UA	Молоковідсмоктувач електричний PICO	122

PICO

Electric breast pump


Congratulations on the birth of your baby!

You will soon discover that breastfeeding your baby is a very rewarding experience. The intelligent all-in-one breast pump simulates the baby's sucking action, providing a comfortable and pain-free service for breastfeeding mothers. For proper and safe breast pump use, please read these instructions carefully and keep them handy for future reference.

FEATURES

- Smart all-in-one breast pump, compact and easy to carry. Allowing mothers to conveniently collect breast milk no matter where they are.
- Large capacity lithium battery with USB interface, no power adapter required for use when going out.
- Three modes in one machine. Massage, pumping, lactation. Realistic breast pumping, simulating the breast pumping made of the baby sucking, stop ping and resting, sucking milk easily and effectively.
- High quality food grade non-toxic plastic, BPA free to let you and baby can use it with peace of mind.
- Completely achieved '0 backflow'. If the bottle accidentally overturned, the milk will not flow to the host resulting in damage to the machine.

PRODUCT COMPONENT DIAGRAM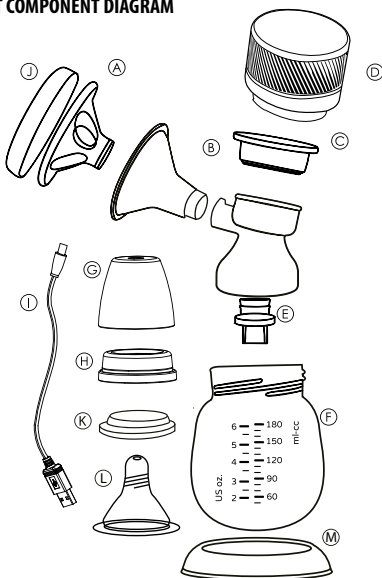

serial number	name	material	heat resistance temp.	cold temp.	qty
A	silicone horn	silicone	200°C	-20°C	1
B	breast shield	PP	110°C	-20°C	1
C	diaphragm	silicone	200°C	-20°C	1
D	host	—	—	—	1
E	leak-proof valve	silicone	200°C	-20°C	2
F	baby bottle – high configuration	PPSU	180°C	-20°C	1
	baby bottle – low configuration	PP	110°C	-20°C	1
G	bottle cup	PP	110°C	-20°C	1
H	nipple set	PP	110°C	-20°C	1
I	USB cable	—	—	—	1
J	dustproof cover	PP	110°C	-20°C	1
K	sealing disc	silicone	200°C	-20°C	1
L	bottle nipple	silicone	200°C	-20°C	1
M	stabilizing base	PP	110°C	-20°C	1

PRECAUTIONS

- Do not use water to clean or wipe the main unit with a damp cloth. If the host accidentally falls into the water, do not power on or touch it, and immediately unplug the power adapter from the power socket.
- Be sure to unplug the power adapter from the power outlet after use.
- Do not use the inhaler in the bathroom or near water.
- Carefully check this product, including the main unit and other inhaler accessories, before each use. If you find any problems or damage, please stop using it.
- If the main unit does not operate normally, please refer to 'Troubleshooting'.
- Please do not leave the main unit and power adapter in a place where high temperature may occur (such as in a car)
- The pump is made up of small parts, be careful not to let your baby put any parts in the mouth.
- If you experience pain with the inhaler, ask your doctor or breastfeeding expert for advice.
- Non-professionals should not disassemble the device by themselves.
- Only use the parts that came with the inhaler.
- Keep this product out of the reach of children.

HOW TO USE AN ELECTRIC BREAST PUMP

1. It is recommended to apply a hot towel to the breast for 5 minutes before use, which is conducive to promoting the smooth flow of the breast, so that the pumping can be better and more effective.
2. Make sure that the parts of the breast pump are thoroughly sterilized and assemble the breast pump correctly as instructed. Mother should sit in a comfortable and relaxed position, using the center of the breast pump

cup to align the nipple and close to the breast. Make sure no air enters to ensure proper suction.

3. Touch the key to enter the massage made 1 by default to stimulate and massage the mammary glands. If you need to increase the suction, you can press '+' to increase the level.

4. When milk starts to flow or you feel squirting, select the pumping made to find your personal comfort suction.


5. After pumping, turn off the power and unplug it.

6. Please disassemble and clean related accessories. (Except the host and USB cable).

7. If you need to use outdoor, please fully charge the host first. If the device flashes for 5 seconds, it will automatically shut down, indicating that the power is exhausted. Please connect the adapter to charge.

ATTENTION! The milk in the bottle should not be too full and it should not exceed the maximum capacity of the bottle. If the maximum capacity is reached, please replace the bottle in time.

INSTRUCTIONS

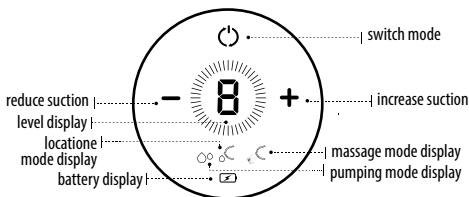SWITCH MODE

Long press to on/off, short press to switch the made.

LEWEL DISPLY

Displays the current mode level, highest level 9 and lowest is level 1.

INCREASE / DECREASE DISPLAY

Short press to increase and decrease level.

MASSAGE MODE

Showing "●●●" delegate works in massage mode.

PUMPING MODE

Showing "○○○" delegate works in pumping made.

LACTATION MODE

Showing "○○●" delegate works in lactation made.

TIPS

1. Automatic shutdown after 30 minutes of continuous work.

2. If it is working, "▽▽▽" flashing and automatically shut down, indicating that the battery is exhausted, please charge it in time.

3. "▽▽▽" Flashes when charging and stops when charging is complete.

TROUBLESHOOTING AND FAQ

HOST FAILS TO START

- Check whether the battery power is exhausted, if there is no response to the power-on, the power is too low.

NO SUCTION

Use the mobile phone charger to check if the adapter is damaged.

INSUFFICIENT SUCTION

- Check if the leak-proof valve is damaged or not installed properly.
- Check whether the breast shield silicone fits well with the breast.

THE BREAST PUMP HAS SUCTION BUT IT IS TOO DIFFICULT TO PUMP

- This is caused by unobstructed breast or mental stress.
- First apply a hot towel for 10 minutes, then use the massage level to pump the milk. After 10 minutes of suction, adjust the level. If you still can't pump, squeeze and massage the breast with your hands after applying heat, and squeeze as much milk as possible to unclog the breast.

BREASTFEEDING PAIN

- When breastfeeding is painful, try to straighten the horn silicone cover to ensure that the nipple is aligned with the silicone mouth. Use a lower level. As the number of pumping increases, the body will adapt to the suction sensation, and there will be no pain.

DISINFECTION METHOD

STERILIZE WITH STERILIZER

All breast pump parts can be sterilized with a sterilizer.
(Except for the host and USB cable)

STERILIZE WITH STEAM IN THE MICROWAVE

When sterilizing in the microwave use a sterilizing container or sterilizing bag. There must be water in the container during sterilizing.

BOILED

Disinfect the breast pump in boiling water for 5 minutes daily. Pay attention to the amount of water not lower than the parts to be sterilized. The parts should not directly touch the bottom of the heating pot to avoid scalding the parts.

MILK BOTTLE CLEANSING LIQUID

Please use milk bottle cleansing liquid for disinfection.

OTHERS

- ○ ○ Please do not use dry heat, UV light or dishwashers, or dryers to sanitize to avoid damage parts of the breast pump.

NOTES

- Dry thoroughly after disinfection
- When using disinfection equipment, please follow the instructions

provided

- Be careful not to damage the tip when cleaning the milk on the valve
- Do not use hard brushers or stain removers

PLEASE DO NOT WASH ELECTRONIC PARYS WITH WATER.

PLEASE AVOID USING DAMP CWTW TO WIPE ELECTRONIC PARYS.

STORAGE AND HEATING OF BREAST MILK

STORAGE PLACE	STORAGE TIME	
	FOR HEALTHY BABIES	FOR PREMATURE AND ILL BABIES
In room temperature	8 – 12 hours	1 – 4 hours
In a fridge (5°C)	3 – 4 days	24 – 48 hours
In a freezer compartment (-10° C)	2 weeks	1 week
In a freezer (-18° C)	6 – 12 months	3 – 6 months

IMPORTANT: Please make sure any expressed breast milk is correctly stored! Incorrectly stored breast milk loses its nutritive value and may even be harmful to your baby.

We recommend Babyono sterile bags for breast milk storing and freezing.

NOTE

- Thawed milk must not be frozen again. Any unused milk should be discarded.
- Do not mix any thawed breast milk with freshly pumped breast milk.
- You can freeze only fresh breast milk.
- Breast milk should be defrosted slowly. To do this, frozen milk should be placed in a refrigerator until it takes a liquid form.
- To warm breast milk, it is recommended to use a water bath with a temperature not exceeding 50°C.
- To preserve its nutritive value, breast milk must not be boiled, heated up directly on a cooker or in a microwave!
- Ready to serve breast milk should have the human body temperature, i.e. no more than 37°C.

ASSEMBLY METHOD

Before starting to assemble the breast pump, wash your hands and make sure to sanitize all parts before use.

Install the silicone horn into the breast shield.
(As shown in Figure 1).

Insert the leak-proof valve into the breast shield, and then screw the milk bottle into it.
(As shown in Figure 2).

Snap the silicone diaphragm into the breast shield. (As shown in Figure 3).

Push the host forward into the diaphragm and screw it clockwise. (As shown in Figure 4)

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