AD-05

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User Manual

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Handleiding

PL

EN

DE

HU

NL

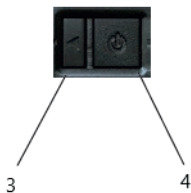
OVERMAX

You • unlimited

1

2

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Drogi Kliencie!

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Introduction

Dear customer!

Thank you for placing trust in us and choosing Overmax.

Thanks to high-quality materials and modern solutions, we provide you with a product perfect for everyday use. We are convinced that the product will satisfy your requirements as it has been manufactured with utmost care. Before you use the product, please read this manual carefully.

Should you have any remarks or questions concerning the product, please do not hesitate to contact us: pomoctechniczna@overmax.pl

Read the manual before use.

Personal breathalyzer AD-05 is a mobile device for testing alcohol content from a breath sample. The product has been equipped with an electrochemical sensor which estimates blood alcohol content in a few seconds!

Description:

1. Folding mouthpiece
2. LCD display
3. Options select key
4. On/off key

1. USE

NOTE!

- Before you use the device, wait for at least 15 minutes after drinking alcohol or smoking cigarettes.
- Before use, rinse your mouth with water.
- It is necessary that you wait for 15 minutes. If you fail to follow this rule, the test results may not be reliable. The readout value „will rise” fast and then gradually „drop”. This results from stubborn alcohol remains in the oral cavity which distort values measured.
- It is advisable to take a breath test at the room temperature in order to assure optimal accuracy.

- It is also advisable to store the device at the room temperature (when the device is not used for a long time) to keep the sensor in a proper condition.
- Do not use and do not store the device at extreme temperatures and in highly humid conditions.
- To use the device, gently press and slip off the battery cover. Put two AAA 1.5V alkaline batteries in and close the battery cover. Using non-alkaline batteries may result in imprecise results

HOW TO USE:

- Remove the folding mouthpiece on the left side of the device.
- **Switching ON:** press on/off key (4) and hold for 2 seconds to switch the device on. The device initiates the sensor heating procedure. The second hand starts counting. 

Alcohol content from breath sample is now taken:

- **Preparing:** when counting is over, you will see a flashing „rdy“ (ready) which means the device is now ready to collect the sample of breath.  **UThe device will wait for 8 seconds for the sample of breath. If it is not detected, the device will show zero - 0.00.
- **Blowing:** blow continuously into the mouthpiece. A long „beep“ sound suggests that the device collects the air. Keep on blowing until you can hear a loud sound in the form of a „double beep“. At this point you have to stop blowing in.
- The device starts analyzing the breath and shows „“ (calculating).
- Wait for a while to see the result in ‰ on the display. 
- **Switching off:** press on/off key (4) and hold for 2 seconds to switch the device off.

IMPORTANT!

1. The test result will be set and displayed in per mil (‰). If you wish to change the alcohol level display format, press the options select key (3) to switch into the desired format.

2. The device may emit a long sound when taking the breath sample and give 0.00. This happens because it is possible to measure a very low content of other substance in your breath (e.g. bad smell). By default, if the device detects no alcohol content, it will emit a long sound and the result will be 0.00.
3. In some cases the device may display "0.00". It happens when the conditions of the test are too cold or too hot. Before you retake the test, wait for a dozen minutes until the sensor inside the device reaches the room temperature.
4. The device will switch off automatically after 30 seconds. You can also switch the breathalyzer off by pressing on/off key (4) and holding for 3 seconds.

RECOMMENDED WAITING TIME BETWEEN TESTS

- It is advisable to wait for a few minutes (3-5 minutes) between every breath test (for low alcohol level).
- With regard to high alcohol level, please wait for at least 10 minutes before you retake the test. This way you let stubborn alcohol remains disappear from the sensor. This is also how you can help keep the proper condition of the sensor and maintain test result reliability.

2. TEST MEMORY MODE:

To check the total number of breathalyzer tests, please follow the description below:

- When counting, press on/off key (4) to display the total number of tests performed. This number helps you decide about time for calibration. For more details, see „Calibration“.
- If you want to reset this counter, long-press the options select key (3) until you can see the counter shows zero.
- Displaying last 16 tests:
Press the options select key (3) to display the history of tests (they will be displayed on the screen, marked with „1“ serial number, and then with relevant result)
- To leave the test memory mode, press on/off key (4) again.

3. SYSTEM MESSAGES

Message	Meaning of the message
H	The device has detected high alcohol level (above 2,00‰ BAC / 0.200%BAC /1.00 mg/L)
Fla	Insufficient or uneven breath sample detected. Please retake the test (see: „how to use“)
Snr	The sensor of the device is not ready. This happens when high alcohol or dirt from the last test remains on the sensor. This being the case, switch the device off and wait for a couple of minutes before another test.
rPt	The device was unable to finish the test. Retake the test. Other cause is high ambient temperature. In such case please retake the test (see: „how to use“)
Lo	Low battery. Use new AAA alkaline batteries
Out	The temperature of the test was too low or too high. Restore room temperature for the device. Before another test, wait for a couple of minutes until the device sensor reaches the room temperature.

EN

4. SAFETY PRECAUTIONS

- Wait for at least 15-20 minutes if you have drunk alcohol or smoked cigarettes before you take the test. Remember to rinse your mouth with water.
- To obtain precise results, it is advisable to calibrate the device every 12 months or 300 tests.
- The device operates at the temperature of 10 – 40°C, with optimal precision for tests at room temperature (i.e. ca. 20 – 26°C).
- If the test result is lower than 0,10‰ or 0,10 % BAC or 0,050 mg / L, it is natural when you didn't drink any alcohol. It happens because in some cases it is possible to detect other substance than alcohol from the breath sample.
- Do not use the device if you wish to specify and decide about driving vehicles. The result displayed is for information purposes only. The manufacturer and retailer waive any legal liability for measured/estimated test value.

5. MAINTENANCE

To keep the device in a good condition, it is important that you use the device at least once a month.

6. CALIBRATION

EN

- Precision of alcohol level testers may change after 6-12 months of use, depending on the number of tests done and working conditions.
- If the device shows inconsistent test results, fails to show any test results or displays extremely high or low results, the product must be calibrated.
- To calibrate it, contact the service center.

The pictures are for reference only. The real design of the products may differ from the pictures presented.

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product name: AD-05

description: breathalyser


meets the essential requirements of the following Directives:

2014/30 / EU - EMC Directive

and meets the requirements of the following harmonized norms and standards:

EN 55011: 2016 - Industrial, scientific and medical equipment - Radio frequency disturbance characteristics - Limits and measurement methods

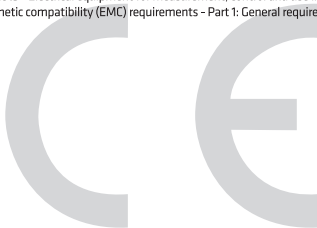EN 61326-1: 2013 - Electrical equipment for measurement, control and use in laboratories - Electromagnetic compatibility (EMC) requirements - Part 1: General requirements

This UE declaration of conformity has been issued under the sole responsibility of the manufacturer. The document together with the technical documentation is the basis for marking the product with the mark 

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Warranty conditions:

1. The Overmax brand with its registered office at A. Kręglewskiego 1, 61-248 Poznań, Poland guarantees that the equipment is operational in accordance with the technical-operating conditions described in the manual.
2. The warranty is granted for a 24 months period from the date of sale. If the Buyer is a company (VAT invoice), the warranty is granted for a 12-month period.
3. The warranty is valid in the country of purchase.
4. Revealed manufacturing defects shall be removed during the warranty period free of charge by the Authorised Manufacturer Service.
5. Any defects or damages discovered and reported within the warranty period will be removed free of charge within 21 working days, but in justifiable cases (for instance, spare parts must be brought from abroad) this period may be extended by another 30 days.
6. Any defects or damages to the equipment should be reported and delivered to the service site immediately after being revealed.
7. Warranty for LCD displays, touch screens, and batteries is 6 months from the date of purchase. In case of displays of a diagonal ranging from 2.5" to 10.1", the presence of up to 3 incorrect pixels (white, black, green, red or blue) is acceptable.
8. Accessories included with the devices (e.g. Chargers, remote controls, keyboards, mice, earphones, cables) are under 3 month warranty from the date of purchase.
9. In case of an unjustified complaint (complaint of efficient equipment or the mechanically damaged one), the person who reports the complaint will be charged for transportation cost.
10. The customer is entitled to replacement of the equipment to another with the same or similar specifications if the service determines that the removal of defects is not possible.
11. The service is not responsible for any loss of data.
12. The warranty will be respected only if the customer attaches (to the faulty device) a damage description, all of the accessories received at the moment of the equipment purchase, and proof of purchase containing the date of sale.
13. The warranty does not cover: mechanical, electrical, thermal or intentional damages and defects caused by them, as well as any defects and disadvantages arising from the use of the device that do not affect

its proper operation, damages resulted from improper usage or non-compliance of the instructions described in the manual. This also applies to signal fading due to the distance from the transmitter, poor antenna installation and damages of the equipment, due to fact, that it has not been delivered to the service in a safe container.

14. The Service may refuse to repair products in case of damage to the warranty seal or any traces of unauthorised repair.
15. If the fault is not covered by the manufacturer's warranty, paid repair may be offered by the Service.
16. All reclaimed items are checked by the Authorised Service for correct operation.
17. Neither BrandLine Group Sp. z o.o. nor the Authorised Service is responsible for any damages or losses arising from the inability to use the device being repaired.
18. The equipment that has not been collected from the service site within 3 months can be charged with storage cost appropriate to the storage time.
19. Delivery of the equipment which is incomplete or lacks of proper packaging, is equivalent to the failure of the warranty conditions by the Buyer, and it can provide a basis to refusal of the repair or extension of the repair period.
20. In case of sending a product for repair, when no defect is found, the expenses for the expert assessment shall be fully borne by the consumer.
21. If a product (sent for repair) works properly, and the reason of its invalid operation is a low battery, the consumer shall bear the cost of the expert assessment.
22. The product warranty shall not exclude, limit or suspend the Customer rights resulting from the guarantee.

List of Authorized Service Centers: **www.overmax.eu/en/service**

Software updates are available on our website www.overmax.eu in "Technical support" tab.

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