Lecoxen®

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Composizione/Composition

Deionized water, Cyclopentasiloxane, Dimethicone, Paraffinum liquidum, Lauryl PEG/PPG-18/18 methicone, Dodecene, Isostearyl alcohol, PEG/PPG-23/50 Copolymer, Methylalkyl cyclosiloxanes, Aesculus hippocastanum (Horse chestnut) bark/seed extract, Propylene glycol, Glycerin, Sodium Chloride, Allium cepa (onion) bulb extract, Zeolite, Benzyl alcohol, Dehydroacetic acid, Tocopheryl acetate, Dimethicone crosspolymer, Sodium benzoate, PEG-6 Caprylic/capric glycerides, Citric acid, BHT, Triticum vulgare (wheat) germ oil.

Formato: Tubo da 30 ml e

Lecoxen®

Protective and healing adjuvant cream

Indications

Cream with protective and moisturizing action on damaged skin and perilesional margins, able to contribute to the formation of an optimal moist microenvironment to promote wound healing. Its use is also recommended after wound closure to prevent the formation of unsightly scars (keloids).

Directions for use

1. Make sure the affected wound is clean.
2. Gently remove any excess exudates before applying the cream.
3. Remove the cap and exert pressure on the tube to release an adequate amount of cream to be applied on the affected area and periwound margins.
4. Gently remove any excess cream with a clean tissue or gauze. For best results, keep **Lecoxen cream** in constant contact with the wound and its margins. If necessary, cover the affected area with a secondary dressing.
5. Close with the cap after use.

Lecoxen cream should be reapplied when changing the dressing or checking the wound progress, or according to the doctor's advice. For the prevention of abnormal scars, it is recommended the use of **Lecoxen cream** for a minimum treatment of 60–90 days. Larger scars can take longer to heal.

Warnings

Do not use the product in case of hypersensitivity to any of the ingredients. Suspend the use of the product in case of skin irritation. If the product is used by third parties, the use of disposable gloves is recommended. Wash your hands well before use. Avoid contact with the eyes; if accidental contact with eyes occurs, immediately wash them with plenty of water. Do not swallow. Keep out of reach of children. Do not leave the package open after use. Use the product within 6 months after opening. Recommended for children aged 3 and up. The product can be used during pregnancy and breastfeeding.

Storage

Store in a cool and dry place, away from heat sources and from light, at a temperature between 4°C and 30°C.

Format

Tube of 30 ml 

EKXPL00111

Rev. 1 del 10/2020

Dispositivo Medico
Medical device
Dispositif médical

 0546

 LOT

see the package

 **EKUBERG**
pharma

Via Tito Schipa, 6
73020 Carpignano Salentino (LE) ITALY
info@ekubergpharma.com
www.ekubergpharma.com