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Mona
ul.Leszczynowa 8
62-070 Dąbrówka
+48 607106591
www.fit4med.com

Neoprene lumbar support belt

A neoprene back support belt is a stabilizer that can be used in the lower back. The product stabilizes the spine, reduces its mobility, and also enforces the proper body posture, alleviating pain in the lumbar region. Despite providing support in the lumbar area, the product is comfortable to wear. The stabilizer is great for people with chronic back pain. Additionally, the neoprene back belt will keep warmth around the spine, which can help reduce pain in that area

Application

- Relief of the spine
- Pain syndrome in the lumbosacral region
- Protection of internal organs (especially kidneys), especially during sports

Contraindications

Discontinue use of the product if any of the following symptoms occur: sensory disturbances, blood flow disorders, allergic rash, or itching. The neoprene belt is intended for healthy individuals to support the rehabilitation process. If you suffer from serious conditions, consult a doctor or physiotherapist before use. Always consult your lead doctor or physiotherapist about the choice of belt and duration of use.

How to Use

- Adjust the stabilizer so that it provides stabilization while also ensuring comfort and ease of wear.
- When the belt is in place, try to maintain a proper body posture.
- Ensure proper hygiene of the product.

How to Use the Neoprene Belt

Wear the belt for a minimum of 30 to 60 minutes daily. Do not exceed 12 hours of continuous use.

Note

When purchasing, the smell of neoprene may be an issue. Neoprene is indeed a material that emits a strong odor, but our belts meet quality testing and toxicological testing requirements conducted on the materials used.

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