VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate www.vevor.com/support

SHOWER STOOL

MODEL: B3500/B3501/B3502/B3503

We continue to be committed to provide you tools with competitive price.


"Save Half", "Half Price" or any other similar expressions used by us only represents an estimate of savings you might benefit from buying certain tools with us compared to the major top brands and does not necessarily mean to cover all categories of tools offered by us. You are kindly reminded to verify carefully when you are placing an order with us if you are actually saving half in comparison with the top major brands.

VEVOR®

TOUGH TOOLS, HALF PRICE

Shower Stool

MODEL: B3500/B3501/B3502/B3503

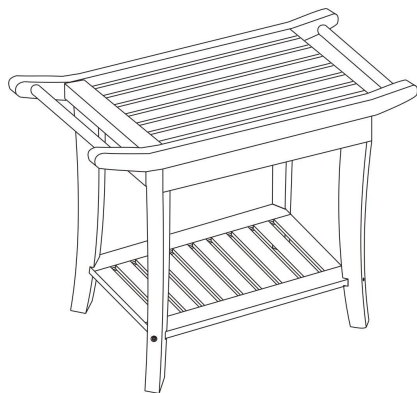B3501/B3503

B3500/B3502

NEED HELP? CONTACT US!

Have product questions? Need technical support? Please feel free to contact us:



Technical Support and E-Warranty Certificate
www.vevor.com/support

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

PARAMETER LIST

Model	B3500	B3501	B3502	B3503
Maximum load bearing (lbs)	400	300	330	300
product size(mm) (No adjustable rubber feet)	500*363*455	610*350*480	485*288*465	560*365*470
net weight (kg)	4.6	4.96	4.03	4.65

PART LIST

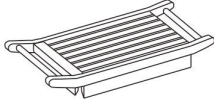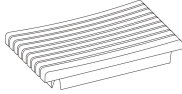number	name	picture		quantity (B3500/B3501 / B3502/B3503)
1	thwart	 B3501/B3503	 B3500/B3502	1
2	shelf			1

3	bench legs		4
4	M6*34hex socket head cap screw		8+ (spare1)
5	M5*29 tapping screw		4+ (spare1)
6	gasket		8+ (spare1)
7	Elastomeric gasket		8+ (spare1)
8	L-type Allen wrench		1
9	Adjustable height stainless steel + rubber foot mat		4+ (spare 2)

SECURITY & WARNINGS

ASSEMBLY PRECAUTIONS

1. Assemble needs to follow these instructions. Incorrect assembly can be a hazard.
2. Wear safety goggles and heavy-duty work gloves during assembly.

3. Do not assemble when tired or when under the influence of alcohol, drugs or medication.
4. Weight and other product properties are only applicable to correctly and completely assembled products.
5. Keep the assembly area clean and well-lit.

USE PRECAUTIONS


WARNING: TO PREVENT SERIOUS INJURY AND DEATH FROM TIPPING

1. This product is not a toy. Please keep the product away from children while assembling.
2. Use as intended only. Do not let children play or jump on the product. Do not stand on the item or use it as a step ladder.
3. Inspect before every use. Do not use it if parts are loose or damaged.
4. Tighten the lower screws every three months to prevent loosening of the legs.
5. Do not exceed the listed weight capacity. Tighten all knobs securely before applying load. Be aware of dynamic loading! The sudden load movement may briefly create an excess load, causing product failure.

SAVE THESE INSTRUCTION


OPERATION

Step 1: Insert the 4 stool legs (3) into the storage board (2) and tighten them with an L-type Allen wrench and 4 self-tapping screws (5) as shown below.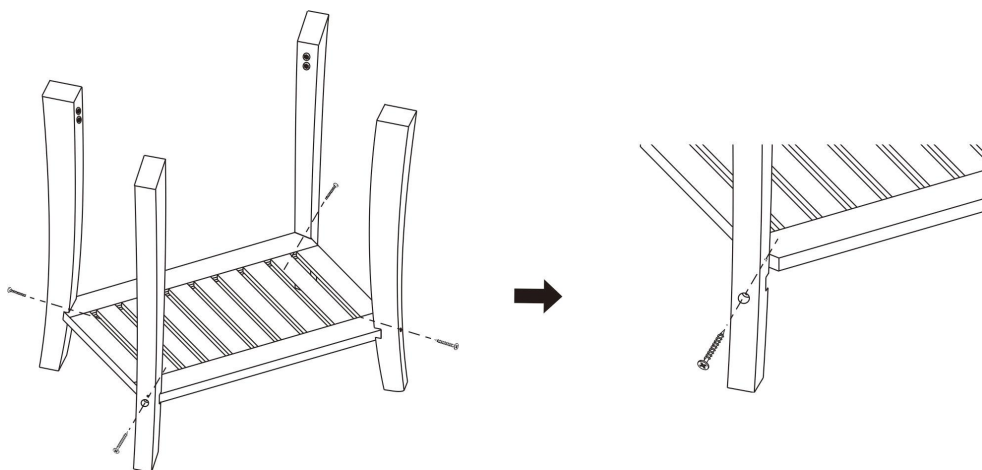


Step 2:

Turn the stool upside down and insert the legs (3) upside down into the board (1).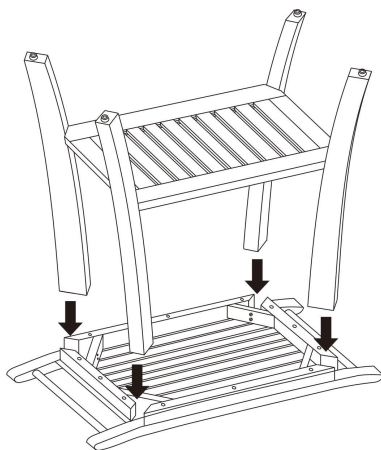

Step 3: Connect 8 M6*34 hex bolts (4), gaskets (6), and ejector (7) to the seat plate (1) and stool leg (3) with an L-shaped hex wrench.

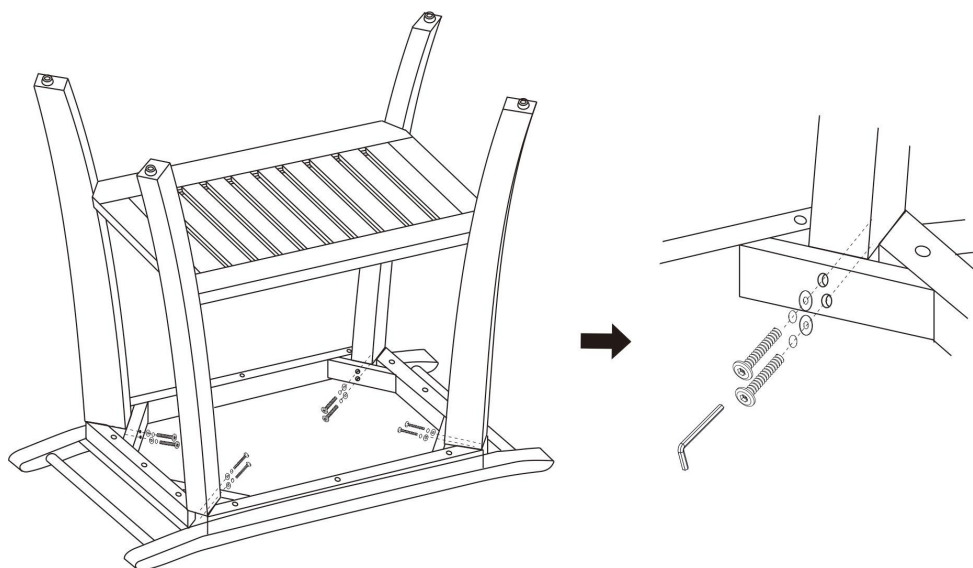Step 4. Adjustable height stainless steel + rubber foot pads are installed at the bottom of the stool legs.

FINISH

Manufacturer: Shanghaimuxinmuyeyouxiangongsi

Address: Shuangchenglu 803nong11hao1602A-1609shi, baoshanqu, shanghai 200000 CN.

Imported to AUS: SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD NSW 2122 Australia

Imported to USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim Place, Rancho Cucamonga, CA 91730

UK	REP
-----------	------------

YH CONSULTING LIMITED.

C/O YH Consulting Limited Office 147,
Centurion House, London Road,
Staines-upon-Thames, Surrey, TW18 4AX

EC	REP
-----------	------------

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VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate

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Importerad till AUS: SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD
Importerad till USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim
E-CrossStu GmbH