VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate

www.vevor.com/support

STAIR CLIMBING CHAIR

MODEL: YHR-W4

We continue to be committed to provide you tools with competitive price.

"Save Half", "Half Price" or any other similar expressions used by us only represents an estimate of savings you might benefit from buying certain tools with us compared to the major top brands and does not necessarily mean to cover all categories of tools offered by us. You are kindly reminded to verify carefully when you are placing an order with us if you are actually saving half in comparison with the top major brands.

VEVOR®

TOUGH TOOLS, HALF PRICE

STAIR CLIMBING CHAIR

MODEL: YHR-W4

NEED HELP? CONTACT US!

Have product questions? Need technical support? Please feel free to contact us:

Technical Support and E-Warranty Certificate
www.vevor.com/support

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

SAFETY INSTRUCTIONS

1. Careful inspection should be carried out before use to ensure that the stretcher structure is firm and there is no damage or loosening.
2. Pay attention to smooth handling.
3. Do not overload. Try to use it on stairs with gentle slopes.
4. When carrying patients with this stretcher, be sure to lock tightly and fasten the safety belt for safety.
5. Don't move too fast when passing pothole areas.
6. In general operation, be careful not to out-torn the stretcher surface.

MODEL AND PARAMETERS

Model	YHR-W4
Max. Load	350lbs
Fully Developed Size	840*535*1360 mm
Folding Size	1015*535*235 mm
Materials	Aluminium alloy
Weight	10.6kg

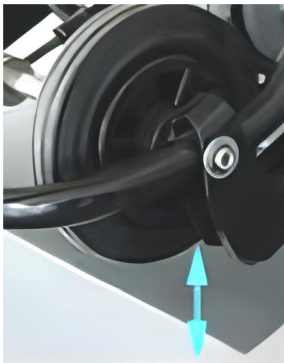
STRUCTURE DIAGRAM

OPERATION TIPS

1. Keeping a safe distance before push down the stairs.

2. Push the first step with the foot pressure fixed rod, cocking the cart to ensure that the front wheels are suspended, relying on the climbing track slide.

3. When pushing, the operator's hand force is downward rather than forward !

MAINTENANCE

1. Keep clean as routine (incl. sterilization).
2. Check frequently for loose parts.
3. Store this product in the place of damp-proof & non-corrosion.

VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate

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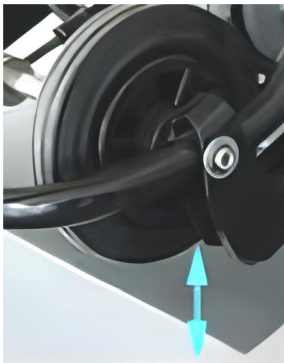
STRUCTURE DIAGRAM

OPERATION TIPS

1. Keeping a safe distance before push down the stairs.

- 2.Push the first step with the foot pressure fixed rod, cocking the cart to ensure that the front wheels are suspended, relying on the climbing track slide.

3. When pushing, the operator's hand force is downward rather than forward !

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