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EN

1. The sensory rehabilitation ball with spikes has been designed for massage, rehabilitation exercises and improving circulation. Made of flexible PVC material, it is resistant to deformation and easy to maintain. Before first use, it is recommended to check the product for any damage and make sure that the surface on which it will be used is clean and dry. A damp cloth and mild detergents are sufficient for cleaning, do not use sharp cleaning tools. **2.** The product can be used with other exercise accessories, such as yoga mats or rehabilitation rollers, but should not be subjected to heavy loads or contact with sharp objects that may damage it. **3.** Care should be taken during use, especially in the case of dynamic exercises, to avoid falls or accidental slipping. The product should be stored out of the reach of small children when not in use. **4.** The ball is intended for both adults and children under adult supervision. The markings on the product indicate from what age it can be used without the risk of choking or improper use. **5.** People with circulatory problems, skin conditions or high sensitivity to tactile stimuli should consult a specialist before use. If you experience discomfort or pain during the massage, discontinue use and consult a doctor. **6.** The product should only be used for its intended purpose. It should not be used as a throw ball on hard surfaces, a seat cushion or a play element without proper supervision. **7.** The ball does not contain electronic components or communication functions, so it is not susceptible to cybersecurity threats. **8.** The product does not have learning or predictive functions, its performance properties remain constant throughout the period of use. To ensure its long-term durability, it is recommended to regularly check the condition of the surface and store it at room temperature. **9.** The products available in our Be-active.pl store undergo thorough quality control. In the event of any problems with the operation of the purchased product or its damage during delivery, please contact the customer service department. **10.** The warranty period for purchased items begins on the day of their delivery. Detailed information about the warranty for individual products can be found on the product website.

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