
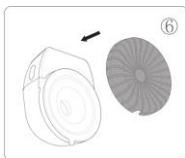
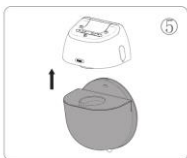
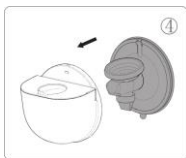
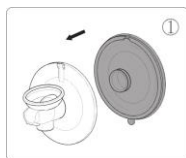
PL.....	3
EN.....	11
DE.....	19
RO.....	28
CZ.....	36
SK.....	43
FR.....	50
ES.....	58
IT.....	66
LT.....	74
LV.....	82
ET.....	89
BG.....	97
HU.....	105

Model: DE-030347

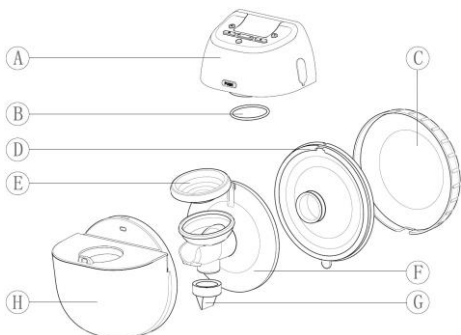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EN**IMPORTANT SAFETY INSTRUCTIONS**

- **The device should not be used by persons (including children) with the limited physical, sensory, or mental capabilities, as well as individuals without the appropriate experience or knowledge, unless they are supervised or received appropriate instructions.**
- **Children must not play with the device.**
- **Only use the charging adapter supplied with the device.**

—INTENDED USE—

The device is intended for expressing the milk from breasts.

This is a product intended for personal use. The use by more than one person carries a health risk and constitutes grounds for invalidating the guarantee.

The product is intended to be used inside. It must not be used for commercial purposes. Any other use shall be treated as improper and dangerous. The manufacturer shall not be liable for injuries that may result from incorrect use of the device.

This manual includes important information concerning the use of the product — read it carefully and keep for future reference.

—CAUTION—

Store the device, its accessories and packaging out of the reach of children. The cables and plastic packaging may wrap around a child's neck or may be swallowed, which may result in choking or suffocation. Keep the product away from children.

The device should be disconnected from the mains before cleaning. Do not use the device with a charging cable other than the one supplied.

—DO NOT RISK IT—

Do not use the device in places where it is forbidden to use electrical equipment and if it may cause disruption or other hazards. Do not turn on and do not use the device near flammable materials. Do not place

any flammable objects on the device as this may lead to damage of the device, or even fire.

Always connect the device to the power supply compliant with the data specified on the nameplate of the charging adapter.

—WATER AND OTHER FLUIDS—


The device's body should not be exposed to water or other liquids, including their immersion. This may negatively affect the operation and lifespan of the electronic and electrical components. You should avoid operation in very moist environments — this may result in damaging the device, and in extreme cases to electric shock.

—BATTERIES—

Avoid exposing the built-in battery to very low or very high temperatures (below 0°C / 32°F or over 45°C / 113°F). The extreme temperatures may affect its life. Avoid contact of the battery with the fluids or metal objects as this may lead to its complete or partial damage. Do not disassemble, modify, short-circuit, or throw the battery into a fire — this may cause explosion and fire. The used or damaged battery should be placed in a special container. Battery in the device cannot be replaced by the user.

STRUCTURE

Body	"	Silicone membrane
Seal	#	Funnel
Dust cover	\$	Valve
! Silicone cover	%	Container

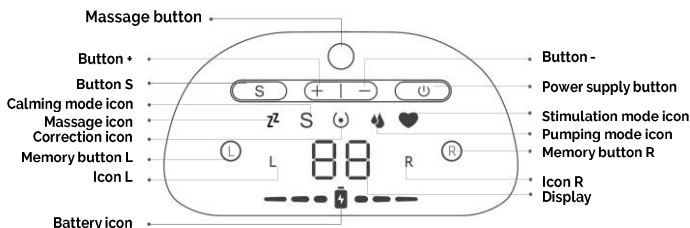TECHNICAL SPECIFICATION

Amme breast pump	
Model	DE-030347
Power supply	5V = 1A
Battery	Li-po 3,7V – 1200 mAh, 4.44Wh
Adapter parameters	
Power supply	100-240V~ 50/60Hz 0.25A
Output	5V = 1A
Certifi cates	LFGB, EMC, RoHS
BPA free	Yes
Massage levels	3
Milk intake intensity levels	9
Bottle/ container for milk	200 ml
Single/Double	Single
LED display	Yes
Charging port	DC
Automati c swit ch-off	Yes

ITEM ASSEMBLY

Fit the notch in the silicone cover (D) to the notch in the funnel (F), and place the cover on the funnel so that the notches align (1). Place the valve (G) on the opening underneath the funnel, and then place the silicone membrane (E) over the opening at the top of the funnel, as shown in the figure (2, 3).

Insert the prepared pump into the milk container (H), taking care so that the notch at the top of the pump was aligned with the milk outlet channel. Press the components tightly to ensure a tight seal (4). Place the body of the device on the connected container with the pump (A) and press firmly (5).

When storing the device, always place the dust cover (C) on the silicone cover (6).

To disassemble, perform the above steps in reverse order. When disconnecting the body from the device, hold down the PUSH button on the body.

DEVICE OPERATION

When disconnecting the device from the power, always grasp the plug, not the cable.

Before the first use, wash and disinfect the silicone elements of the product.

To charge the device, connect the DC plug of the included cable to the charging port at the side of the device. Plug the other end of the cable into the powered adapter included with the set.

After start of the charging, the indicators next to the battery icon on the display will start to flash. As the charging progresses, the indicators next to the battery icon will be longer, indicating the approximate battery charge level. When the battery is fully charged, all the indicators will emit a steady light. After the end of charging, disconnect the charging cable from the device and from the power supply.

During charging it is not possible to use the product.

To turn on and turn off the device, press the power supply button and hold it for about 2 seconds.

After switching on, pressing the power supply button shortly switches between the main operating modes in the following sequence: stimulation mode, pumping mode, correction mode. Soothing mode is an additional mode, which is switched on separately with button S during operation in the simulation or pumping mode. The second additional mode is the massage mode which is switched on separately by pressing the massage button and holding it for about 2 seconds.

The breast pump should be applied to the breast with the central part of the silicone shield, so that the nipple is placed in the shield's opening.

Description of operation modes:

- **Stimulation mode:** through imitating the mechanism of suction of the breastfed newborn, that is quick, it stimulates the breast to produce milk; after two minutes of operation, the stimulation mode is automatically switched to the pumping mode
- **Pumping mode:** it creates a slower but stronger mechanism of suction by the newborn; it switches off automatically after 30 minutes of operation
- **Correction mode:** corrects the inverted or flat nipples

- **Soothing mode:** the mode operating in combination with the stimulation or pumping mode
- **Massage mode:** activates vibrations of the device; may operate in combination with other modes (apart from the correction mode) of operation or independently; when operating independently, it turns off after 5 minutes

The intensity of operation of the stimulation, pumping and correction modes may be regulated using buttons + (plus) and – (minus). Stimulation and pumping modes have 9 intensity levels, the correction mode has 3 levels.

The intensity of the massage mode is regulated by pressing the massage mode shortly when the mode is active. The massage mode has 3 intensity levels.

The device has the ability to save two operating settings for the stimulation and pumping mode. Turn on the device in the stimulation or pumping mode and choose the desired operation intensity. Then press memory button L or R and hold it for about 2 seconds. After remembering the setting on the display screen, icon L or R (depending on which button was pressed) will flash a few times.

The saved setting will be recalled after briefly pressing the assigned memory button.

During operation, the device's display alternately shows the operating time in minutes and the intensity level of the selected operation mode.

To transfer the contents of the container, turn off the device and disconnect the body from the container by holding the PUSH button on the body and pulling it away. Then pour out the contents of the container, e.g. into a bottle, through the milk outlet channel found in the upper part of the container.

STORING THE PUMPED MILK

Storage temperature	Storage time
19°C–26°C	4–6 hours
< 4°C	72 hours to 8 days
-15°C	2 weeks

-18°C	3–6 months
-20°C	6–12 months
The above data are for informational purposes only.	

- Do not use the microwave or the boiling water to thaw the milk.
- The thawed milk may not be refrozen again.
- Do not mix the freshly pumped milk with the thawed milk.
- Dispose of the milk left after feeding.

CLEANING

WARNING! *The device's body cannot be immersed in water or washed under running water!*

Before cleaning, the device should be turned off.

Do not use any chemicals, alkaline agents, abrasives or disinfectants, as they may adversely affect the components of the device.

The cover of the device's body should be cleaned using a dry cloth or a paper towel. In the case of strong dirt it is allowed to use a cloth slightly dampened with lukewarm water. You should immediately clean the washed places using a dry cloth or a paper towel and allow the device to dry fully.

The silicone elements coming into contact with the milk should be cleaned after each use and disinfected once a day.

Silicone membrane (E) and valve (G) are delicate elements. They should be cleaned with particular care and should not be subjected to prolonged disinfection in high temperature. Otherwise, they may be deformed or damaged, which may have an impact on the quality of the product's operation.

Funnel (F) is not suitable for warming or sterilisation in the microwave.

EXPLANATION OF SYMBOLS

- 9<D9@B@H? MA> =>OB> <9G; > K<R<E= 9<<HK<B@NH IE<9E
K@NBMBGL

PAP

1 RF ; HE=>LBGMB@
 <HKN@M>=<9K< ; H9K<
 I 9<D9@B@F 9M4K9E

2H ; > NL>= LHEER
 BLB>

2A> LRF ; HEF >9GL MA9MA>K< B MA> HI >KMB@F 9GN9EB MA>
 I 9<D9@B@ PABA RHNLAHNE K9= ; >7HK< NL>

1 RF ; HE=>LBGMB@ I KHM<MBIG <BLL &H? MA> =>QB> PBA
 =HN< B BLNMBIG <HG<>KGB@MA> 9=9I M4K

2A> I KH=NKMB LNB9; B 7HKA9G=EB@7HH=LMN L

Pursuant to Directive WEEE 2012/19/EU the product is marked with the symbol of a crossed-out waste container. The product cannot be disposed of with municipal waste, but should be delivered to the designated collection and recycling centre for used electrical and electronic equipment that recycles such hazardous waste. Collection of such type of waste in designated places or the proper process of their recycling contributes to the protection of natural resources. The appropriate recycling of used electrical and electronic equipment has a positive impact on human health and the environment. You should contact the communal administration or the device's seller in order to obtain the information about the disposal points for used electrical and electronic equipment.

The batteries and accumulators that must be disposed of to specially marked containers, handed over to special waste collection points or to sellers of electrical or electronic equipment, are marked with the symbol of selective collection. They shall not be disposed of together with the unsorted municipal waste. When recycling batteries and accumulators, you should comply with local regulations and the EU legislation.

THE USE OF THE MANUAL

Without the previous written approval of the manufacturer, no part of this operating manual may be copied, distributed, translated or handed over in any form, or using any electronic or mechanical means, including by creating photocopies, registration or storage in any systems of storage or making the information available.

All the brands and names of the products are trade marks or registered trade marks of the companies' data.
The graphic appearance or colour of the product may differ from the actual product.

WARRANTY AND SERVICE

The product has a 2-year warranty, starting from the date of the product's sale. In case of defect the product should be returned to the shop in which it was purchased, remembering about the need to present the proof of purchase. The product should be complete, and ideally it should be wrapped in the original packaging.

EU DECLARATION OF CONFORMITY

mPTech Sp. z o.o. declares that the product complies with basic requirements and the remaining provisions of the so-called "New Approach" Directives of the European Union

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