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### USER MANUAL O831

**Importer:** VITOLOG.PL Sp. z o.o., ul. Międzyleska 6, 50-514 Wrocław, Poland

**Product features:** Haluks separator 2 pieces; material: SEBS;

**How to use:** Please wash and dry your feet before first use. Gently insert your toes into the appropriate compartments. The separator should be comfortable and should not exert excessive pressure on the feet. Starting with short periods of time to let your feet get used to it, gradually increase the time you wear the spacers. Wear shoes that are not too tight in the front area. After removing the separators, wash them thoroughly in soapy water and dry them.

**Warnings:** Please read the instruction manual before use and keep it for future reference. Avoid using on cracked, irritated or damaged skin. If you experience pain, irritation or discomfort, stop using immediately. It is not recommended to use during sleep and use for a long time without a break. It is recommended to consult a doctor before use if you have foot glides. Monitor your feet and body reactions when using spacers. If you notice any disturbing symptoms, discontinue use and consult your doctor. The product is not a toy. Keep away from children.

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Dovozca: VITOLOG.PL Sp. zoologická záhrada, ul. Międzyleska 6, 50-514 Wrocław, Polsko

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