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Model NT15AM

Introduction

Thank you for choosing our product. We are confident that it will meet your expectations. We encourage you to read this document before using the product for the first time. These instructions are intended to provide the necessary information on the safe use of the medical device. The manual provides detailed guidance on the device's functions and capabilities. Retaining the document allows this information to be easily found in the future, facilitating correct use.

Description of the medical device

The toilet elevation cap is used to provide extra height to enable people to get on and off the toilet safely. It is intended for people with various disabilities or mobility difficulties. With the elevation cap, people with mobility difficulties can use the toilet independently, increasing their independence and improving their quality of life. A side screw manually fixes the toilet so that the toilet seat is not easily turned. The lid prevents splashing and easily removes unpleasant odours. Tool-free assembly, easy disassembly and cleaning (maintenance).

Intended use

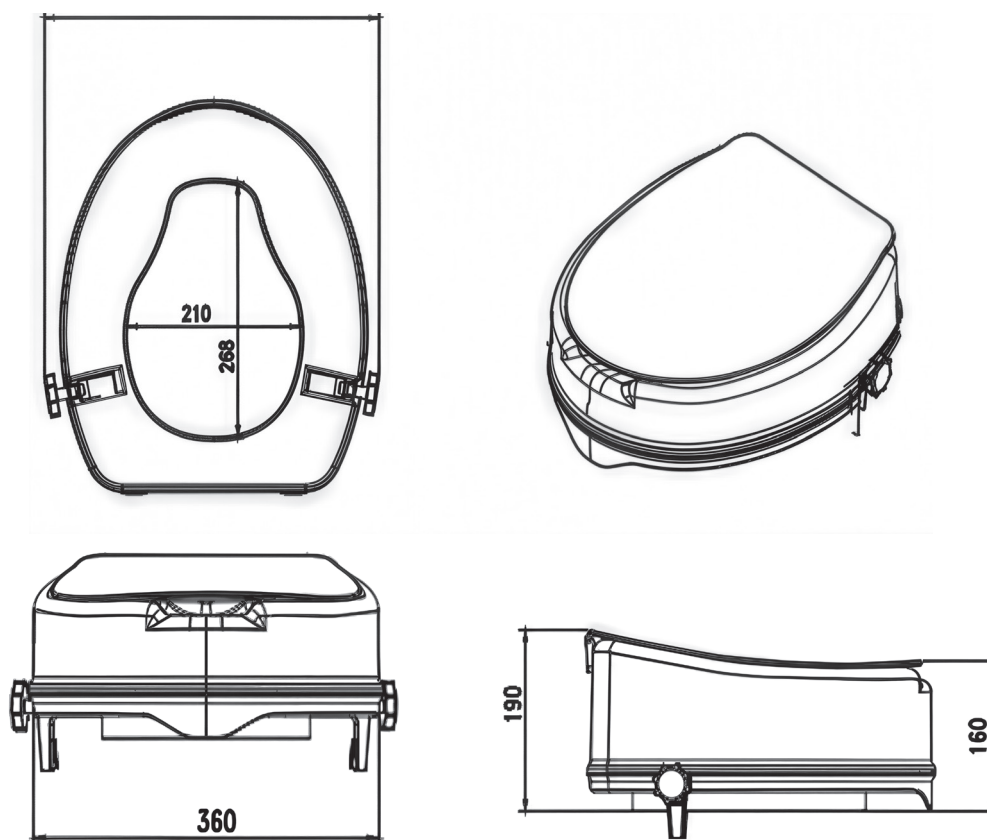
The toilet elevation cap is designed for people with reduced mobility, such as the elderly, people after surgeries, or people with disabilities, making it easier for them to use the toilet safely and comfortably.

Contraindications

People with balance problems, coordination disorders, spinal instability or contraindications to sitting should avoid using the product due to the risk of falling and aggravating their condition.

Assembly

The product is packaged in its original packaging and delivered ready for assembly. To fit the cap, simply place it directly on the toilet seat and tighten the screw on the front. Regular cleaning with a damp cloth and mild detergents will ensure the hygiene and durability of the product.

**Dimensions of the finished product in millimetres (mm)**

The user or their carer should consult a healthcare professional if they are unsure whether the medical device is appropriate for the patient in question.

Maximum permissible user weight – 180kg.

Any use other than for the intended purpose is considered inappropriate ☐ and DANGEROUS. The manufacturer and/or distributor shall not be liable for damage caused by improper, incorrect and/or unreasonable use.

Safety and usage rules

1. Before using, always check that the cap is correctly and firmly attached to the toilet bowl.
2. Ensure that the adjustment knob on the front of the cap is always tightened securely.
3. The toilet cap is designed for use with common toilet bowls. It is a universal product. Verify that the device has been correctly fixed before use.

Maintenance

- Regular cleaning of the product should be carried out with water, a sponge and mild detergents intended for plastics, such as soap or a mild cleaning agent approved by the National Institute of Hygiene. It is important to dry the product thoroughly after each use.
- Avoid the use of: strong detergents, abrasives, bleach, petrol and all solvents, as these may cause damage to the product, visible defects and lead to a loss of warranty.

Storage and transport conditions

The product should be stored in a dry and well-ventilated place, protected from moisture and extreme temperatures to prevent damage to the materials. Contact with chemicals that may affect its durability and performance should be avoided. During transport, the product should be secured in such a way as to prevent displacement and damage. Use appropriate protective packaging.

Disposal

Disposal of the product must be carried out in accordance with the applicable local environmental regulations.

Product life

The expected product life is up to 5 years if used in accordance with the information in the instructions for use. The product can be used after this period if it is undamaged and meets the specified requirements for safe use.

Warranty

- The product is covered by a 24-month warranty from the date of purchase, against any defects in material or workmanship.
- The warranty covers the free replacement and/or repair of parts found to be manufacturing defects.
- The warranty does not cover parts subject to normal wear and tear.
- The warranty is no longer valid if the product has been tampered with/modified, if the defect is the result of misuse or if the damage cannot be attributed to the manufacturer.

NOTE

Any serious incident involving the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is residing.

Product content: Polypropylene (PP).

Colour: White.

Product weight: 1.8 kg

Explanation of symbols

	Conformity of the device with EU Regulation 2017/745 on medical devices		Medical device
	Take a look at the instructions for use		Maximum user weight
	Model		Batch/series number
	Date of production		Warning
	Unique identification code of the device		Medical device manufacturer

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ACTION S.A.