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Cechy produktu:

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EN: OPERATING MANUAL 0111

Product features:

- Chinese rubber cups
- The set includes: 4 pcs.

How to use:

The cupping treatment should be performed in a room with a temperature of 20-22°C. The treatment should last up to 15 minutes. Depending on the treatment site, it is recommended to lie on your stomach, on your side or sitting. The cups are placed symmetrically on both sides of the body. Wash your skin with soap and water and lubricate it. Areas with intense facial hair should be shaved. Grab the bottom edge of the bulb with both hands while pressing the top with your thumb to squeeze the air out of the bulb. Press lightly on the desired point on the body and remove your fingers. Cover the patient with a blanket and leave for up to 15 minutes. After this time, press the skin with your thumb to release the air from the bubble. Lubricate the skin again. During the treatment you should feel warm and stretched, but not painful. After the procedure, the person should stay in bed. In bad weather, you should not go outside for at least two days after the procedure.

Warnings:

Consult your doctor before use. The product can only be used under the supervision and on the recommendation of a doctor. For the first treatment, it is recommended to use smaller cups. After each use, the product should be thoroughly cleaned with a disinfectant or an alcohol wipe. Do not use bleach, strong detergents or rough surfaces to clean. Do not wash in the dishwasher. High temperatures should not be used for disinfection. Bubbles must not be boiled or infused. During the procedure, special care should be taken to ensure that the pressure is not too high. Cupping should not be used in children and over 80 years of age. If you experience pain during the procedure, contact your doctor immediately. If any disturbing symptoms occur during the procedure (nausea, excessive sleepiness, palpitations, etc.), the procedure should be stopped immediately and if the symptoms persist, take the patient to the hospital immediately. The product is not a toy. Use the product as intended. Keep the device away from children. Contraindications: Do not use in case of: skin damage and inflammation; allergic, fungal, bacterial, purulent; increased fragility of blood vessels; occurrence of poor blood clotting; shortness of breath; autoimmune diseases; multiple sclerosis; anemia; in states of general weakness; increased temperature; unstabilized blood pressure; pregnancy; menstruation, in the case of subcutaneous hemorrhages and swelling; cancer diseases; tuberculosis; occurrence of convulsions; after drinking alcohol. Do not use in people with heart disease. Do not use in people taking anticoagulants. Do not use after meals

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