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Copper Compression Socks.

The socks, thanks to the compression system, stimulate blood circulation, providing a feeling of relief. They help reduce swelling and pain in the feet and legs. They cover the feet, ankles, and calves, applying pressure to key points. They provide comfort while wearing. Made of thin material, they are discreet and fit any type of shoe.

Usage Instructions: Before using compression socks, consult your doctor to ensure there are no contraindications. Do not wear compression socks at night.

Compression socks should not be worn for more than 6 hours unless prescribed for extended use.

Warning: If you suffer from or have a history of any of the following conditions, consult a healthcare professional about the suitability of this product:

- Heart or circulatory problems
- Arterial or venous issues
- Diabetes, phlebitis, or severe varicose veins
- Blood clots
- Limited mobility

Or if:

- You are pregnant
- You have recently undergone surgery
- You are already using prescribed compression products

If you have doubts about using this product, consult your doctor. Prolonged use may cause itching, redness, or irritation; if these symptoms occur, discontinue use immediately and consult a doctor.

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