SISSEL®

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SISSEL® NECKPILLOWS

General information

- Read the following user manual carefully and observe the safety instructions and warnings.
- The SISSEL® neck pillow is an orthopedic pillow that supports the cervical spine due to its special shape and thus relieves the entire shoulder-neck area.
- The product and accessories may only be used in accordance with these operating instructions or as instructed by appropriately trained specialists.
- SISSEL® GmbH accepts no liability for the use of the system outside the applications described in these instructions for use.
- The user manual must be kept carefully and within easy reach, and must be enclosed if the product is passed on.
- The regulatory requirements applicable to this product in the respective country of use must be taken into account.

Safety instructions

- ! Check the undamaged condition of the product before each use. In case of damage, the function of the product and safety is no longer guaranteed.
- ! Do not use the product in the close area of heat sources (e.g. radiators, stove top, open flame, flying sparks) and avoid direct sunlight.
- ! Keep the area of use free of sharp or pointy objects.
- ! Use only accessories recommended by the manufacturer.
- ! In case of uncertainty or existing health problems in the shoulder neck area or cervical spine, use the product only after consulting a doctor.

Product information

SISSEL® Neck pillow

Scope of delivery consisting of one of the following products:

- 1x SISSEL® Soft Bambini, 1x user manual, 1x cover
- 1x SISSEL® Soft, 1x user manual, 1x cover
- 1x SISSEL® Soft Plus, 1x user manual, 1x cover
- 1x SISSEL® Soft Deluxe, 1x user manual, 1x cover
- 1x SISSEL® Classic, 1x user manual, 1x cover
- 1x SISSEL® Classic Plus, 1x user manual, 1x cover

Technical Data

SISSEL® Soft Bambini

Dimensions (LxWxH): 37 x 25 x 9 cm

Material: Filling: Viscoelastic foam
Cover: 100 % cotton

Gewicht: 0,60 kg

SISSEL® Soft

Dimensions (LxWxH): M: 47 x 33 x 11 cm

L: 47 x 33 x 14 cm

Material: Filling: viscoelastic foam
Cover: Velour
(75 % cotton, 25 % polyester)

Weight: M: 0,98 kg

L: 1,16 kg

SISSEL® Soft Plus

Maße (LxBxH):	47 x 33 x 11/14 cm
Material:	Filling: Viscoelastic foam Cover: Velour (75 % cotton, 25 % polyester)
Weight:	1,16 kg

SISSEL® Soft Deluxe

Dimensions (LxWxH):	63 x 33 x 10/12 cm
Material:	Filling: Viscoelastic foam Cover: Satin (50 % cotton, 50 % polyester)
Weight:	1,20 kg

SISSEL® Classic

Dimensions (LxWxH):	M: 47 x 33 x 11 cm L: 47 x 33 x 14 cm
Material:	Filling: polyurethane foam Cover: Velour (75 % cotton, 25 % polyester)
Weight:	M: 0,58 kg L: 0,74 kg

SISSEL® Classic Plus

Dimensions (LxWxH):	47 x 33 x 11/14 cm
Material:	Filling: polyurethane foam Cover: Velour (75 % cotton, 25 % polyester)
Weight:	0,74 kg

The SISSEL® Soft Plus, the SISSEL® Soft Deluxe and the SISSEL® Classic Plus are height-variable by an insert to adjust the neck pillow individually optimal.

Intended use

- Use the SISSEL® neck pillow instead of your usual pillow for sleeping. The pillow is designed especially for side sleepers, but also for back sleepers.
- The special shape positions the cervical spine in an anatomically correct, physiological position and relieves the neck muscles and intervertebral discs. In this way, the SISSEL® neck pillow makes a significant contribution to relaxing the neck muscles and thus promoting deep, healthy, restful sleep.
- Lie on your side with your head on the SISSEL® neck pillow so that the high side is against your shoulder and the cervical spine is supported by the pad. The SISSEL® neck pillow is correctly positioned when the high side fills the area where the cervical spine meets the shoulder.
- When using the SISSEL® neck pillow, the new sleeping position may initially cause a change in sleeping habits for a few nights. As a rule, this adjustment lasts a few days. Only in exceptional cases can it take longer.

Indication

- pain in the shoulder-neck area and the cervical spine
- tension in the neck muscles with accompanying tension headaches
- arthrosis and/or intervertebral disc problems of the cervical spine (consultation with specialist)
- therapeutic support for cervical spondylosis (consultation with specialist)
- accompanying therapy for chronic instability syndrome of the cervical spine (consultation with specialist)
- rehabilitation support for acute or chronic neck pain, tension of the neck muscles and shoulder girdle (consultation with specialist)
- prevention of tension and overstraining malpositions through anatomically correct positioning of the cervical spine during sleep
- avoidance of overloading the cervical spine through physiological positioning of the head in the side position during sleep

Contra Indication

- lack of compliance (due to fear or pain)
- acute injuries and/or sprains of the cervical spine
- acute cervical spine syndrome (cervical syndrome)
- acute cervical vertigo (dizziness)
- acute radiating pain
- discomfort
- movement disorders
- acute stroke
- acute herniated disc of the cervical spine
- acute inflammatory processes such as meningitis and encephalitis
- acute or chronic nerve root inflammation of the cervical spine
- spastic torticollis (spasmodic torticollis)
- neurological diseases or malformations
- tumour diseases

Regulatory requirements

The current product of SISSEL GmbH complies with the following regulatory requirements:

Medical Devices Regulation (EU) 2017/745

⚠ All serious adverse events that occur in relation to the product must be reported immediately to the manufacturer and the competent authority.

Usage

- Check the packaging and the product for any damage upon delivery.

 Never use a damaged product.

- Check the product for integrity and contamination before each use.
- Use the product as intended.
- Cleaning measures may be necessary after use (see Care & Hygiene).

Care & Hygiene

- This product is suitable for repeated use. In general, the applicable national regulatory requirements for the hygienic reprocessing of medical devices must be observed and fault-free function must be ensured.
- SISSEL® neck pillows cannot be disinfected. If used by more than one person, additional hygiene measures may be necessary.
- The PU foam core and the core made of visco-elastic foam can be hand-washed at lukewarm temperature using a mild detergent. Carefully squeeze out by hand, never wring out! Then leave to air dry on a grid (place a towel underneath)

for a sufficiently long time (approx. 1 week). Do not dry in the sun or on the heater.

- Discoloration of the pillow core is possible and does not affect the function of the product.
- The velour cover, the cotton cover and the satin cover are machine washable at 40°C. Then air dry. Pay attention to the washing label.
- Store the SISSEL® neck pillow in a dry, clean place out of direct sunlight when not in use.

Disposal & Environment

- Product: The product and the accessories supplied with it can be disposed of with household waste.
- Packaging: No special types of packaging are used. The respective valid disposal guidelines must be obeyed.

Warranty

- The warranty is governed by the legal regulations.
- Any warranty is excluded if goods delivered by SISSEL® GmbH have been processed, treated or modified by others without consent or if the instructions for use have not been followed.
- The user of this product must comply with the regulatory requirements for medical devices applicable at the place of use.

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