VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate www.vevor.com/support

TOILET SAFETY RAILS

MODEL: FBL760600

We continue to be committed to provide you tools with competitive price.

"Save Half", "Half Price" or any other similar expressions used by us only represents an estimate of savings you might benefit from buying certain tools with us compared to the major top brands and does not necessarily mean to cover all categories of tools offered by us. You are kindly reminded to verify carefully when you are placing an order with us if you are actually saving half in comparison with the top major brands.

VEVOR[®]

TOUGH TOOLS, HALF PRICE

TOILET SAFETY RAILS

MODEL: FBL760600

NEED HELP? CONTACT US!

Have product questions? Need technical support? Please feel free to contact us:

Technical Support and E-Warranty Certificate
www.vevor.com/support

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

Warning-To reduce the risk of injury, users must read the instructions manual carefully.

WARNING:

Read the manual completely to avoid harm, product damage, or warranty avoidance. Seek professional help if you encounter discomfort, pain, or worsening symptoms. If necessary, use supervision in order to assemble and use this product correctly and safely. Do not exceed the 300lb weight limit.

Regularly check for secure assembly and contact support if in doubt. Ensure the product's legs are even and locked before use, and discontinue usage if there are signs of wear, damage, or missing parts. The product is to be used on flat, stable surfaces. Do not disassemble if not necessary, as it might compromise the longevity of the product. Do not modify the product unless specified in the manual. Use only parts approved by VEVOR for this specific product. Confirm the compatibility of the toilet rail with your toilet before first use. The product isn't built to withstand pressure or impact and isn't to be used as a ladder or loaded with heavy objects.

Keep away from children and pets. Contains potentially hazardous small parts - choking hazard. Exercise caution during operation and assembly. Avoid extreme temperatures or open flames. If an allergic reaction occurs, cease use immediately and consult a healthcare professional.

Warranty Policy

We warrant your new VEVOR product to be free from defects in materials and workmanship 3 years of the original purchase. However, please keep in mind that our guarantee policy does not cover counterfeit or replica items, nor does it extend to products purchased from unauthorized resellers or in used condition.

Please note that each purchase is eligible for only one guarantee redemption, and we cannot repeatedly refund, replace, or exchange your product. We also kindly ask that you handle the product with care and use it according to the instructions provided. Unfortunately, we cannot cover damage resulting from misuse or abuse of the product. This warranty is not transferable and applies only to the original purchaser.

IMPORTANT

1. Don't exceed the weight capacity - 300 lb max.
2. Use on stable surfaces only.
3. Regularly inspect for signs of wear, damage, or loose parts.
4. Clean regularly using a mild detergent and warm water, rinse off any soap residue. After cleaning, dry thoroughly.
5. Before use, confirm with your healthcare provider that the product meets your specific needs. The following guidelines are not a substitute for medical advice.

SAVE THESE INSTRUCTION

PARAMETER LIST

Model	FBL760600
Rated load (lbs)	300
Height adjustment range(mm)	125mm
Net weight(kg)	1.22
Product size(mm)	405*350*175

PART LIST

When unpacking this product, check to make sure the parts listed below are included and carefully inspect for any damage that may have occurred during transit. If any part is missing or damaged, do not attempt to assemble or use the product.

NO	PICTURE	PART NUMBER	QTY
1		Toilet seat	1
2		Clamp	2

OPERATION

1. Installation Procedures

1. As shown in the figure, place the toilet seat ① upside down on a flat surface (you can see the raised inner circle in the middle of the seat and the front raised part).

2. As shown in the figure, insert the two clamps ② through the threaded guide slots on each side of the bottom of the seat, and press down firmly on the clamps to rotate the bracket knobs clockwise (note: the clamps should not be locked all the way to the bottom, otherwise the seat may not be placed properly on the toilet).

3. As shown in the picture, place the seat with the clamps assembled squarely on the toilet (so that the middle bulge of the seat is completely inside the toilet and the front bulge of the seat snaps onto the front edge of the toilet) and adjust the clamp knobs to secure the seat to the toilet.

Manufacturer: Zhongshan Bliss Medical Instrument Co., Ltd
Address: 3-6F & 7F-1, No. 63, Tongxing East Road, Yumin Community Xiaolan, Zhongshan, Guangdong, 528414, CHINA
Imported to AUS: SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD NSW 2122 Australia
Imported to USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim Place, Rancho Cucamonga, CA 91730

UK	REP
-----------	------------

Pooledas Group Ltd
Unit 5 Albert Edward House, The Pavilions
Preston, United Kingdom

EC	REP
-----------	------------

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VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate

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Tillverkare: Zhongshan Bliss Medical Instrument Co., Ltd. **Address:** 3-6F &
Importerad till AUS: SIHAO PTY LTD. 1 ROKEVA
Importerad till USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim Place,
SHUNSHUN GmbH