**lorelli®**  
designed in EU

ELECTRIC BREST PUMP

**ALICE**

V 1.2

## MANUAL INSTRUCTION

find us on  
[www.lorelli.eu](http://www.lorelli.eu)

# CONTENT

<b>EN</b>	Manual Instruction.....	6
<b>ES</b>	Instrucciones de uso.....	12
<b>IT</b>	Istruzione per l'uso.....	19
<b>FR</b>	Mode.d'emplai.....	26
<b>BG</b>	Инструкция за употреба.....	32
<b>GR</b>	ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ ΚΑΡΕΚΛΑΣ.....	39
<b>AR</b>	دليل التعليمات.....	46
<b>DE</b>	Bedienungsanleitung.....	53
<b>CZ</b>	Návod k použití.....	60
<b>HU</b>	Használati utasítás.....	66
<b>RU</b>	Инструкция по эксплуатации.....	73
<b>SRB HR</b>	Uputstvo za upotrebu.....	80
<b>ME BIH</b>		
<b>RO</b>	Instrucțiuni de utilizare.....	86
<b>NL</b>	Gebruikshandleiding.....	92
<b>AL</b>	Insrksion për përdorim.....	99
<b>TR</b>	Kullanım talimatı.....	106
<b>PL</b>	Instrukcja użytkownika.....	112
<b>MK</b>	Упатствата за употреба.....	119

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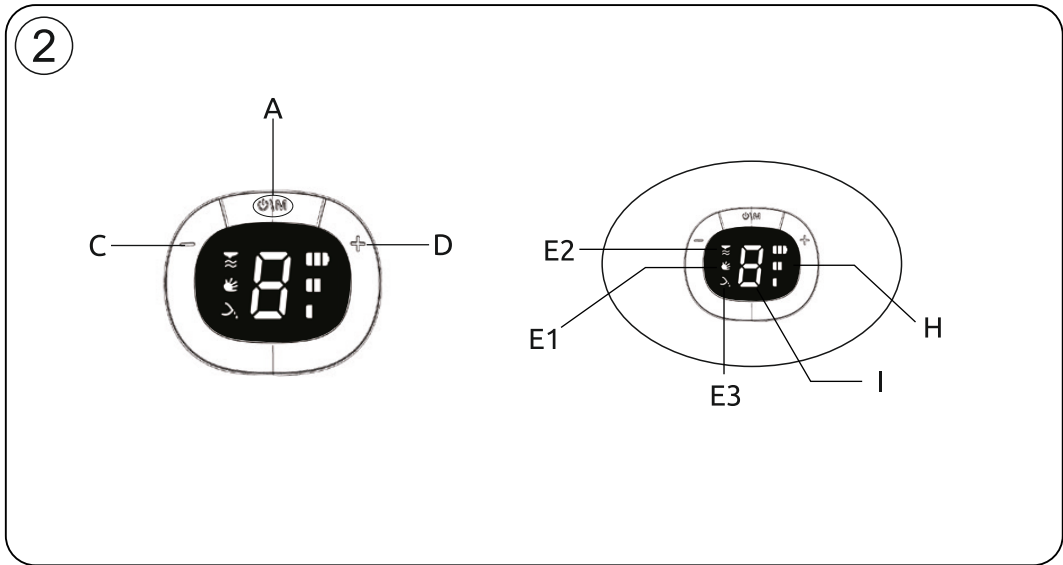
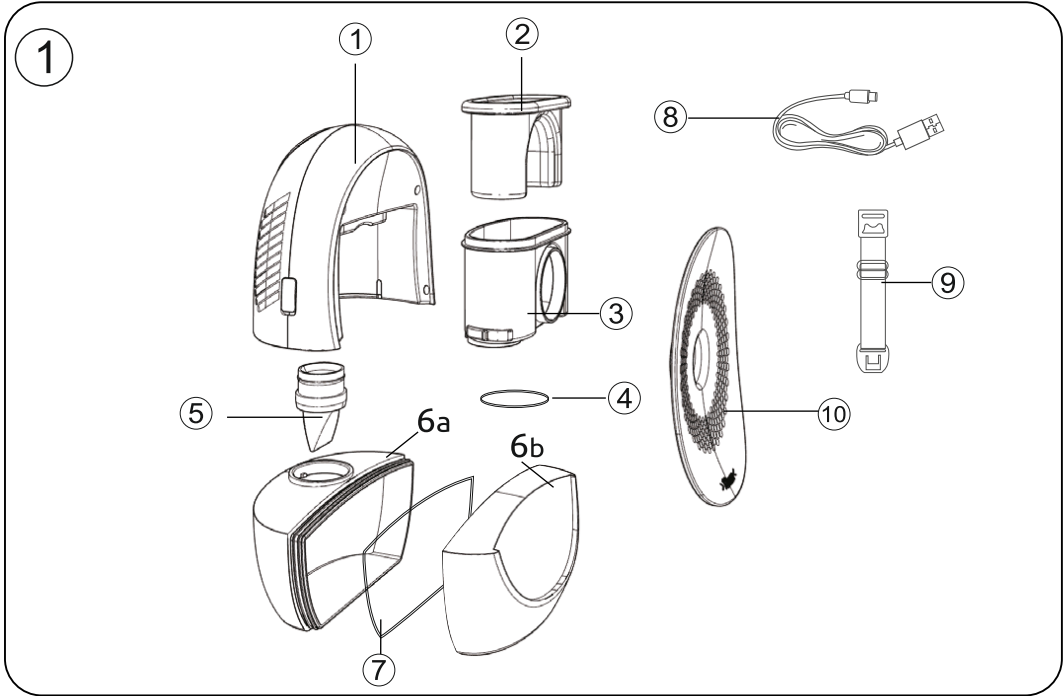
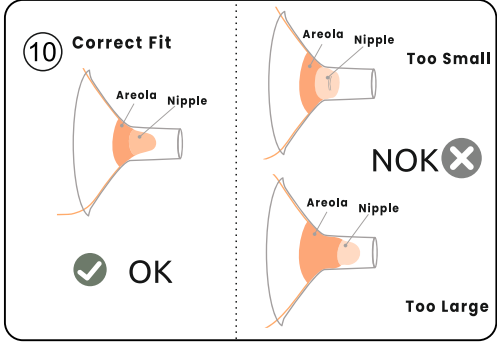
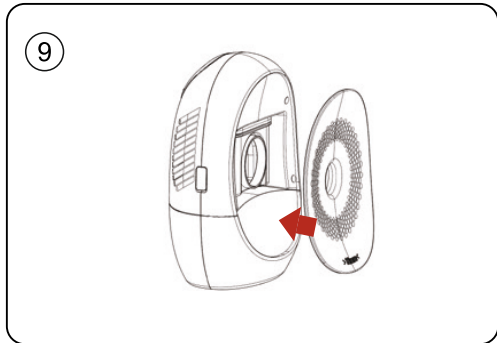
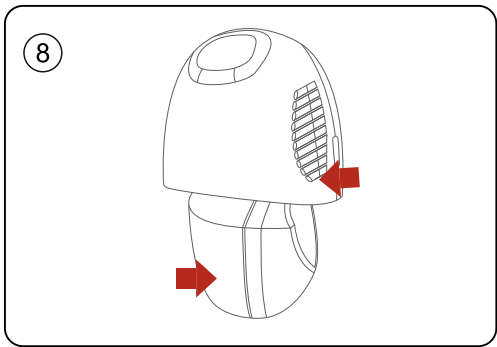
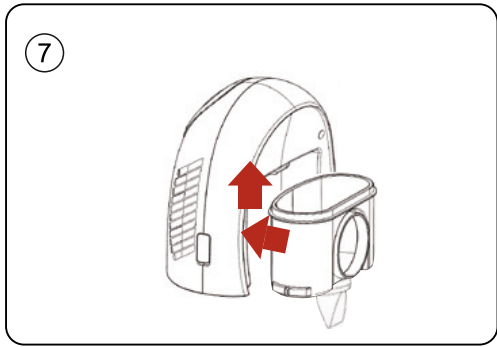
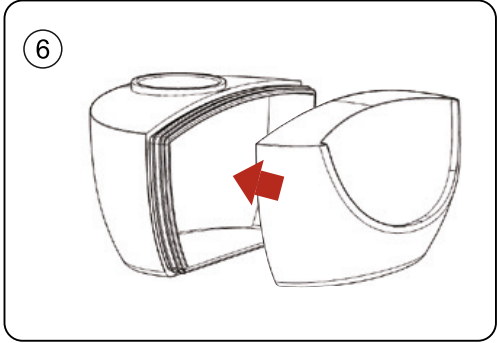
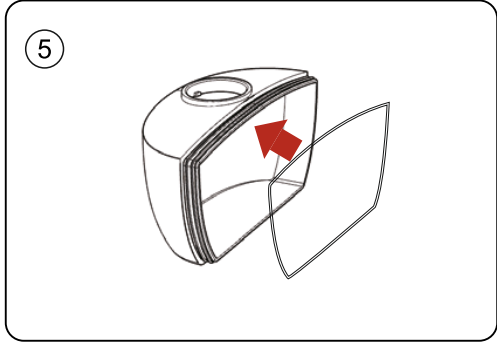
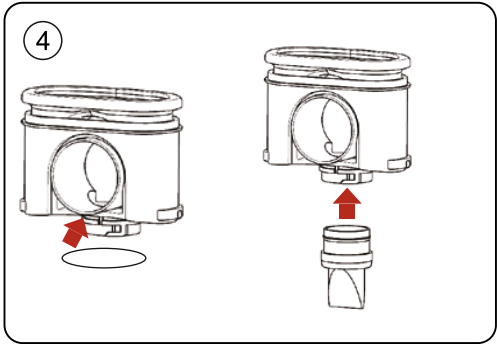
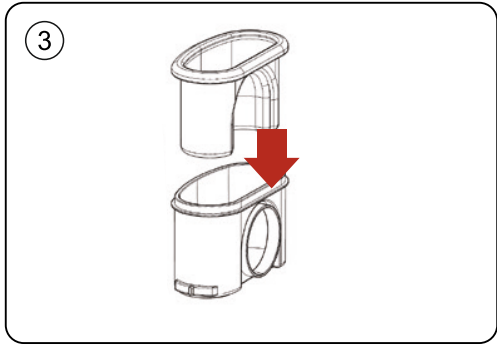
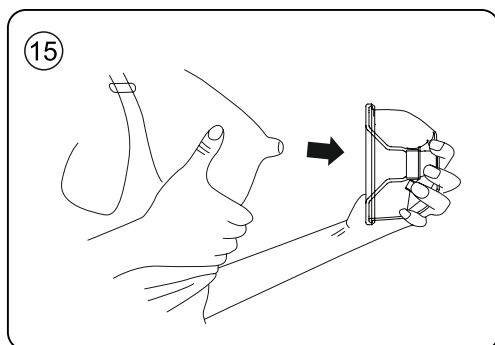
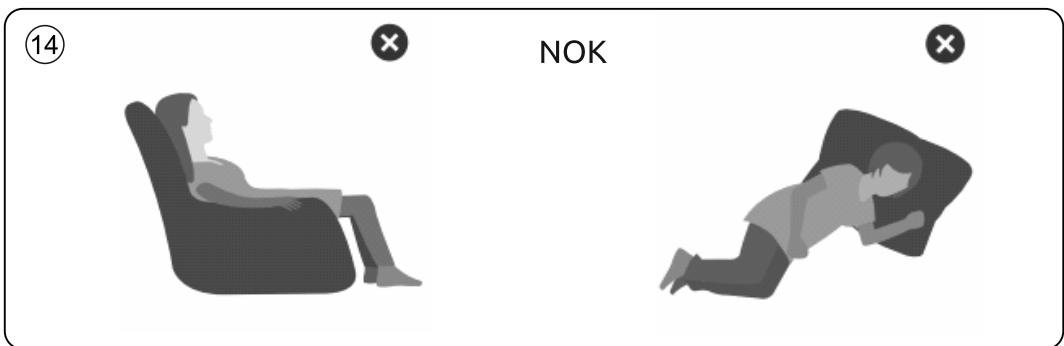
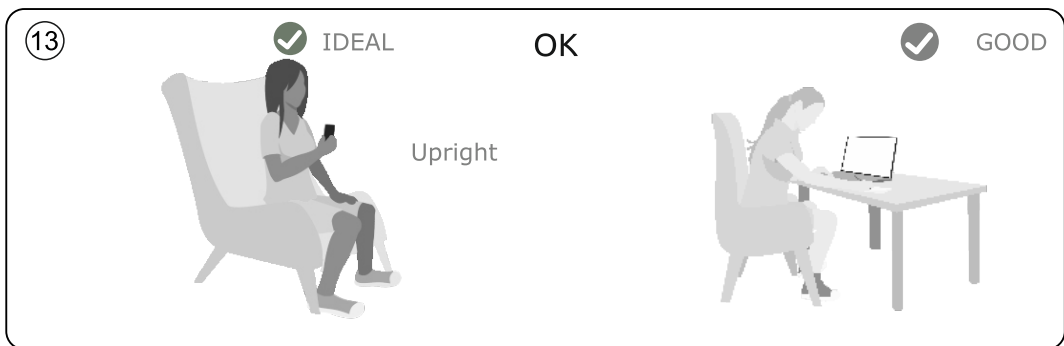
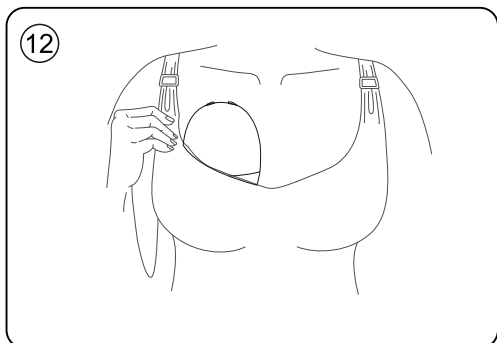
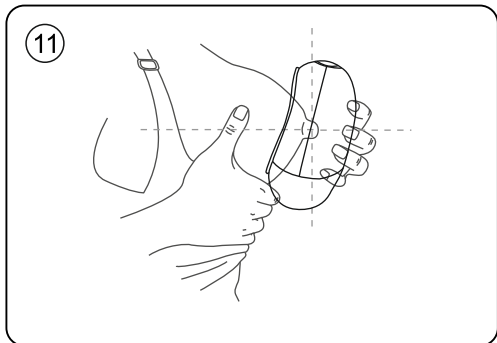
# FEATURES

# Before using the breast pump carefully read this user manual and keep it safe for future reference.

EN

## Important safety information.

Before using the breast pump carefully read this user manual and keep it safe for future reference.

### Intended use

The electric breast pump is intended for expressing and collecting breast milk from the breasts of nursing women. The device is intended for one user

## SAFETY TIPS

### Contraindications

**WARNING!** Never use the breast pump while pregnant, as pumping can induce labour.

To prevent injury and damage, important notes are explained using different symbols. The symbols and their meanings are as follows:

**WARNING!** – indicates potentially hazardous situations which, if not avoided, could result in death or serious injury.

**ATTENTION!** - indicates potentially hazardous situations which, if not avoided, may result in minor or moderate injuries.

## WARNING

### Warnings to avoid choking, strangulation and injury:

- Do not allow children or pets to play with the motor unit, adapter or accessories.
- Children should be supervised to ensure that they do not play with the appliance
- **Always unplug** the breast pump immediately after use.
- Do not leave the breast pump unattended while plugged in.
- This breast pump is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities or lack of experience and knowledge. Such persons can only use this breast pump if they are supervised or have been instructed by a person responsible for their safety on how to use the device.

### Warnings to avoid electric shock and burns:

- Inspect the breast pump, including the adapter, for signs of damage before each use. Do not use the breast pump if the adapter or plug has been damaged, if the device is not working properly or if it has been dropped or submerged in water.
- Use only the suggested adapter and specified voltage to prevent adapter overheating, damage or accidents.
- Do not touch the AC adapter or pump during a thunderstorm.
- Do not place the pump/battery/AC adapter in direct sunlight or rain, near heaters or other hot appliances, or in places exposed to extremely high or low temperatures, high humidity or dust.
- Do not place heavy objects on the pump or damage the cable or plug. Do not stretch or twist the cable. Always hold the AC adapter when unplugging. Make sure the plug is firmly inserted into the outlet. Do not use an outlet that is loose.
- Do not allow the pump/battery/AC adapter to get wet.
- Do not disassemble or modify the pump/battery/AC adapter. Do not throw the battery into fire or expose it to heat.
- Immediately unplug the AC adapter and remove the battery and stop using the pump if you notice an abnormal odour, heat, discolouration, deformation, or anything unusual while using or storing the pump.

Warnings to avoid poisoning, infection and to ensure hygiene:

- For sanitary reasons, the breast milk pump is intended for repeated use by one user only.
- Before each use clean, rinse and disinfect all parts of the pump except the drive unit and the adapter.
- Only store breast milk that has been expressed with a cleaned and disinfected pump.
- Do not use the breast pump if the silicone diaphragm appears damaged or defective.
- Do not clean the pump parts with antibacterial or abrasive cleaning agents.

**Precautions to avoid problems and pain in breasts and nipples:**

- Do not attempt to remove the pump while it is attached to the breast with suction. Turn off the device and break the seal by sliding a finger between your breast and the pump funnel. Then remove the pump from the breast.
- Never use the breast pump when you are drowsy or sleepy to avoid inattention during use.
- Always turn off the breast pump before removing it from the breast to release the seal.
- Never use accessories or parts from other manufacturers or those not specifically recommended by the manufacturer. This could cause the device to malfunction and may affect electromagnetic compatibility (EMC). Using such accessories or parts will void your warranty.
- Do not pump for more than 5 minutes without stopping if no breast milk comes out. Try pumping at a different time of day.
- If the process becomes very uncomfortable or painful, stop using the pump and consult your lactation consultant.
- If the pressure causes you discomfort or pain, turn off the device, break the seal by sliding a finger between the breast and the pump body, and remove the pump from your breast.
- Never drop or insert any foreign objects into the openings.

**ATTENTION**

**Warnings to prevent personal injury, product damage and malfunction:**

- Do not allow the adapter and motor unit to come into contact with water.
- Keep the adapter and motor unit away from hot surfaces to avoid overheating and deformation of these parts.
- Never place the motor unit or adapter in water, in a dishwasher or in a sanitiser.
- Although the breast pump complies with the applicable EMC directives, it may still be sensitive to stronger emissions and/or cause interference with other equipment. This may cause the breast pump to switch off or to go into error mode. To prevent interference, keep other electrical equipment away from the breast pump during expression (see 'Technical information').
- There are no user-serviceable parts in the motor unit of the breast pump. No equipment modification is permitted. Doing so will void your warranty.
- Do not drop or hit the pump/AC adapter. Do not press hard on the LCD display.
- The drive unit and adapter do not require cleaning and sterilization as they do not come into contact with breast milk.
- Never use alcohol or other organic solvents to clean the pump.
- Do not use the pump in a way or for purposes other than those described in the instructions. Doing so may result in accidents or damage to the pump.

**PARTS**

**Picture 1:**

1. Main unit
2. Silicone diaphragm
3. Distributor body
4. O-ring
5. Valve

**6 a ; 6 b** - front and back of 140 ml container.

**7.** Container seal

**8.** USB type C power cable (no adaptor included)

**9.** Bra extender (compatible with nursing bra only)

**10.** Silicone membranes (funnels) - 4 pcs.

## PREPARING FOR USE

### Cleaning and disinfection

The drive unit and adapter do not require cleaning and disinfection as they do not come into contact with breast milk. Before the first use, clean and disinfect all other parts of the breast pump. Clean these parts after each subsequent use and disinfect them before each subsequent use. Dry well. **Attention!:** Please dry the silicone membrane before use to avoid damage to the motor.

**Attention!:** Never immerse the drive unit or adapter in water, in a dishwasher or in a sanitiser because this will cause irreversible damage!

1. Disassemble the pump completely. Remove the valve as well. Be careful when removing the valve and when cleaning it. If it breaks, your breast pump will not work properly.

2. Clean all parts except the drive unit and adapter in hot water with a little mild liquid dish soap and then rinse thoroughly. To clean the valve, rub it gently between your fingers in warm water with a little liquid detergent. Do not insert objects into the valve as this may damage it.

**Warning! - Do not use antibacterial or abrasive cleaning agents.**

**Note:** Make sure you clean and disinfect the relevant parts of the breast pump. Wash your hands thoroughly before handling the cleaned parts!

**Warning! - Be careful as the cleaned parts may still be hot. Start assembling the breast pump only when the cleaned parts have cooled.**

## USAGE

### Buttons ( Picture 2 )

**A.** ON/OFF button and mode change button ; **C.** Reduce suction; **D.** Increase suction;

### Indicators

**E1** - Massage mode; **E2** - Stimulation mode; **E3** - deep drain; **I** - Suction strength indicator; **H** - Battery charge indicator.

### BATTERY OPERATION

**ATTENTION!!!** Before first use, fully charge the battery for 8 hours continuously!

Connect the USB plug of the power cable to the socket of the pump, and the other side of the power cable to the adapter, which is not included in the set of the device.

The battery indicator starts flashing until the battery is fully charged.

**Important!** The battery cannot be charged for more than 24 hours.

## PUMP ASSEMBLY

1. Place the silicone diaphragm into the distributor body and press the seal with your fingers along the edge down to make a snug fit (Picture 3).

2. Place the O-ring in the channel of the distributor, then place the valve in the protruding hole and push it with your fingers as far in as possible so that it fits snugly along the rim (Picture 4).

3. Place the seal along the edge of the container body (Picture 5). Install the front and back part of the container (Picture 6).

4. Install the distributor body to the main pump unit:

- arrange it so that the opening is on the back side, then press it up against the main unit until it is fully seated (Picture 7).

5. Attention! In order to install the main unit to the container, it is important that they are perpendicular. Insert the valve attached to the main unit into the opening of the container body, then press down slightly and turn clockwise until they lock together (Picture 8).

6. Place the silicone membrane (selected according to the size of your nipple) into the pump opening, with the direction of the membrane indicated by an arrow matching the direction of the container. Then press with your fingers until a perfect seal forms (Picture 9).

## SETTINGS

 ON/OFF button - hold the button to turn on the pump. Massage mode starts automatically for 2 minutes, then changes to pumping mode. By default, the pumping time is 30 minutes, the pump will automatically turn off after the time is up.

 Mode change button - with a short press. With pressing the indicated button, the modes start to change successively to massage, pumping and stimulation.

-  Massage mode – 9 levels
-  Stimulation mode – 9 levels
-  Deep drain mode - 9 levels

Apart from that, the pump also offers a manual adjustment of each mode in the order of 9 degrees, by means of the + and - buttons.

Important: The level of milk in the bottle must not exceed the level of the valve!

Don't be concerned if the flow of milk suddenly stops. Continue pumping calmly, but not for more than 5 minutes. If you are not expressing more milk, try another time of day. It is not necessary to use all modes of operation, only the one that is most comfortable for you. The pump automatically switches off after 30 min.

## WHEN TO EXPRESS BREASTMILK

If breastfeeding goes well, it is advisable (unless advised otherwise by your healthcare professional/ breastfeeding consultant) to wait until your milk supply and breastfeeding schedule are established (normally at least 2 to 4 weeks postpartum) before you start expressing.

### Exceptions:

— If you have trouble breastfeeding at first, regular pumping can help build and maintain your milk supply.

— If you are expressing breast milk for your baby to be given in the hospital.

— If your breasts are engorged (painful or swollen), you can express some breast milk before or between feeds to ease the pain and help your baby latch more easily.

— If you have sore or cracked nipples, you may wish to express your milk until they are healed.

— If you are separated from your baby and wish to continue to breastfeed when you are reunited, you should express your milk regularly to stimulate your milk supply.

You need to find the optimum times of the day to express your milk, for example just before or after your baby's first feed in the morning when your breasts are full, or after a feed if your baby has not emptied both breasts. If you have returned to work, you may need to express during a break. Using a breast pump requires practice and it may take several attempts before you succeed. Fortunately, the Lorelli electric breast pump

is easy to assemble and use so you will soon get used to expressing with it.

## Nipple to flange size guide

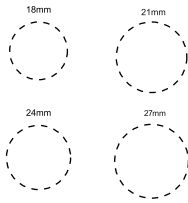
### Steps to determine the correct funnel size according to the nipple size:

1. Stimulate your nipple until it perks up (don't sip this step).
2. Slide the hole over each nipple.
3. Select the hole that can comfortably fit the very base of your nipple. (Foto 10).
4. Get your measurement based on the sizing chart

**Note:** sizing is an "art" Based on your nipple elasticity, you may need to size down if you have elastic nipple, or size up if you have less elastic nipples. If you feel discomfort, you may need to experiment with different sizes.

The correct size helps to achieve optimal breast milk flow.

#### NIPPLE MEASURE TOOL

Nipple Diameter Measurement	Recommended Flange Size or Flange Insert Size
16 - 18 mm	18mm
19 - 21 mm	21mm
22 - 24 mm	24mm
25 - 27 mm	27mm

## OPERATING THE BREAST PUMP

1. Wash your hands thoroughly and make sure your breasts are clean.
2. Press the assembled pump body against the breast (Picture 11). Make sure your nipple is centred so the massage cushion (funnel) can create an airtight seal. Make sure your nipple fits properly in the cushion (funnel) (Picture 10). You can use the breast pump while wearing a bra (Picture 12). With the bra extender (compatible with nursing bras only) in the pump kit, you can adjust the circumference.
3. Take one of the allowed positions shown in picture 13. Attention, picture 14 shows the unfavourable positions in which the pump does not function properly and there is a possibility of damage.
4. Press the on/off button. The breast pump automatically starts working in massage (stimulation) mode and you can change the massage level using the increase and decrease buttons.
5. When the breast milk flows, you can change the mode from massage to deep drain - via the mode button and choose the most comfortable level.  
**ATTENTION:** Don't always use the maximum pumping level, choose the one most comfortable for you.  
Note: Don't worry if your milk doesn't flow right away. Keep calm and keep pumping. The first few times you use the breast pump, you may need to set it to a stronger suction setting to get your breast milk flowing.
6. Depending on your personal comfort, you may decide to use a stronger suction mode. You can always switch back to a lower suction mode.
7. When you are done expressing, turn off the pump with the ON/OFF button and carefully remove it from your breast (Picture 15).
8. Remove the silicone membrane (funnel) and container from the motor unit.
9. Clean the rest of the used parts of the breast pump according to the instructions in the "Cleaning and disinfection" section.

## STORING BREAST MILK

**Warning:** To ensure proper hygiene, only store breast milk that has been expressed with a cleaned and disinfected pump.

- Breast milk can be stored in the refrigerator (not on the door) for up to 48 hours. Expressed breast milk should be refrigerated immediately. If you store breast milk in the refrigerator and add to it during the day, only add breast milk that has been expressed in a disinfected bottle.

- Breast milk can be stored in the freezer for up to three months if it is in disinfected bottles with a disinfected screw ring and sealing disc. Place a label on the bottle marking clearly the date and time of expression and use the older breastmilk first.

- If you are going to feed your baby the expressed breast milk within 48 hours, you can store it in the refrigerator in an assembled bottle.

### **WARNING!**

Always put expressed breast milk in the refrigerator or freezer immediately.

Only store breast milk that has been expressed with a disinfected breast pump in disinfected bottles.

Never refreeze thawed breast milk.

Never add fresh breast milk to already frozen breast milk.

## TROUBLESHOOTING

**Problem:** I experience pain when I use the breast pump.

**Solution:** One possible cause is excessive suction. You don't need the full force of suction produced by the pump. Initially, use "Massage" mode for massage stimulation, and then you can switch to pumping mode. If the pain does not subside, stop using the breast pump and consult your lactation consultant. If the pump does not come off the breast easily, you can break the seal by sliding your finger between the breast and the massage cushion.

**Problem:** The breast pump is not working.

**Solution:** First, check that you have assembled the breast pump correctly. To prevent interference, keep other electrical equipment, such as mobile phones or laptops, away from the breast pump during expression. Check that the battery is charged.

**Problem:** I don't feel any suction.

**Solution:** Check that you have correctly assembled the breast pump by following the steps in the "Preparation for Use" section. Make sure the valves are properly placed and check for damage or wear. Replace the valve if necessary. Also, make sure the pump is placed correctly against the breast to create a vacuum.

**Problem:** The pump has too much suction.

**Solution:** Follow the instructions below if you experience too much suction from the breast pump:

Make sure you only use parts recommended by the manufacturer.

Make sure you have assembled the breast pump with the massage cushion: not using the massage cushion may cause too much suction.

When you use the breast pump for the first time, you may feel too strong suction at first. Practising can help. If the problem persists, consult your lactation consultant.

## USAGE AND STORAGE CONDITIONS

Keep the breast pump out of direct sunlight as prolonged exposure to the sun can cause discoloration. Store the breast pump and its accessories in a safe, clean and dry place.

If the appliance has been stored in a hot or cold environment, place it in the usage environment to let it reach a temperature within the usage conditions (10°C to 30°C) before you use it.

Make sure that the voltage of the power adaptor is compatible with both the power source and the breast pump battery. Usage of a power adaptor other than recommended by the manufacturer may cause irreversible damage to your breast pump.

Use only Lorelli power adaptor or a certified USB type C power adapter with the following specifications:

INPUT: AC 100 – 240 V; 50/60 Hz

OUTPUT: 5.0V; 1000 mAh

## TECHNICAL SPECIFICATIONS

Fixed time : 30 minutes

Noise level: <60dB

Environmental conditions for operation : Temperature 10-30 °C, Relative humidity 15%-93%,

Environmental conditions for storage : Temperature -25-50 °C, Relative humidity 15%-93%,

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TECHNOLOGY CO.,LTD.
Importator:Didis Ltd.