Fidi Free

Breast pump

Laktator

Milchpumpe

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Tiralatte

Tire-lait

Extractor de leche

Borstkolf

Krūties siurblys


Odsávačka

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
Mellszívó

Pompa de san




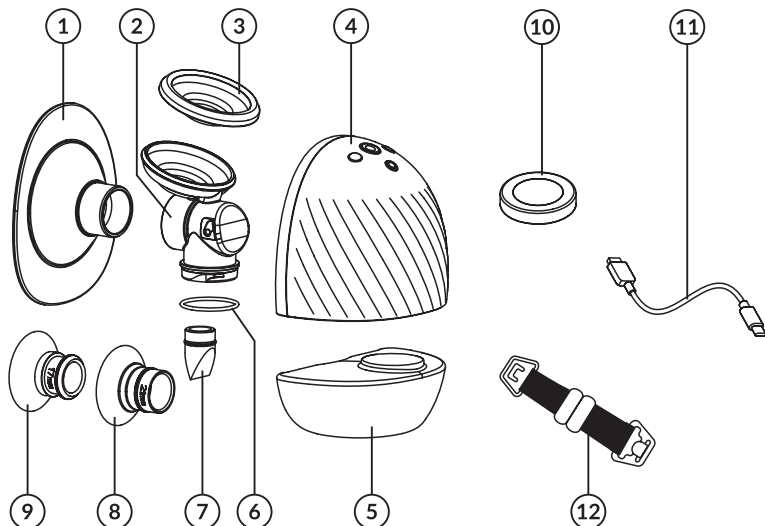






A


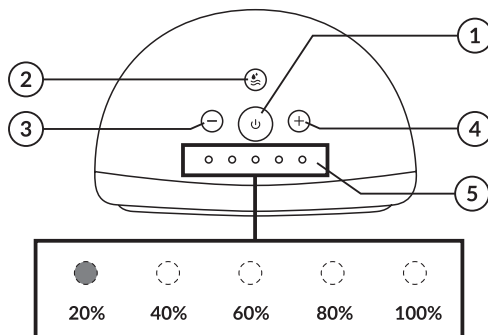
B

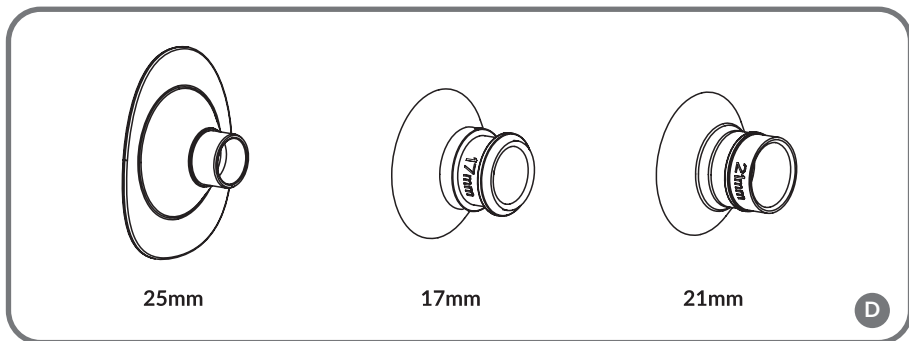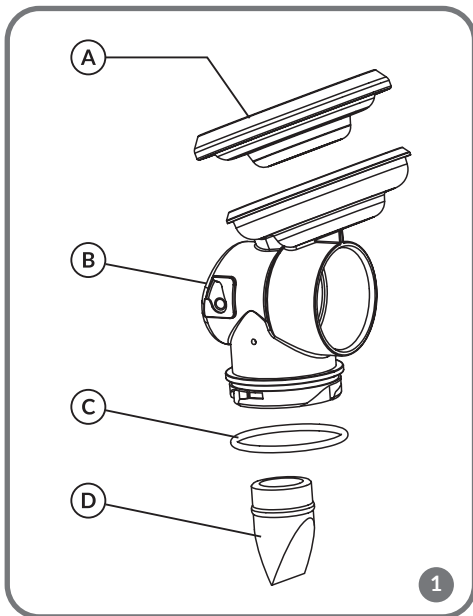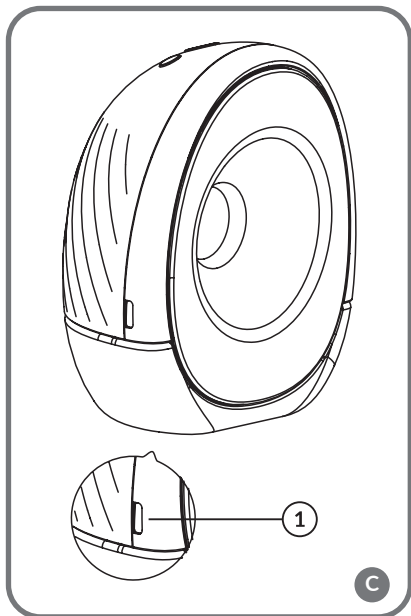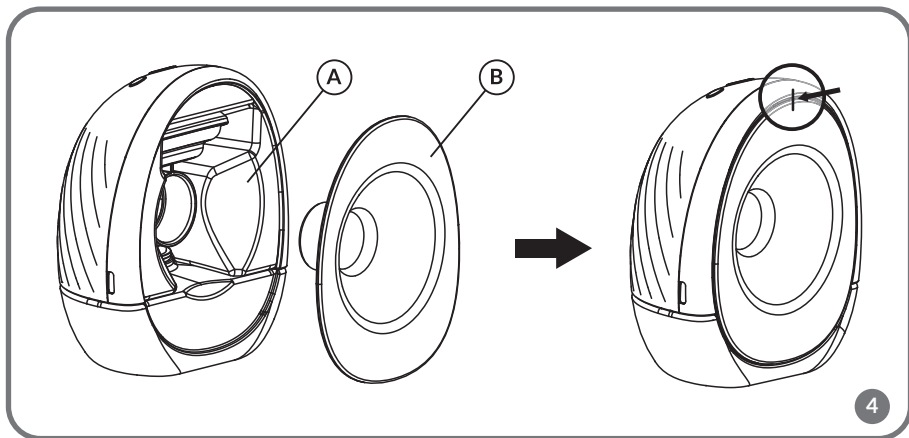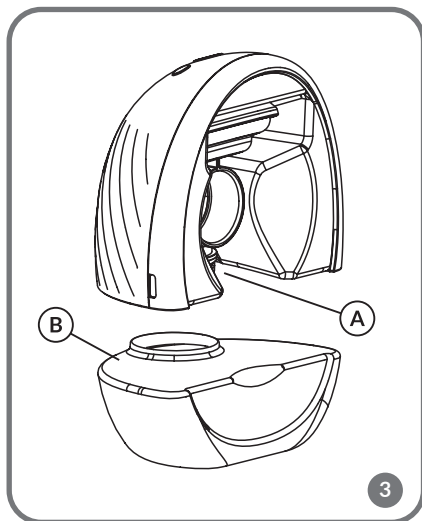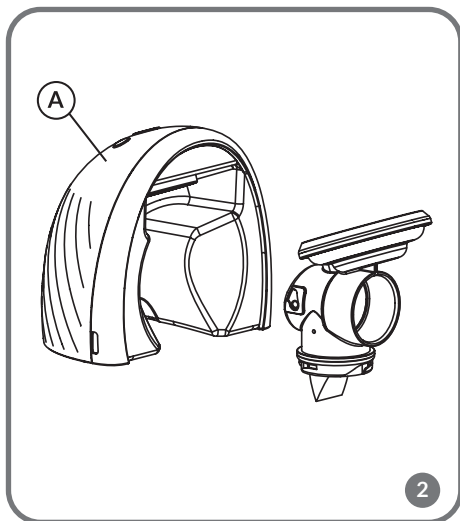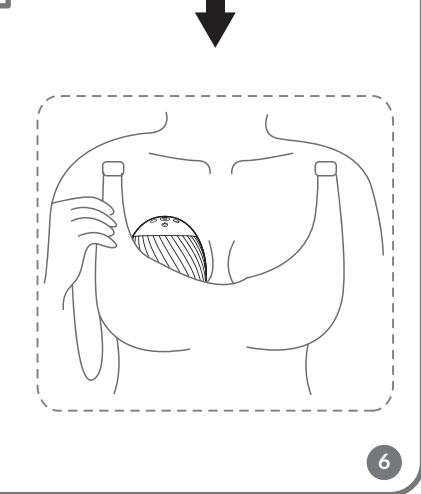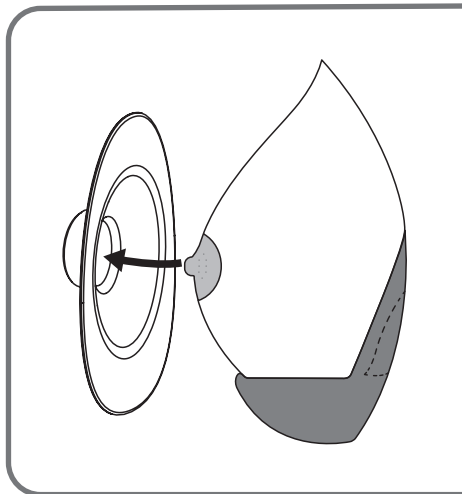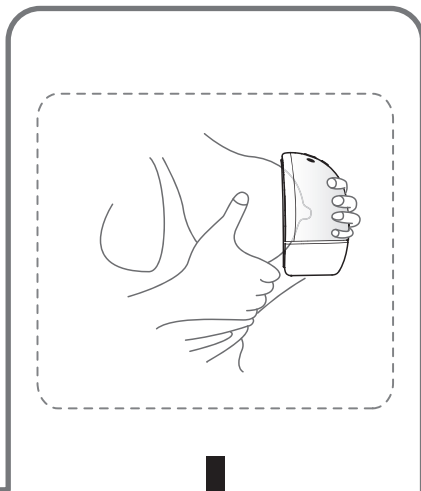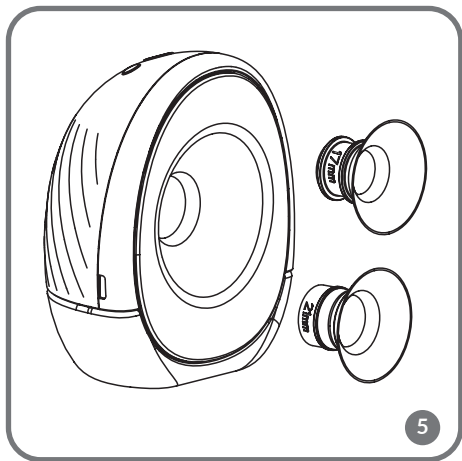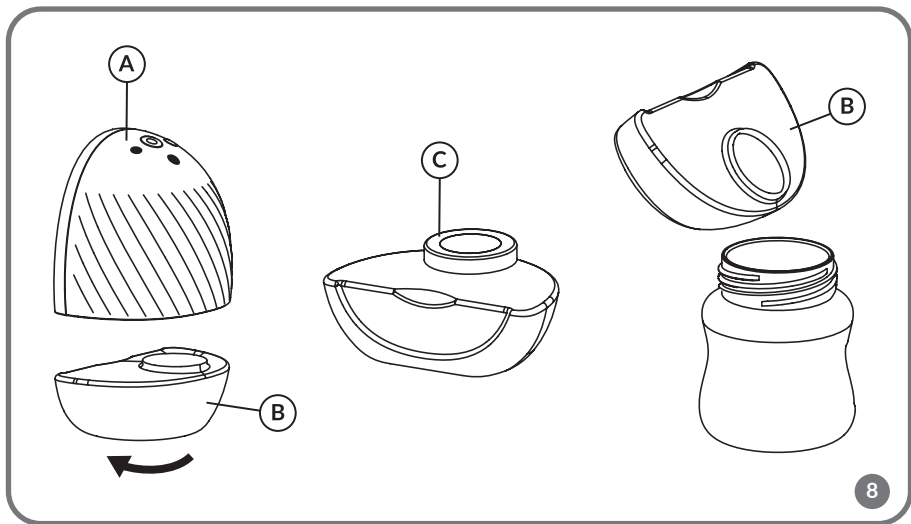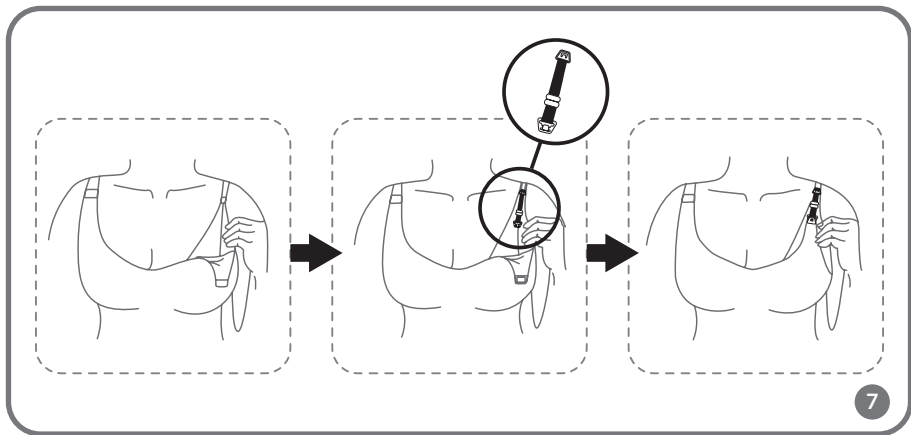Dear Customer!

In case of any questions or comments on the purchased product, please contact us: help@lionelo.com

Producer:

BrandLine Group Sp. z o. o.

A. Kręlewskiego 1, 61-248 Poznań, Poland

Device information

Operating time: up to 2 hours

Battery: Li-ion, 3.7 V / 1300 mAh

Power supply: USB-C, 5V DC 2A

Charging time: 3 hours

Operating temperature: 5-40°C

Room humidity: 10%~90% RH

Important information

Please read this instruction manual before operating the product to familiarise yourself with its functions and use it as intended.

Not following the safety instructions and use which is not in accordance with the instructions may cause damage to health.

Keep this instruction manual for future reference.

1. Do not repair or modify the device. These operations can only be performed by an authorised service centre. Use only original spare parts.
2. The device may be used by children of at least 8 years of age and persons with reduced physical, sensory or mental capabilities, as well as persons without relevant knowledge and experience in the use of the device, if they are supervised or have been previously instructed in the safe use of the device and the risks involved.
3. This device is not a toy. Do not allow children to play with it. Keep the device out of the reach of children. Cleaning and maintenance should not be performed by children without adult supervision.
4. Do not start the device if you notice any damage. This applies to both the device itself and its accessories, such as the USB cable.
5. Do not use the device if it begins to malfunction, has been dropped or submerged in water, becomes excessively hot, discolouration, bulging or other abnormal phenomena occur. Contact the manufacturer's authorised service centre.
6. Never leave a device connected to a power socket unattended.
7. The device is not resistant to high temperatures. Use the device away from heat sources, high temperatures, hot surfaces, direct sunlight, sources of spark, open flames, oils and sharp edges.

8. Do not use chemicals to clean the device. Do not wash the main motor device with water.
9. Switch off the device after each use.
10. Do not immerse the entire device in water or other liquids. Do not use near water.
11. Do not misuse the device.
12. Do not use the device or touch the power supply with wet or damp hands. Make sure all breast pump components are dry before assembling the device.
13. Before cleaning or moving the device, switch it off and disconnect it from the power source.
14. Before and after each use, wash and disinfect all parts that will adhere to the body and come into direct contact with the milk.
15. Thoroughly dry each item before use and storage.
16. For hygienic reasons, the device should be used by one person.
17. If using the breast pump causes discomfort or pain, stop using the machine immediately.
18. Do not use the device throughout your pregnancy.
19. Do not use the device when tired or asleep.
20. Special care is required when using the device near children.
21. Do not use the device while lying down or during strenuous exercise or activity that could cause injury.
22. If breast milk does not appear for more than 5 minutes while using the device, stop using the breast pump and consult your doctor.
23. If you have an implanted pacemaker or other electrical device, it is recommended that you consult your doctor before using the device.
24. When used at the highest permissible temperature (30°C) for 30 minutes, the silicone insert can reach a temperature of 45°C.
25. It is advisable to replace the components in contact with the milk every approx. 3-6 months, depending on how often the device is used, and if the device is not used for a long period of time.

The product complies with the requirements of European Union directives.

In accordance with Directive 2012/19/EU, this product is subject to separate collection.

The product should not be disposed of with municipal waste, as it may pose a risk to the environment and human health. Return the used product to a recycling centre for electrical and electronic equipment.

Parts of the device (fig. A)

- | | |
|----------------------------|------------------------|
| 1. Funnel cap (25 mm) | 7. Valve |
| 2. Connector | 8. Funnel (21 mm) |
| 3. Membrane | 9. Funnel (17 mm) |
| 4. Device body | 10. Milk container lid |
| 5. Milk container (150 ml) | 11. USB-C power cable |
| 6. Packing ring | 12. Bra clasp |

Control panel (fig. B)

1. On/off switch
2. Mode change
3. Decreasing intensity
4. Increasing intensity
5. Intensity level indicator

Charging

Note! The device has a built-in lithium-ion battery. It is recommended to use a 5V 2A power supply to charge the device. Use only the USB-C power cable (1, fig. C).

Note! It is not possible to use the device while it is charging.

The charge level of the device can be checked by pressing the increase and decrease intensity buttons (3,4, fig. B) simultaneously for approximately 2 seconds. One illuminated LED of the intensity level indicator (5, fig. B) indicates approximately 20% of the remaining battery power.

A low battery level will be signalled by a flashing left icon on the intensity level indicator (5, fig. B).

The intensity level indicator (5, fig. B) will flash during charging. When fully charged, all icons on the indicator will light up.

Before first use

Before the first use of the device, all parts in contact with the milk must be washed, disinfected and dried (see: Cleaning and disinfection).

The device is equipped with a 25 mm funnel cap. The set also includes two funnels in sizes 17 mm and 21 mm (fig. D), to allow the correct size to be fitted.

Assembly

Ensure that all parts are dry, clean and disinfected before assembling the device.

1. Attach the membrane (A, fig. 1) to the upper part of the connector (B, fig. 1) and then the packing ring (C, fig. 1) and the valve (D, fig. 1) to its lower part.
2. Place the whole into the device body (A, fig. 2). Ensure that all parts are correctly fitted and that the membrane is tightened firmly.
3. Fit the milk container (B, fig. 3) by screwing it clockwise to the bottom of the connector (A, fig. 3). Ensure that the container is fitted correctly.
4. Place the funnel cap (B, fig. 4) in the connector (A, fig. 4) and make sure it is pressed firmly and in the correct position. You can also choose between two funnel sizes of 17 or 21 mm (fig. 5) to be inserted into the funnel cap.

Note! The absence of the packing ring may weaken the performance of the device.

Fitting the device

1. Make sure that the chosen funnel fits well and that the nipple is in the tunnel (fig. 6). Food intake should not be painful. If using the breast pump causes discomfort or pain, stop using the machine immediately.

2. Make sure the device fits securely to the breast and that there is no space between the breast and the main insert. A poor fit will affect the amount of breast milk pumped.

The device can be worn with a standard nursing bra, either sitting or standing. Do not lean while using the device. Make sure the device fits securely on the body before each use. For stiff and inflexible bras, use the bra clasp provided (12, fig. A). To do this, unhook the bra strap, then attach the clasp and adjust the length according to your preference (fig. 7).

Device operation

1. To switch on the device, press and hold the switch (1, fig. B). A short press of the switch will interrupt or resume operation of the device.
2. To stop the device, press and hold the switch (1, fig. B), then gently fold back the main silicone insert and remove the device.

Note! The device will automatically switch off after approximately 30 minutes of operation.

Stopping operation of the device for more than 5 minutes will automatically enter standby mode.

Operating modes

The device has two modes – a prolactation massage mode and a food intake mode. The device automatically switches from massage mode to food intake mode after approximately 2 minutes of operation.

The mode can be changed manually by pressing and holding the mode change button (2, Fig. B).

Intensity settings

The device has 5 levels of intensity, which can be adjusted using the intensity decrease and increase buttons (3, 4, fig. B). Intensity adjustment is possible in both operating modes of the device.

The intensity level is indicated by the indicator LEDs (5, fig. B).

Note! A higher intensity level does not mean that more milk will be pumped. Make sure the chosen intensity is comfortable. If you don't feel the breast milk being pumped, make sure the breast pump is fitted correctly.

Milk storage

At the end of the session, unscrew the milk container (B, fig. 8) and separate it from the device body (A, fig. 8).

The milk can be stored in a container closed with the lid provided (C, fig. 8), or poured into another container.

Troubleshooting

1. Poor or no food intake

Possible cause	Solution
The membrane is damaged or has been incorrectly fitted.	The membrane must be fitted correctly or replaced.

Leaky connection between the membrane and the device body.	Make sure the membrane is tightly connected to the device body.
Malalignment of the main silicone insert.	Make sure the funnel size is well chosen and the nipple is in the tunnel.

2. The device does not switch on or charge

Possible cause	Solution
Check that the correct power cord is used and that it is connected correctly.	Use the power cord provided and make sure it is connected correctly.
The battery discharged.	Charge the device. The LED on the discharged device will switch on after about 2 minutes after being connected for recharging.

3. Milk leakage

Possible cause	Solution
The membrane is damaged or has been incorrectly fitted.	Stop pumping immediately, switch off and wash the device, and fit the membrane correctly or replace it.

Cleaning and disinfection

- Before first use, disinfect all components that will come into contact with milk (milk container, silicone pads, membrane, valve, ring).
- The main device should be cleaned with a dry and soft cloth.
- To remove milk residues, wash the parts in contact with the milk thoroughly under running water with a mild detergent.
- To disinfect, place all items in boiling water for 5 minutes. Make sure all elements are immersed in the water and none are touching the dry part of the container, as the elements may melt.
- Disinfection using a steam steriliser for longer than 30 seconds is not recommended.
- Leave all elements to dry completely before the next use.

NOTE

The main device and the power cord must not be disinfected or cleaned with water. Disinfection applies only to accessories in direct contact with the milk.

Be careful when cleaning, as damaged elements may cause the breast pump to not function correctly.

The photos are for illustrative purposes only, the actual appearance of the products may differ from those shown in the images.

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