# **VEVOR<sup>®</sup>**

**TOUGH TOOLS, HALF PRICE**

Technical Support and E-Warranty Certificate [www.vevor.com/support](http://www.vevor.com/support)

## **TUB TRANSFER BENCHES**

We continue to be committed to provide you tools with competitive price.

"Save Half", "Half Price" or any other similar expressions used by us only represents an estimate of savings you might benefit from buying certain tools with us compared to the major top brands and does not necessarily mean to cover all categories of tools offered by us. You are kindly reminded to verify carefully when you are placing an order with us if you are actually saving half in comparison with the top major brands.

# VEVOR<sup>®</sup>

TOUGH TOOLS, HALF PRICE

TUB TRANSFER BENCHES

MODEL:FBL630239

## NEED HELP? CONTACT US!

Have product questions? Need technical support? Please feel free to contact us:

**Technical Support and E-Warranty Certificate**  
**[www.vevor.com/support](http://www.vevor.com/support)**

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

<b>Symbol</b>	<b>Symbol Description</b>
	<b>Warning:</b> To reduce the risk of injury, the user must read the instructions manual carefully.
	This symbol, placed before a safety comment, indicates a kind of precaution, warning, or danger. Ignoring this warning may lead to an accident. To reduce the risk of injury, fire, or electrocution, please always follow the recommendation shown below.

## PART LIST


When unpacking this product, check to make sure the parts listed below are included and carefully inspect for any damage that may have occurred during transit. If any part is missing or damaged, do not attempt to assemble or use the product.

NO	PICTURE	PART NUMBER	QTY	NO	PICTURE	PART NUMBER	QTY
①		Seat frame	1	⑥		Backrest tube	2
②		Seat frame	1	⑦		Backrest	1
③		Armrest tube	1	⑧		Shower buckle	1
④		Chair foot	2	⑨			
⑤		Chair foot	2	⑩			

## SEMBLY INSTRUCTIONS

### 1. Installation Steps


1. Invert seat frame ① and seat frame ② on a flat surface, then align the pipe openings of seat frame ① and seat frame ② and insert them (ensuring that the marbles on the front and rear main frames pop out normally).

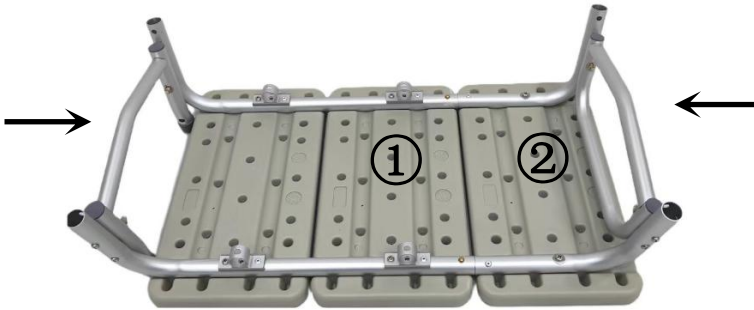
2. Insert the two chair legs ④ into the main frame tubes of the seat frame ①, and then insert the two chair legs ⑤ into the main frame tubes of the seat frame ② (ensure that the marble on each chair leg can pop out normally)


3. Place the assembled main frame directly on a flat surface, then align the nozzle of the armrest tube ③ with the armrest installation tube of the main frame ① and insert it (ensuring that both marbles of the armrest tube can pop out normally)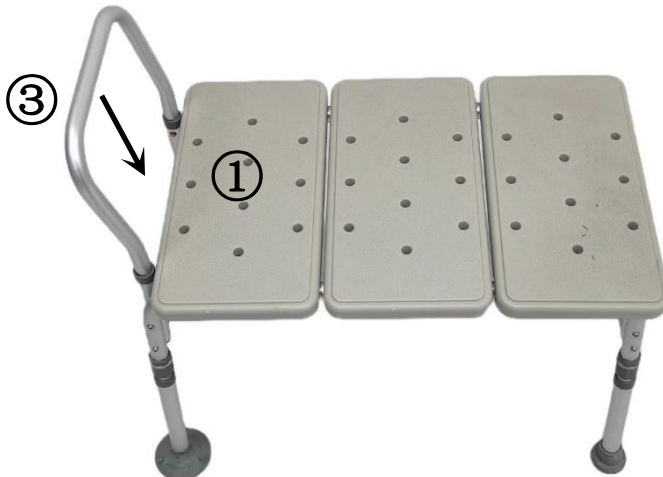

4. Insert two backrest tubes ⑥ into the two tube openings of backrest ⑦ respectively (ensure that the marble of each backrest tube can pop out normally).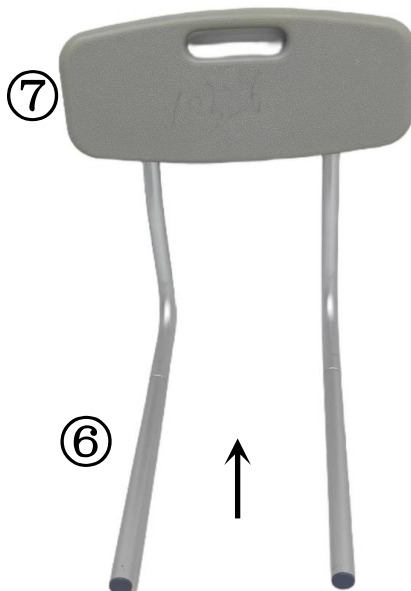

5. Align the backrest tube of the backrest with the backrest installation seat at the bottom of the seat frame and insert it (ensuring that the marble of each backrest tube can pop out normally), then clip the shower buckle onto the armrest tube.

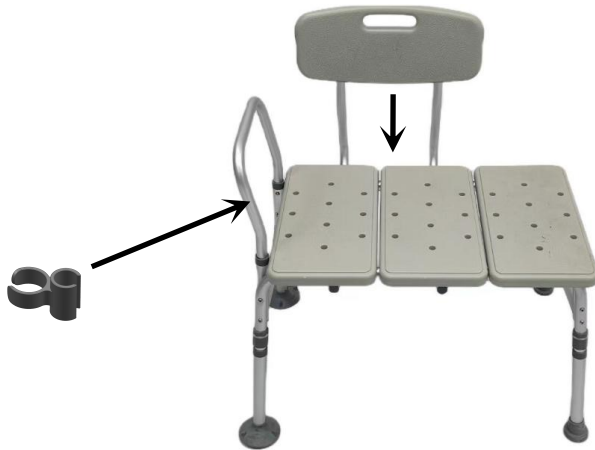#### SUBSEQUENT CONSULTATION: READ CAREFULLY

##### Warning:

- \*Put the chair stably on a flat floor
- \*Building the chairs in good condition.
- \*Stay away from open flames.
- \*Avoid the overload at the chair.

##### Precautions before assembly:

- \*Prepare a clean and dry mounting area.
- \*Verify that the product is not damaged.
- \*Check that none of the pieces are missing.

##### Maintenance:

- \*Clean with a soft fabric.
- \*Regularly remove leaves and or water collected on the canvas.
- \*Check the seating regularly.

##### Storage conditions:

\*The fabric should be stored completely dry, in a clean and dry place to avoid the risks of mold.

Caution with the risks of pinching or cuts during assembly.

The manufacturer declines all responsibility in case of damages due to a bad installation.

### Weight limited

Manufacturer: Zhongshan Bliss Medical Instrument Co., Ltd  
Address: 2F-2, 3-6F & 7F-1, No. 63, Tongxing East Road, Yumin Community, Xiaolan, Zhongshan, Guangdong, 528414, CHINA

**Imported to AUS:** SIHAO PTY LTD. 1 ROKEVA STREET EASTWOOD NSW 2122 Australia

**Imported to USA:** Sanven Technology Ltd. Suite 250, 9166 Anaheim Place, Rancho Cucamonga, CA 91730

<b>UK</b>	<b>REP</b>
-----------	------------

YH CONSULTING LIMITED.  
C/O YH Consulting Limited Office 147, Centurion House, London Road, Staines-upon-Thames, Surrey, TW18 4AX

<b>EC</b>	<b>REP</b>
-----------	------------

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**VEVOR<sup>®</sup>**

**TOUGH TOOLS, HALF PRICE**

**Technical Support and E-Warranty Certificate**

**[www.vevor.com/support](http://www.vevor.com/support)**

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Fabricante: Zhongshan Bliss Medical Instrument Co., Ltd
Importado a AUS: SIHAO PTY LTD. 1 ROKEVA CALLE ASTWOOD
Importado a EE. UU.: Sanven Technology Ltd. Suite 250, 9166 Anaheim
C/O YH Consulting Limited Oficina 147, Centurión
E-CrossStu GmbH