INSPT0337/04.2024/V2

PT 0337

ORTEZA

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YOUNG

- PL

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INSTRUCTIONS FOR USE
- SK

NÁVOD NA POUŽITIE
- RO

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EN

PT 0337 Wrist orthosis with a thumb brace YOUNG

INSTRUCTIONS FOR USE

INTENDED USE:

A wrist orthosis with a thumb brace YOUNG is a reusable medical device intended to support the treatment and rehabilitation process according to indications.

INDICATIONS:

- overload changes
- post-trauma thumb stabilisation
- post-trauma wrist suport
- post-surgery care
- rehabilitation following removal of a plaster cast

STORAGE:

Store the product in a dry place. Protect it from high temperatures and direct sunlight.

TARGET GROUP:

Device designed for persons showing the symptoms described in the indications.

PRECAUTIONS:

The product should be used only on intact skin. Do not apply directly to wounds, damaged or pathologically affected skin. The use of ointments, oils, creams and similar substances may adversely affect the properties of the product, reduce its durability and diminish its effectiveness. In the case of persistent or intensified pain, allergic reaction, skin irritation or any other adverse effects related to the use of the product, discontinue use and consult a doctor. The appropriate use of the product may require the specialist medical staff advice. For hygienic reasons and for the sake of effectiveness, the product is intended for use only by one patient. Adjustment of the correct size determines the proper functioning of the device. Each use of the product should be preceded by verification of the signs of wear or aging. Only undamaged product guarantees optimal performance. Do not use the device continuously for 24 hours unless your doctor advised otherwise.

INFORMATION ON INITIAL PREPARATION OF THE PRODUCT:

It is recommended to wash the product before first use.

MAINTENANCE:

Close the fasteners before washing. Take out the palm rest and the steel shank.

INSTRUCTIONS FOR USE:

Open the fasteners, removing the stabilising straps from the rings. Put on the orthosis ensuring the thumb remains in a natural anatomical position, and the stabilising straps and rings for fastening the stabilising straps are located on the dorsal side. The stabilising steel shank should be located on the thumb side, while the shaped splint should be on the palmar side. Fasten the strap stabilising the thumb, then the straps stabilising the wrist. Adjust the peripheral straps, so that they ensure stabilisation (well-fitted stabilising splints), without restricting the movement of fingers, free blood circulation in the hand, or causing the fingers to turn blue.

COMPOSITION PT0337:

Polyamide 55%, Polyester 39%, Elastane 6%

Any serious incident related to the product must be reported to the manufacturer and competent authority of the Member State where the user or patient is resident.

PRODUCT ADJUSTMENT:

Size chart is available on the packaging.

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SIGVARIS S.A.