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INSTRUCTIONS FOR USE

Issue number and date: 1 / 11.2024

Rectangular shower stool with height adjustment

Model KP100AM

Introduction

Thank you for choosing our product. We are confident that it will meet your expectations. We encourage you to read this document before using the product for the first time. These instructions are intended to provide the necessary information on the safe use of the medical device. The manual provides detailed guidance on the device's functions and capabilities. Retaining the document allows this information to be easily found in the future, facilitating correct use.

Description of the medical device

The rectangular shower stool (seat width - 49 cm) with height adjustment is an auxiliary product that supports users with limited mobility in performing hygiene-related activities. Made of PVC (polyvinyl chloride), the stool is fitted with height-adjustable aluminium legs. Rubber foot tips are designed with safety in mind, effectively eliminating the risk of slipping on wet surfaces.

Intended use

The medical device is intended to facilitate the performance of hygiene-related at home and in hospital, especially for people with reduced mobility. It can be used both in the shower and in the bath, providing stability and comfort while bathing.


Contraindications

People with balance problems, coordination disorders, spinal instability or contraindications to sitting should avoid using the product due to the risk of falling and aggravating their condition.


Set composition

Item number	Item description	Quantity
1	Outer tube	1 pc.
2	Inner tube	1 pc.
3	Telescopic tube	4 pcs.
4	M6x35 screw	5 pcs.
5	Washer	5 pcs.
6	Spring pin	4 pcs.
7	Seat	1 pc.


Assembly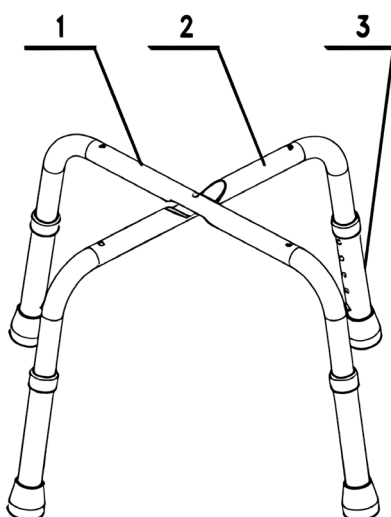


Step 1: Connect the outer tube to the inner tube as shown above.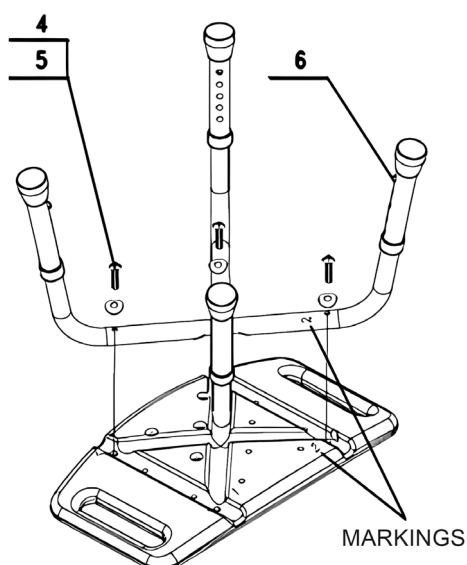

Step 2: Then arrange the seat as shown above and join the pieces together. Tighten all screws using lock washers.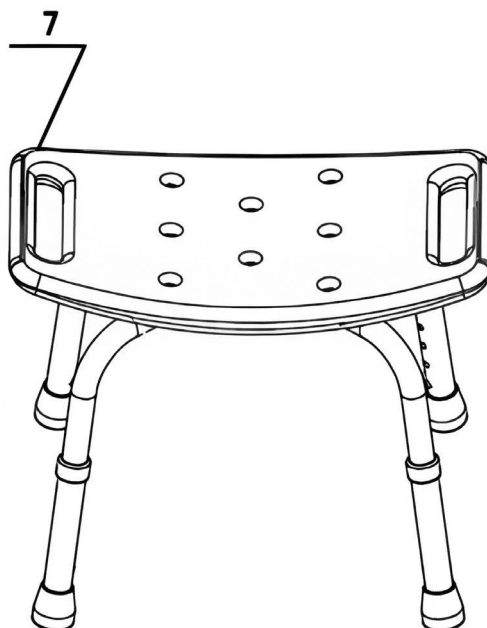

Step 3: After assembling the components, place the stool on a flat surface. Use the supplied spring pins to adjust the height of the telescopic tubes. The stool should rest firmly on the ground after correct assembly. Periodically check the condition of the spring pins.

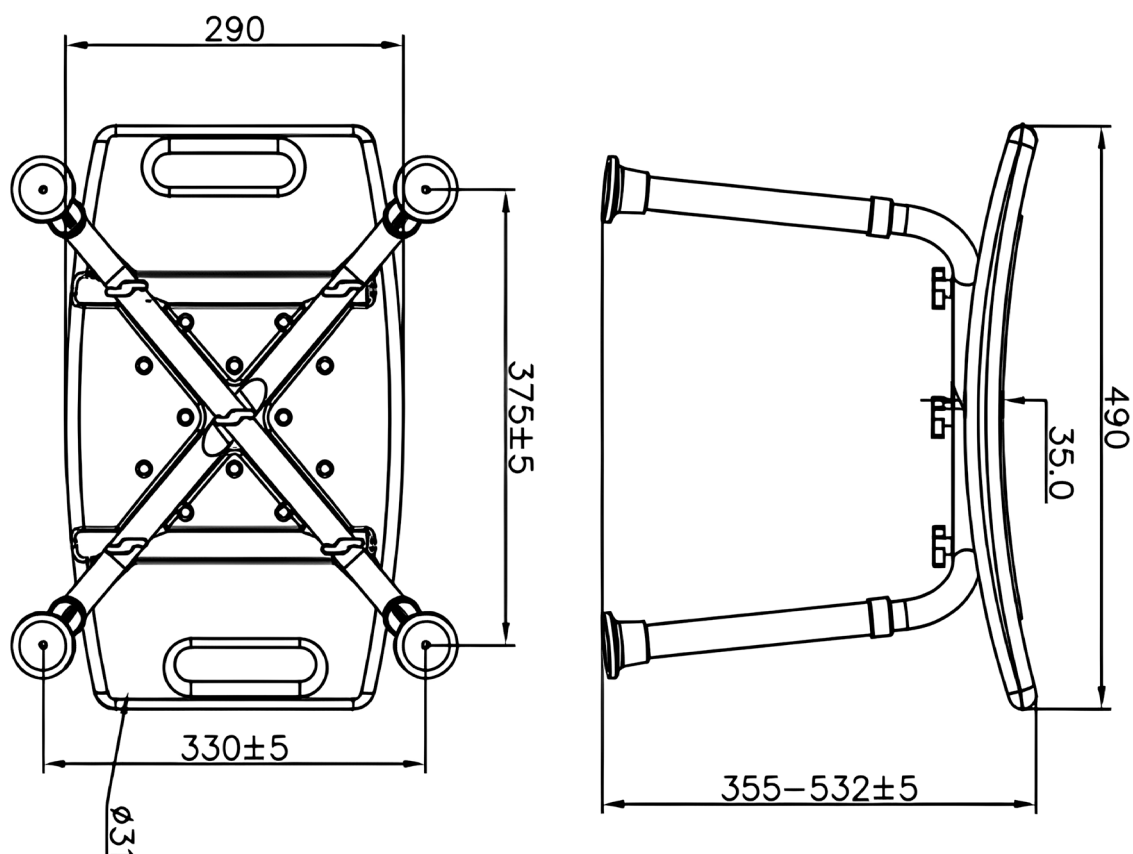Dimensions of the assembled product in millimetres (mm)

The user or their carer should consult a healthcare professional if they are unsure whether the medical device is appropriate for the patient in question.

Maximum permissible user weight – 110kg.

Any use other than for the intended purpose is considered inappropriate and DANGEROUS. The manufacturer and/or distributor shall not be liable for damage caused by improper, incorrect and/or unreasonable use.

Safety and usage rules

1. The stool is designed for use on flat and level surfaces.
2. Using a stool on uneven ground that does not provide stability can lead to tipping over and injury.
3. Standing on a stool is prohibited.
4. The maximum permissible load for the stool is 110kg.
5. Do not use the stool if you notice any visible or perceptible damage (e.g. cracks in the frame), as this could lead to an accident.

Maintenance

- Regular cleaning of the stool should be carried out with water, a sponge and mild detergents designed for plastics, such as soap or mild cleaners approved by the PZH. It is important to dry the stool thoroughly after each use.
- Avoid the use of: strong detergents, abrasives, bleach, gasoline and any solvents, as these can cause damage to the frame, visible defects and lead to a loss of warranty.
- It is a good idea to check the condition of structural components at least once every three months, including fixing bolts, as well as pipes for any cracks. In addition, it is advisable to inspect the rubber cap regularly; if it is found to be worn, it should be replaced with a new one.

Storage and transport conditions

The stool should be stored in a dry and well-ventilated place, protected from moisture and extreme temperatures to prevent damage to the materials. Contact with chemicals that may affect its durability and performance should be avoided. During transport, the stool should be secured in such a way as to prevent movement and damage. Use appropriate protective packaging.

Disposal

Disposal of the product must be carried out in accordance with the applicable local environmental regulations.

Product life

The expected product life is up to 5 years if used in accordance with the information in the instructions for use. The product can be used after this period if it is undamaged and meets the specified requirements for safe use.

Warranty

- The product is covered by a 24-month warranty from the date of purchase, against any defects in material or workmanship.
- The warranty covers the free replacement and/or repair of parts found to be manufacturing defects.
- The warranty does not cover parts subject to normal wear and tear.
- The warranty is no longer valid if the product has been tampered with/modified, if the defect is the result of misuse or if the damage cannot be attributed to the manufacturer.

NOTE

Any serious incident involving the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is residing.

Product content: PCV, aluminium, synthetic rubber, steel. **Colour:** White/grey. **Product weight:** 2.2 kg.

Explanation of symbols

	Conformity of the device with EU Regulation 2017/745 on medical devices		Medical device
	Take a look at the instructions for use		Maximum user weight
	Model		Batch/series number
	Date of production		Warning
	Unique identification code of the device		Medical device manufacturer

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ACTION S.A.