
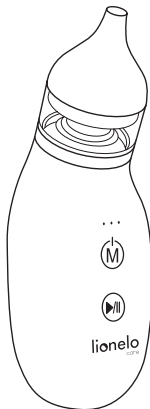
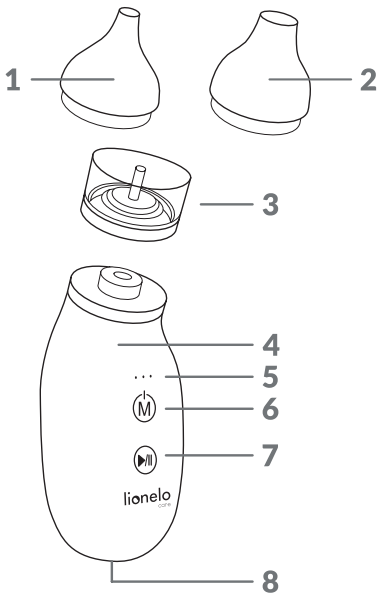
BabyAir

Baby nasal aspirator
Aspirator do nosa dla dzieci
Nasensauger für Babys
Детский назальный аспиратор
Aspiratore nasale per bambini
Aspirateur nasal pour bébé
Aspirador nasal para bebés
Baby neusaspirator
Kūdikiu nosies aspiratorius
Dětská nosní odsávačka
Baba orrszívó
Aspirator nazal pentru bebeluși
Nässug för bebisar
Baby nesesug
Baby næseaspirator
Vauvan nenäimuri

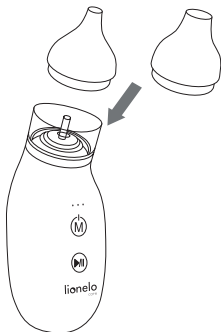
A

1

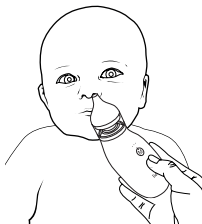2

Dear Customer!

In case of any questions or comments on the purchased product, please contact us: **help@lionelo.com**

Producer:

BrandLine Group Sp. z o. o.

A. Kręglewskiego 1, 61-248 Poznań, Poland

IMPORTANT INFORMATION

1. Before using the product for the first time, please read the user manual carefully and keep it for future reference.
2. This device may be used by persons with reduced physical, sensory or mental abilities, as well as by persons without relevant knowledge and experience in the use of the device, if they are supervised or have been previously instructed as to the safe use of the device and the risks involved.
3. The product is only intended for domestic use and is not intended for medical or commercial purposes.
4. The product is a hygienic device designed for suctioning secretions and mucus to cleanse the nasal cavity of children from birth to 12 years of age, intended for home use.
5. If the child begins to feel unwell while using the device, stop using the device immediately and seek medical advice.
6. Do not use the product for purposes other than intended. The manufacturer bears no liability for damage caused by inappropriate use of the product.
7. This product is not a toy. Do not allow children to play with the device. Keep the device out of the reach of children.
8. Limit the use of the product to only a few seconds at a time in uninterrupted mode.

9. Clean the aspirator tips and secretion tank after each use.
10. To prevent pressure imbalances in the eardrums caused by vacuum pressure, infants and young children should close their mouth or swallow saliva while using this product.
11. Insert the aspirator tip into the nasal cavity gently, without applying excessive force.
12. Use a power adapter with the recommended specifications: 5.0V DC 1.0A.

Contraindications

- Nosebleed
- Acute head or neck injury or trauma
- Coagulopathy or haemorrhagic disorders

Before using the device, please read the user manual carefully. Incorrect use of this product can pose health hazards and lead to the risks/complications listed below:

- Mechanical injury
- Nosebleed or internal bleeding
- Mucosal bleeding
- Discomfort and pain

CHARGE LEVEL

Indicator light	Battery charge level
Green	31–100%
Orange	11–30%
Red	4–10%
Flashing red	1–3%

Important! Do not charge the device continuously for more than 24 hours, as

this may damage the battery.

In sleep mode, the device switches off automatically after approximately 1 minute.

DEVICE DESCRIPTION (FIG. A)

1. Aspirator tip (for deep nasal suctioning)
2. Aspirator tip (for shallow nasal suctioning)
3. Secretion tank x2
4. Main unit
5. Start/pause button
6. On/off switch / Mode change / Charge level indicator light
7. Indicator light of suction force level
8. USB-C port

MOUNTING

Attach the secretion tank to the main unit (Fig. 1) and then place the aspirator tip of your choice (for deep or shallow suctioning) onto the tank.

OPERATING INSTRUCTIONS

1. To start the device, press and hold the switch (6, Fig. A).
2. Press again to set the required suction force. The indicator light (7, Fig. A) shows the selected suction force level (1 to 3).
3. Place the aspirator tip gently at the entrance of the baby's nostril (Fig. 2).

Important! The child should remain seated while using the device.

4. Press and hold the start button (5, Fig. A) to start using the device.

Important! The secretion may not appear immediately after the device is started.

CLEANING AND MAINTENANCE

- Gently remove the aspirator tip and unscrew the secretion tank.
- Wash the aspirator tip and secretion tank in warm water with a mild detergent.
- If necessary, clean the main unit with a damp cloth. Never submerge the

main unit in water or any other liquids.

- Before reassembling the device, check to make sure that all parts are clean and dry.
- The aspirator tips and the secretion tank can be sterilised by steam or UV radiation.

The photos are for illustrative purposes only, the actual appearance of the products may differ from that shown in the photos.

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