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4-Wheel Aluminium Rollator

Wersja/Version: GRASS

Typ/Type: VCBK

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English version on next page

Users with visual impairments can find the PDF files together with further information on our website:

<www.viteacare.com>.

☞ Contact your specialist dealer when required.

Alternatively users with visual impairments can have the documentation read out by a helper.

Table of contents

Introduction	21
Indications.....	22
Contraindications	22
Product description.....	22
List of elements in the package.....	23
Usage	23
Assembly and dismounting the rollator	23
Braking system	24
Adjustment of braking.....	25
Height adjustment of handles	25
Folding.....	26
Elbow crutches attachment.....	26
Overcoming obstacles (thresholds, kerbs)	26
Safety instructions	26
Technical specifications	28
Service and repair.....	29
Care and maintenance	29
Cleaning.....	29
Disinfection.....	29
Maintenance plan.....	30
Life span.....	30
Product disposal	30
Symbols	31
Contact details	33

Introduction

Four wheel aluminium rollator equipped with two fixed rear wheels with brake and locking the wheels and the two front wheels swivel. It lets you easily navigate through any surface, inside and outside the building. Please ensure that you complete the attached warranty form and return it to your distributor so that we can provide you with the best possible service.

ATTENTION

Always observe the basic precautions according to the information given below regarding safety when using the product. Read the entire manual carefully before use. Please keep these instructions for future use.

WARNING!

Improper use can lead to death, serious injury, dangerous conditions or product malfunction.

ATTENTION!

Improper use may lead to personal injury and/or damage to the product.

SUGGESTIONS!

Follow the instructions below to keep the product in good working order.

If the device is used by another person, a copy of these instructions for use must be provided.

These instructions for use apply to the design and specification of the product at the time of publication. Due to design changes, some figures and photos in the manual may not correspond to the product purchased. We reserve the right to make design changes.

The manufacturer disclaims any liability for personal injury or property damage that may result from improper or unsafe use of this product.

Indications

Rehabilitation supports are an ancillary equipment to facilitate tilting and moving of patients with different types of motor disorders. They can be used as an aid for patients after surgery (lower limbs) who are unable to use crutches, showing no coordination disorders, patients with mild dysfunction of the lower limbs or weakness. They are used in patients with impaired balance preventing them from independent vertical position and locomotion, as well as various types of circulatory dysfunction (e.g. intermittent claudication). Supports can also be used as an aid for geriatric patients.

Contraindications

Contraindications to upright standing, upper limb injuries and dysfunctions preventing support, coordination disorders.

Product description

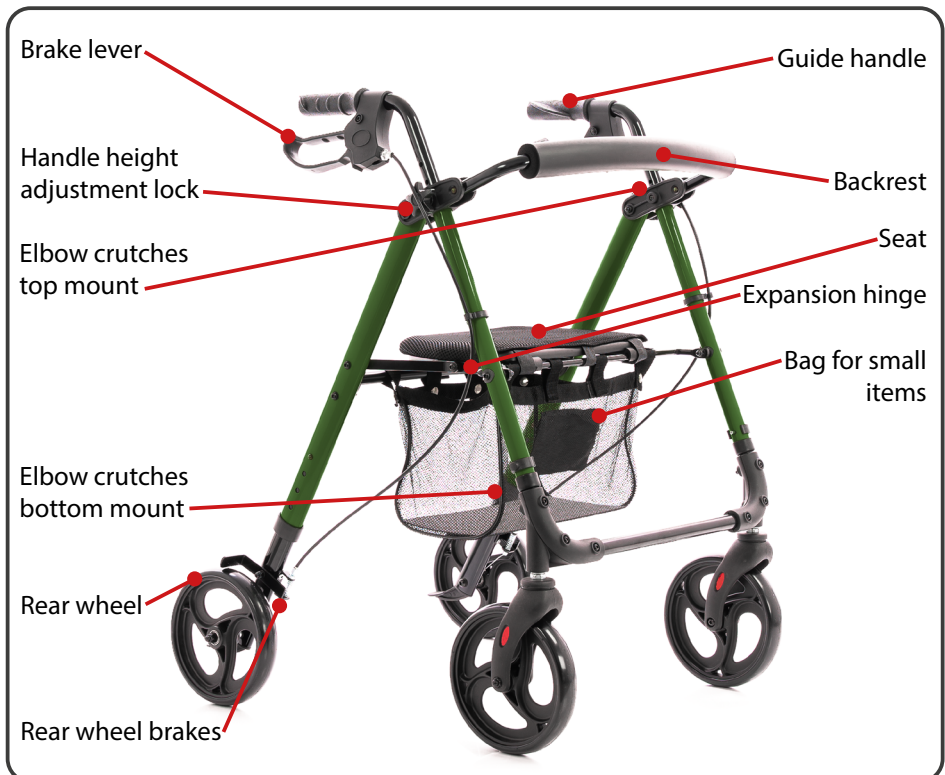
Illustration 1.

List of elements in the package

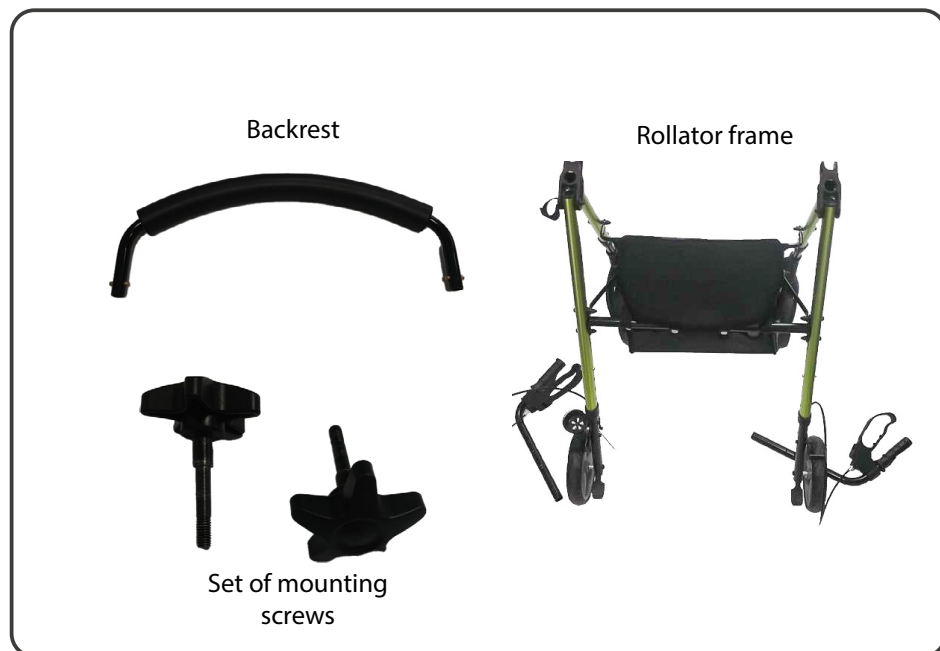
Illustration 2. Elements that should be included in the package.

Usage

Assembly and dismounting the rollator

- The product is packaged for assembly.
- When removed from the packaging, check that it contains all the parts as listed above.
- Ensure that all components have no visible mechanical damage. If in doubt, contact your distributor.
- Do not use the support until all parts are properly connected.
- Assembly - please refer to Figure 1.
- Start unfolding the support by pulling the frame apart, then lock the horizontal expansion hinges by pressing down on the bottom of the small item bag.

Illustration 3. Rollator frame.

- Fit the handles into the slots in the side frames (Illustration 4.).
- Adjust the appropriate height of the handles, the same for each handle.
- Insert a knob with a screw into the mounting hole on the outside of the side frame and then tighten it.
- Make sure that the handle is securely fastened in the support frame.

Illustration 4. Handle assembly.

- The backrest must be assembled in the brackets provided for it on the side frame of the support (the latches on the backrest must snap into the slots provided in the brackets).
- Once the backrest is mounted, pull it to make sure it is properly attached.

Illustration 5. Backrest assembly.

Braking system

- The support is equipped with a multifunctional brake. The brake handle is located under the handle. It can operate as a standard brake and as a parking brake.
- To use the brake as standard, pull the brake handle upwards.
- To use the parking brake, push the brake handle down until you can hear the characteristic sound of the brake being applied (use the right and left brake at the same time). This will cause the rear wheels to lock.
- To unlock the wheels, pull the brake handle upwards.

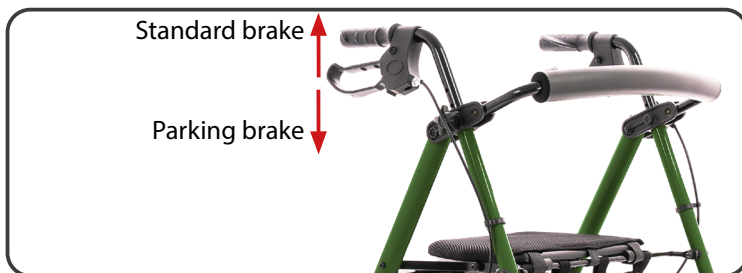
Illustration 6. Brake operation.

Adjustment of braking

- To adjust the brake force, first loosen the nut located above the brake adjusting screw just above the wheel (Illustration 7.).
- Then turn the adjusting screw counterclockwise until the correct brake force is reached.
- Once the brake force is adjusted, tighten the nut.
- After adjusting the brake force it is important to check the function of the brake system. Squeeze the brake handle to check that the brake blocks fit tightly against the wheels and are able to stop the support, then pull the handles down to apply the parking brake and check that it is working properly.

Illustration 7. Brake adjustment.

Height adjustment of handles

- The handles should be at a height that provides comfort and a sense of security for the user. It is important to make sure that both handles are set at the same height.
- To change the height of the handles, unscrew the knobs for securing the handles, remove the washers, remove the screws and slide the handle frames out or in as required.
- Once you have adjusted the correct height, tighten the knob (Illustration 8.).
- Ensure that the knob is properly tightened to prevent the handles from moving.

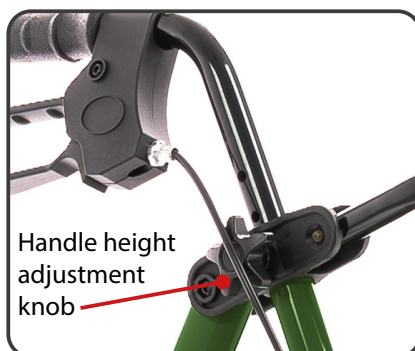
Illustration 8. Handle/grip locking screws.

**WARNING!**

Special care must be taken when folding the rollator as there is a risk of jamming fingers or hands.

Folding

- To fold the rollator for ease of transport, pull the horizontal expansion hinge upwards using a small amount of force.

Elbow crutches attachment

The diameter of the crutch/cane must not be larger than the diameter of the top handle.

- On the right side of the frame there are two crutches holders: a lower and an upper one.
- Place the crutch in the bottom bracket and then push into the top handle
- Ensure that the crutch/stick is firmly seated in the holder and that it does not fall out of the holder when moving.

Overcoming obstacles (thresholds, kerbs)

- To drive the rollator over a threshold or kerb, bring to the edge of the obstacle, holding the handles, tilt back which will lift the front wheels up.
- After the front wheels have been placed on the obstacle, bring the rear wheels to the edge of the obstacle, then, holding the handles of the rollator, lift up slightly.

Safety instructions**WARNING!**

Any 'serious incident' relating to a device must be reported to the manufacturer and to the competent authority of the Member State in which the user is resident.

**ATTENTION!**

The product is intended for people with disabilities and musculoskeletal conditions. If you are unsure whether your health condition allows you to use the product, consult a healthcare professional.

**WARNING!**

The product must not be modified in any way; this will void the warranty.

**WARNING!**

Exceeding the user's maximum weight will void the warranty. The manufacturer is not responsible for injury or damage caused by failure to comply with the maximum user weight.

**WARNING!**

Please read these instructions for use before using the rollator and keep them for future reference.

- a. At standstill and the use of seat should apply the locking rotation of the wheels both rear wheels.
- b. The stand will not replace the wheelchair, the seat is used only for temporary resting.
- c. The rollator is used only to provide rest to one user. Do not stand on it..
- d. The rollator should be used on even surfaces at an incline less than 5%.
- e. With the rollator you can only use on surfaces where all four wheels touching the ground are to use all of four wheels.
- f. Use of the product on the icy, wet, greasy, strongly gritty surfaces is unacceptable - it creates a risk of falling.
- g. Before each use of this product:
 - Perform visual inspection of all structural components of the product,
 - Check the operation of the braking system. Squeeze the brake handles to see if the brake pads are fit properly to wheels and are able to stop rollator. Press the handles down to activate the parking brake and check if it works properly.
 - Carefully check the stability and condition of the support frame.
- h. Do not use the support in the event of any visible or perceptible damage (e.g. cracks in the frame). It may cause an accident.
- i. Maximum weight that can be moved in the bag - 5kg.
- j. Do not leave the rollator with the user close to open sources of fire.
- k. Be aware that the surface temperature of the upholstery and the frame parts may be increased after exposure to the sun or other heat source. Furthermore, with low temperatures there is a danger of overcooling metal parts of the product. It may cause burns or frostbite parts of the body.
- l. All repairs must be carried out by qualified personnel. And during the warranty period only by the manufacturer's service.
- m. Do not exceed the maximum height of the handles to push, rollator will lose stability.
- n. Push the rollator with both hands simultaneously in the direction of travel.
- o. Before using, make sure that all screws are tight.
- p. Make sure that the lock used to folding and unfolding that product work properly.
- q. Due to the possibility of jamming the fingers put extreme caution in the area of horizontal hinge strut and side frame hinge.

WARNING!

Maximum user weight is 136 kg

Technical specifications

Maximum user weight	136 kg
Maximum bag load	5 kg
Maximum support height	995 mm
Height of guide handles	830-995 mm
Support width	650 mm
Length of the rollator	690 mm
Seat height adjustment	525-600 mm
Seat width	370 mm
Seat length	240 mm
Length of the rollator when folded	345 mm
Weight of the rollator	8,69 kg
Minimum turning radius	80°
Diameter of wheels	205 mm
Type of wheels	Full
Maximum safe slope	5% (2,86°)
Frame material	Aluminum
Description of the environment in which the product will be used	  
Storage and usage temperature	From +5°C to +40°C
Air humidity for storage and use	From 30% to 70 %

Service and repair

The inspection should be carried out regularly and should be preceded by a thorough cleaning of the product (so as to reveal any hidden damage). If repairs are needed or if you have any questions about the product, please contact your distributor.

Care and maintenance

Cleaning

- Clean the support frame with a damp cloth, the seat with mild detergents.
- Remove dirt from moving rollator components (wheels, bolts, clamp locks) on an ongoing basis.
- In order to maintain the performance of the product, it is extremely important to keep all its components clean and store in appropriate conditions. Storage in rooms with very high humidity (baths, laundries, etc.), or in the open air, combined with a lack of hygiene, will result in a rapid loss of performance and aesthetics.
- Do not expose the rollator to extreme weather conditions (rain, snow very strong sun), as certain components may change colour.

Must not be used:

- solvents
- toilet cleaners
- sharp brushes and hard objects
- chlorine-containing agents
- corrosive cleaning agents

Do not clean the rollator with pressure or steam unit!

Disinfection

If the product is used by a large number of people (e.g. in a nursing home), commercially available disinfectants should be used. Prior to disinfection, the top surface should be cleaned. Only tested and approved disinfectants may be used for disinfection.

Information on recognised and approved disinfectants and disinfection methods can be obtained from the national health authority.

WARNING!

The use of disinfectants can damage the outer surfaces, which can reduce the long-term functionality of components. Follow the instructions provided by manufacturer of the chemical product.

Maintenance plan

Frequency	Subject of the inspection	Comments
Before the first installation	Checking the condition of all components in the package	Check that no components have visible mechanical damage
Before each use	Wheel and brake inspection	Brakes can wear out, check the condition of the brakes before each use. Wheels should be checked for wear and cleaned of dust and sand
Every two weeks	Screws and nuts - making sure they are securely fastened	Ensure that all screws and nuts are tightened securely
Every two months	Cleanliness, general condition	In the event of mechanical damage, contact the manufacturer immediately
Once a month	Wheel axles	Wheel axles must be maintained, by spraying a few drops of preservative-lubricant between the axle and the hub

Life span


The expected average life span of a medical device is 4 years, provided it is used as intended and all maintenance and servicing. The lifespan of a medical device also depends on the frequency of use, the environment in which it is used and its care. By using spare parts, the life of the device can be extended. Spare parts are generally available for a further 5 years after a model is discontinued.

The indicated lifespan does not constitute additional guarantee.

Product disposal

The product must be disposed of in accordance with current local and national regulations. For proper disposal of packaging, metal components, plastic components, neoprene, silicone and polyurethane materials, contact your local waste disposal company or authorised distributor.

Symbols

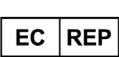	Medical device
	Production date (printed on the product)
	Manufacturer
	Distributor
	Authorised representative in the EU
	Conformité Européene (conformity with EU requirements)
	Catalogue number (to be reordered)
	Serial number
	LOT number
	Unique identification code
	Refer to the instructions for use
	Warning
	Possibility of jammed fingers
	Permitted temperature

	Protect from moisture
	For indoor/outdoor use
	Keep away from sunlight
	Maximum safe inclination
	The product is designed to be transported by car
	The product is not intended to be used as a seat for transport in a motor vehicle
	Maximum user weight

Contact details

Manufacturer

mdh Sp. z o.o.

Address: ul. Maratońska 104, 94-007 Łódź, Poland

tel. +48 42 674 83 84, fax. +48 42 636 52 21

www.mdh.pl

www.viteacare.com

We reserve the right to make technical changes due to technical progress.

viteacare.com